

# BIOSIM APP CONNECTION GUIDE

---

## Biosim app makes programming quick and easy



Allows you to set up and personalise your user's i-Limb / i-Digits




Train and optimise your user's muscle signals on the myograph



Update i-Limb / i-Digits to the latest firmware


### 1 Download the Biosim App from the Apple App Store

Please note you need to be connected to the internet to sign up to Biosim App

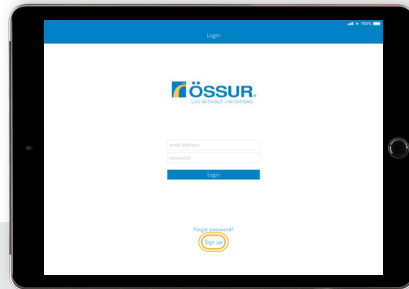


### 2 Click Sign Up

- Enter email address and create password
- Accept terms and conditions
- Verify your account via link on activation email which you will receive

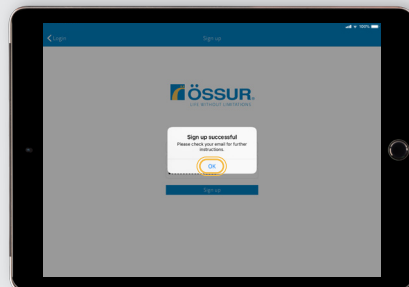

### 3 Connect to i-Limb / i-Digits: Tap on the required serial number

- Note the letter prefix identifies the device type
- Serial number can be found on the wrist area of i-Limb (or within the chassis if XS or flexion wrist) and on the wristband of i-Digits

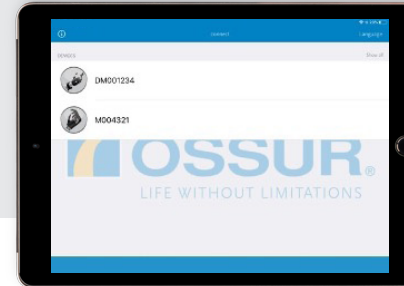

### 4 Insert last 4 digits of serial number



## TROUBLESHOOTING

- Ensure you have registered as a clinician
- Ensure you have activated Biosim account via email sent to you (check spam folder if not received)
- Ensure the prosthesis is powered on
- Ensure Bluetooth of Apple device is turned on
- Fully close down Biosim App, power cycle prosthesis and try again



**Biosim app makes programming quick and easy**



Allows you to set up and personalise your user's i-Limb / i-Digits




Train and optimise your user's muscle signals on the myograph



Update i-Limb / i-Digits to the latest firmware


## 1 Download the Biosim App from the Apple App Store

Please note you need to be connected to the internet to sign up to Biosim App

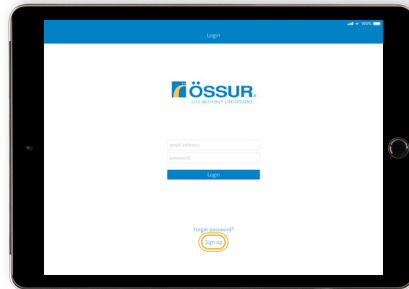


## 2 Click Sign Up

- Enter email address and create password
- Accept terms and conditions
- Verify your account via link on activation email which you will receive

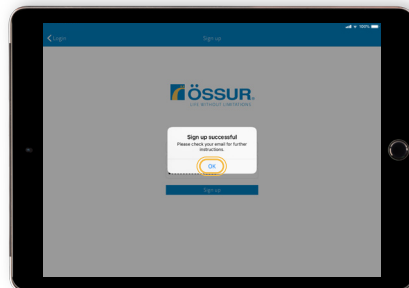

## 3 Connect to i-Limb / i-Digits: Tap on the required serial number

- Note the letter prefix identifies the device type
- Serial number can be found on the wrist area of i-Limb (or within the chassis if XS or flexion wrist) and on the wristband of i-Digits

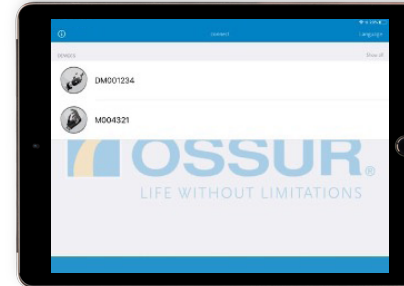

## 4 Insert last 4 digits of serial number



## TROUBLESHOOTING

- Ensure you have registered as a clinician
- Ensure you have activated Biosim account via email sent to you (check spam folder if not received)
- Ensure the prosthesis is powered on
- Ensure Bluetooth of Apple device is turned on
- Fully close down Biosim App, power cycle prosthesis and try again







[WWW.OSSUR.COM](http://WWW.OSSUR.COM)

CRE-9650

 **ÖSSUR**  
LIFE WITHOUT LIMITATIONS