


MY i-LIMB APP CONNECTION GUIDE



My i-Limb App puts the control in your hands



Access all available
grips at the touch
of an icon via Quick
Grips

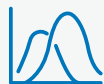

Review and practice
your muscle signals
on the myo-graph




Change which grips
are programmed on
your prosthesis

1 Download the My i-Limb App from the Apple App Store

Please note you need to be
connected to the internet to
sign up to My i-Limb App




4 Connect to your i-Limb / i-Digits: Tap on your serial number

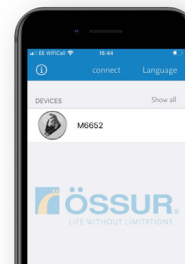


2 Click Sign Up

- Enter email address and
create password
- Accept terms and conditions
- Verify your account via link
on activation email which
you will receive

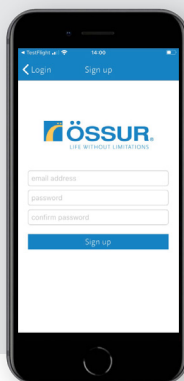

5 Insert last 4 digits of serial number

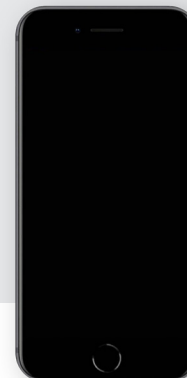

3 First time connection: Insert your security code

Your unique 5 digit security
code can be found by pressing
'I' icon on top left of screen



TROUBLESHOOTING

- Ensure you have activated My i-Limb account via email sent to you (check spam folder if not received)
- Ensure your prosthesis is powered on
- Ensure Bluetooth of Apple device is turned on
- Fully close down My i-Limb App, power cycle prosthesis and try again



My i-Limb App puts the control in your hands



Access all available
grips at the touch
of an icon via Quick
Grips



Review and practice
your muscle signals
on the myo-graph



Change which grips
are programmed on
your prosthesis


1 Download the My i-Limb App from the Apple App Store

Please note you need to be
connected to the internet to
sign up to My i-Limb App



2 Click Sign Up

- Enter email address and
create password
- Accept terms and conditions
- Verify your account via link
on activation email which
you will receive

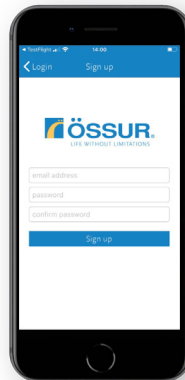


3 First time connection: Insert your security code


Your unique 5 digit security
code can be found by pressing
'I' icon on top left of screen



4 Connect to your i-Limb / i-Digits: Tap on your serial number

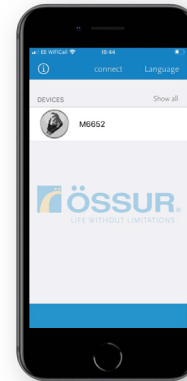

5 Insert last 4 digits of serial number

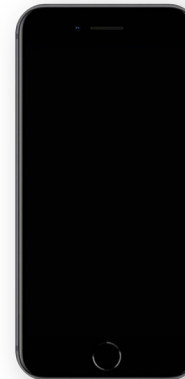

TROUBLESHOOTING

- Ensure you have activated My i-Limb account via email sent to you
(check spam folder if not received)
- Ensure your prosthesis is powered on
- Ensure Bluetooth of Apple device is turned on
- Fully close down My i-Limb App, power cycle prosthesis and try again





WWW.OSSUR.COM

CRE-9650

 **ÖSSUR**
LIFE WITHOUT LIMITATIONS