

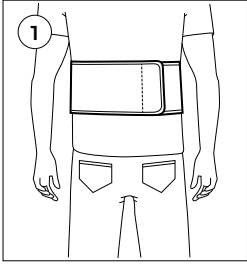



Wrap belt around waist and secure in back.

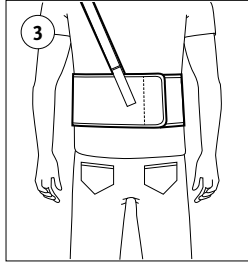


Fasten belt at front and back as shown.

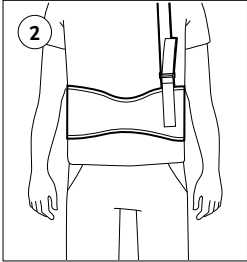


Pull strap to adjust.

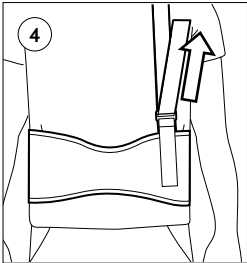


Put arm band on upper arm.

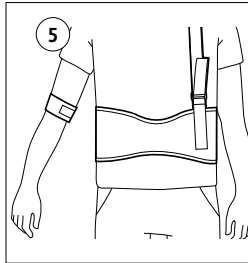


Secure arm band to waist.

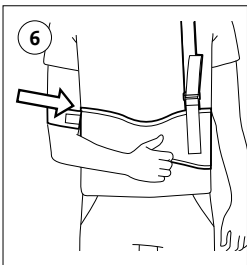


Place wrist band around wrist.

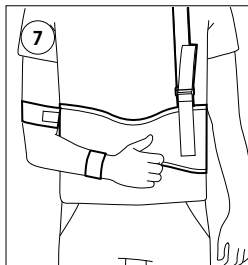

Locate wrist fasteners.

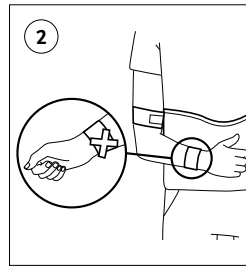

Secure wrist band to waist.

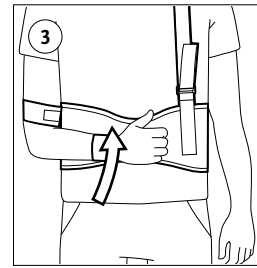

## ENGLISH

### FITTING INSTRUCTIONS FOR THE PREMIUM SHOULDER IMMOBILIZER

#### INDICATION FOR USE

For shoulder conditions that may benefit from immobilizing the shoulder, such as:

- Pre or post-operative
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

#### CAUTIONS

Patients should be instructed to inspect their shoulder and this product before and after each use. Modification to this product are the sole responsible of the prescribing physician.

#### NOTE

If Gel Pads are included with the product, refer to the separate Gel Pad instructions.

#### WASHING INSTRUCTIONS

1. The brace can be hand washed using a mild detergent and cold water.
2. Lay brace out flat on a towel and air dry.

#### GUIDELINES AND CAUTIONS

1. Do not bleach. Do not squeeze or wring out the brace. Do not wash in the washing machine.
2. Do not place in dryer. Do not place near heater.

**EN – Caution:** This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

**Össur Americas**  
27051 Towne Centre Drive  
Foothill Ranch, CA 92610, USA  
Tel: +1 (949) 382 3883  
Tel: +1 800 233 6263  
ossurusa@ossur.com

**Össur UK Ltd**  
Unit No 1, S.Park  
Hamilton Road  
Stockport SK1 2AE, UK  
Tel: +44 (0) 8450 065 065  
ossuruk@ossur.com

**Össur Europe BV**  
De Schakel 70  
5651 GH Eindhoven  
The Netherlands  
Tel: +800 3539 3668  
Tel: +31 499 462840  
info-europe@ossur.com

**Össur Deutschland GmbH**  
Augustinusstrasse 11A  
50226 Frechen  
Deutschland  
Tel: +49 (0) 2234 6039 102  
info-deutschland@ossur.com

**Össur Canada**  
2150 – 6900 Graybar Road  
Richmond, BC  
V6W 0A5 , Canada  
Tel: +1 604 241 8152

**Össur Nordic**  
Box 770  
191 27 Sollentuna, Sweden  
Tel: +46 1818 2200  
info@ossur.com

**Össur Iberia S.LU**  
Calle Caléndula, 93 -  
Miniparc III  
Edificio E, Despacho M18  
28109 El Soto de la Moraleja,  
Alcobendas  
Madrid – España  
Tel: 00 800 3539 3668  
orders.spain@ossur.com  
orders.portugal@ossur.com

**Össur Europe BV – Italy**  
Via Baroaldi, 29  
40054 Budrio, Italy  
Tel: +39 05169 20852  
orders.italy@ossur.com

**Össur APAC**  
2F, W16 B  
No. 1801 Hongmei Road  
200233, Shanghai, China  
Tel: +86 21 6127 1707  
asia@ossur.com

**Össur Australia**  
26 Ross Street,  
North Parramatta  
NSW 2151 Australia  
Tel: +61 2 88382800  
infosydney@ossur.com

**Össur South Africa**  
Unit 4 & 5  
3 on London  
Brackengate Business Park  
Brackenfell  
7560 Cape Town  
South Africa  
Tel: +27 0860 888 123  
infosa@ossur.com



**Össur hf.**  
Grjótháls 1-5  
110 Reykjavík  
Iceland




www.ossur.com

© Copyright Össur 2019

IFU 1449\_001

Rev. 2

**ÖSSUR**  
LIFE WITHOUT LIMITATIONS

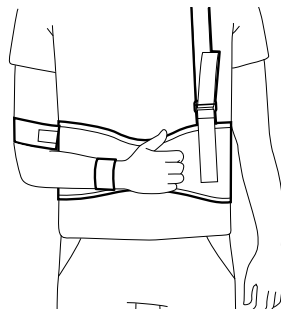

Instructions for Use

PREMIUM SHOULDER  
IMMOBILIZER