ENGLISH

INDICATIONS FOR USE

Elbow conditions which may benefit from compression, such as:

- Elbow pain and swelling
- Medial and lateral epicondylitis (tennis/golfers elbow)

CONTRAINDICATION

Must not be used by individuals for whom compression is contraindicated.

WARNINGS

- The brace should not be used for extended periods of time, and removed during long periods of rest, e.g. sleep.

• Patients with already compromised blood circulation in their extremities, like those with diabetes or peripheral vascular disease need to be extra careful, and are advised to consult with their doctor before using the brace. If you experience pain, swelling, sensation changes, or if your extremities show signs of insufficient blood flow (turning blue, white or cold) while using this brace, discontinue use immediately and consult with your medical professional.

- This product is designed to be worn directly on the skin. Never wear the brace directly over an open wound. Inspect your elbow and the brace before and after each use. If any problems occur related to use, please contact your certified medical professional.
- Contains natural rubber latex which may cause allergic reaction.

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Rubber, Polyester, Polyamide, Elastane, Cotton, Polyurethane, Styrene-Ethylene-Butylene-Styrene based Thermoplastic Elastomers (TPS-SEBS)

CARE INSTRUCTIONS

Wash on a gentle cycle at max 30 °C with mild detergent and air dry (do not tumble dry). For products with hook and loop please secure the hook and loop on the straps before washing to avoid damaging the knit. Please wash regularly to retain the compression of the knitted fabric.

FITTING INSTRUCTIONS

- If between sizes on the sizing chart, always select the smaller size for optimal fit.
- To avoid damaging the knit, fold the hook and loop straps back on themselves.

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