# ENGLISH

## DESCRIPTION OF SYMBOLS

**MD** Medical Device

## USES

Support for weak and/or injured knees.

## FITTING INSTRUCTIONS

1. Remove shoes if necessary.
2. Hold the support so the wider side is at the top and knee opening at the front.
3. Pull the support over your knee and position it so the kneecap fits into the opening.

## WASHING INSTRUCTIONS

This product may be cleaned using mild detergent and cold water. Do not bleach. Do not wash in washing machine or place in dryer. Lay brace flat on a towel to air dry. Do not squeeze.

## GENERAL SAFETY INFORMATION

**WARNING:** In cases of functional change or functional loss, the patient should stop using the device and contact a healthcare professional.

**WARNING:** If the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

## REPORT A SERIOUS INCIDENT

**Important notice to users and/or patients established in Europe:**

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

## LIABILITY TEXT

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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©Copyright Össur 2020 IFU 0563 1461-001