

Instructions for use



Padded Shoulder Immobilizer



Life Without Limitations®

INDICATION FOR USE:

Shoulder injuries or instabilities

FITTING INSTRUCTIONS

1. Place patient's arm inside sling.
2. Enclose hand in sling and place thumb in thumb holder.
3. Place strap around neck and secure through buckle.
4. Adjust both ends of shoulder strap for comfortable fit.
5. Apply waist strap to brace. Adjust both ends of waist strap.
This will keep the sling from rotating.

Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

Warning: This device will not prevent or eliminate risk of injury. Do Not Over-tighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.



Össur Head Office

Grjótháls 5
110 Reykjavík, Iceland
Tel: +354 515 1300
Fax: +354 515 1366
mail@ossur.com



Össur Americas

27051 Towne Centre Drive
Foothill Ranch, CA 92610,
USA
Tel: +1 (949) 382 3883
Tel: +1 800 233 6263
Fax: +1 800 831 3160
ossurusa@ossur.com