NORSK

ANVENDELSE

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EN - Caution: This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

Össur Head Office
Gröfðals 5
Reykjavík Iceland
Tel: +354 515 1300
Fax: +354 515 1366
mail@ossur.com
www.ossur.com

Össur Americas
27412 Aliso Viejo Pkwy
Aliso Viejo, CA 92656, USA
Tel: +800 233 6263
Fax: +1 949 362 3888
ossurusa@ossur.com

Össur Canada
120-11371 Dyke Road
Richmond, BC
V7A 0A1, Canada
Tel: +1 604 241 8152
Fax: +1 604 241 8153

Össur Europe
Ekkerestrifj 4106-4114
P.O. Box 120
5690 AC Son en Breugel
The Netherlands
Tel: +800 3539 3668
Tel: +31 499 462840
Fax: +31 499 462841
info-europe@ossur.com

Össur Iberia
Parque Europolis
Calle Edimburgo N°14
28332 Las Rozas – Madrid
Tel: 916 36 06 93
Fax: 916 37 21 43

Össur UK
Building 3000
Manchester Business Park
Aviator Way
Manchester M22 5TC, UK
Tel: +44 161 490 8500
Fax: +44 161 490 8501
ossuruk@ossur.com

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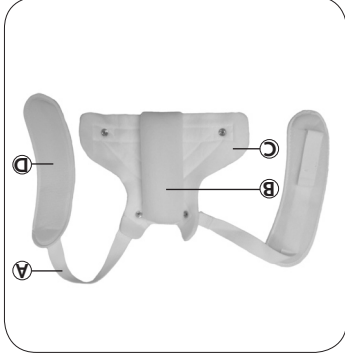
Life Without Limitations

ENGLISH

USES
Clavicle fractures and dislocations, selected soft tissue injuries and postural problems.

CAUTION
Patients should be instructed to inspect their back, shoulders and this product before and after each use.

PHOTO CAPTIONS

- A. Arm straps
- B. Removable foam insert
- C. Felt pad
- D. Arm pad

FITTING INSTRUCTIONS

1. Position the removable foam insert vertically on the felt pad.
2. Place arms through arm straps so that felt pad is located on back and the foam insert is between shoulder blades.
3. Adjust by pulling the arm straps tight. Fasten the contact closures on the arm pad.
4. The cushioned arm pad should be adjusted so that it fits under the arm and in front of the shoulder. The straps should be tight enough to pull the shoulders back.

WASHING INSTRUCTIONS

This product may be cleaned using mild detergent and cold water. Remove foam insert before washing. Do not bleach. Do not wash in washing machine or place in dryer. Lay brace flat on a towel to air dry. Do not squeeze or wring out brace.

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