**Össur Head Office**  
Grjóthals 5  
110 Reykjavík, Iceland  
Tel: +354 515 1300  
Fax: +354 515 1366  
mail@ossur.com

**Össur Nordic**  
P.O. Box 67  
751 03 Uppsala, Sweden  
Tel: +46 1818 2200  
Fax: +46 1818 2218  
info@ossur.com

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## ENGLISH

### USES

Ideal to provide compression and support for strains and weakness of the abdominal area; post natal use, post surgery of abdominal area.

### FITTING INSTRUCTIONS

1. Secure support around abdomen holding the binder with both hands. Align to center of body. Wrap around rib cage securing with contact closure. Contact closure attaches in front of the binder.
2. When securing contact closure, ensure the binder fits tightly enough to provide even, constant pressure without having any impact on the patient's breathing.

### WASHING INSTRUCTIONS

The brace may be cleaned using a mild soap and cold water. Do not wash in washing machine. Lay brace out flat on a towel and air dry. Do not squeeze. Do not place in dryer. Do not place near heater. Do not use bleach or strong detergents.

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**EN – Caution:** Össur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Össur products and components are used with other recommended Össur or authorized components. If un-usual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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**Össur Iberia S.A.**