ENGLISH

USES

Effective treatment for Tennis Elbow (epicondylitis), Golfers elbow and other overuse injuries.

FITTING INSTRUCTIONS

1. Loop strap through the plastic ring and slide up forearm.
2. Position gel over top of the forearm.
3. Tighten strap by wrapping snugly but do not over tighten.
4. For Hot Therapy, place gel pad in microwave and heat in 15 second increments until desired temperature is reached. Use caution when applying hot therapy as gel may become very hot.
5. For Cold Therapy, place gel pad in refrigerator until desired temperature is reached.

WASHING INSTRUCTIONS

Remove gel pad. This brace can be cleaned using a mild detergent and cold water. Lay brace out flat on a towel and air dry. Do not bleach. Do not put in washing machine or dryer. Re-insert Gel Pad in brace.

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EN – Caution: This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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 **ÖSSUR**[®]
LIFE WITHOUT LIMITATIONS

Instructions for Use

Tennis Elbow Support with
Hot/Cold Therapy

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©Copyright Össur 2015