**Össur Head Office**  
Grótháls 5  
110 Reykjavík, Iceland  
Tel: +354 515 1300  
Fax: +354 515 1366  
mail@ossur.com

**Össur Nordic**  
P.O. Box 67  
751 03 Uppsala, Sweden  
Tel: +46 1818 2200  
Fax: +46 1818 2218  
info@ossur.com

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## ENGLISH

### USES

Ideal for Sprains & strains, Gamekeeper's Thumb, De-Quervain's Tendonitis, post cast healing and other selected soft tissue injuries.

### FITTING INSTRUCTIONS

1. Connect the shortest thumb strap to the contact closure.
2. Slide your thumb into the opening.
3. Apply other straps starting with those closest to your thumb. Do not over tighten. Readjust for comfort.
4. The brace should be positioned straight down the thumb side of your forearm.

### WASHING INSTRUCTIONS

Hand wash with mild soap in cold water, rinse until water is clear. Wrap in towel to remove excess water. Do not twist or wring. Air dry only. Do not use heat, drying machine or bleach.

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**EN – Caution:** Össur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Össur products and components are used with other recommended Össur or authorized components. If un-usual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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