Ossur Head Office
 Grjótahals 5
 Reykjavík Iceland
 Tel: +354 515 1300
 Fax: +354 515 1366
 mail@ossur.com

Ossur Asia-Pacific
 26 Ross Street
 North Pararamatta
 Sydney NSW
 2152 Australia
 Tel: +61 2 88382800
 Fax: +61 2 96305310
 info-asia@ossur.com

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WASHING INSTRUCTIONS

1. This product may be cleaned at any time using a mild detergent and cold water.
2. Lay brace out flat on a towel and air dry.
- Do not use bleach or strong detergents. Do not wash in washing machine.
- Do not place in dryer or near heater.

FITTING INSTRUCTIONS

1. Undo toe and calf straps.
2. Step foot into brace.
3. Secure toestrap by affixing the contact closures.
4. Apply calf strap by wrapping behind the leg and affix contact closures.


PHOTO CAPTIONS

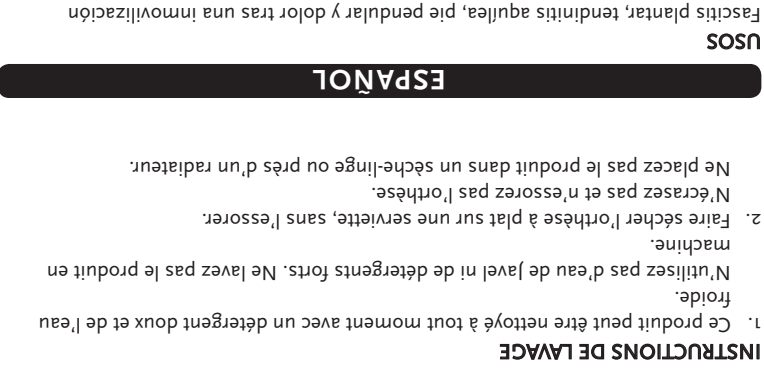1. The patient should be instructed to inspect their foot, ankle, and product before and after each use.
2. Do not walk or attempt to place weight on the foot while using this splint without approval from your physician. This is not a weight-bearing product.
3. Do not use outdoors, or on slippery or uneven surfaces.
4. Contact your certified medical professional if any problems occur with the use of this product.
5. Do not use this product if plantar flexion contracture is greater than 10°.
6. This product is not designed to treat uncontrollable, severe plantar flexion spasticity.
7. Patients with foot neuropathy (insensitive feet) should not use this product.

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