Instructions for use

Form Fit[®] Neoprene Knee
Support with Stabilized
Patella

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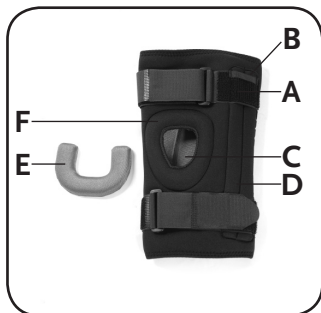
EN- This product has been designed and tested based on single patient usage. It is a single-use device and should never be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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ENGLISH

USES

Patellar tendonitis, subluxation dislocation, post-operation rehabilitation, post arthroscopic surgery, chondromalacia, bursitis, arthritis, and tendonitis.

CAUTION

Patients should be instructed to inspect their knee and the product before and after each use. Modifications to this product are the sole responsibility of the prescribing physician.

PHOTO CAPTIONS

- A. Adjustable contact closure straps
- B. Pull Tabs
- C. Patella Opening
- D. Supportive stays
- E. Universal buttress (with Universal version only)
- F. Sewn in buttress (all other versions)

FITTING INSTRUCTIONS

1. Undo all contact closure straps.
2. Place foot through largest opening and slide brace onto leg using pull tabs.
3. Place patella into the patella opening. Adjust brace so the stays are centered evenly around the knee.
4. Secure all contact closure straps. Adjustments may be necessary as knee swelling patterns change.

WASHING INSTRUCTIONS

1. The brace can be cleaned using a mild detergent and cold water. Do not squeeze or wring out the brace. Do not wash in washing machine.
2. Lay brace out flat on a towel and air dry. Do not place in dryer. Do not place near heater.

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