Össur Head Office
Gríðhalls 5
110 Reykjavík, Iceland
Tel: +354 515 1300
Fax: +354 515 1366
mail@ossur.com

Össur Nordic
P.O. Box 67
751 03 Uppsala, Sweden
Tel: +46 1818 2200
Fax: +46 1818 2218
info@ossur.com

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Instructions for Use

Elastic Knee Sleeve
Elastic Premium Knee Sleeve

Life Without Limitations®

ENGLISH

USES

Support for weak and/or injured knees.

FITTING INSTRUCTIONS

1. Remove shoes if necessary.
2. Hold the support so the knee opening is at the front (premium model only).
3. Pull the support over your knee as you would put on a sock. For premium model, position it so the kneecap fits into the opening.

WASHING INSTRUCTIONS

Hand wash using cold water (30°C/86°F) and mild detergent. Rinse thoroughly and line dry only.

Do not bleach.

Do not machine wash or dry.

Do not squeeze.

For best results, rinse after every use.

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EN – Caution: Össur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Össur products and components are used with other recommended Össur or authorized components. If un-usual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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