

ENGLISH



Medical Device

INDICATIONS FOR USE

For use with short leg cast.

CONTRAINDICATIONS

None known.

WARNINGS AND PRECAUTIONS

- Patients should be instructed to inspect their cast, foot, and product before and after each use.
- If discomfort occurs, patient should contact the prescribing physician immediately.
- This product is for single patient use only.
- If there is a change or loss in device functionality or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

FITTING INSTRUCTIONS

1. Unfasten all contact closures.
2. Insert casted foot into shoe making sure cast is positioned all the way back to the heel.
3. Fasten all contact closure straps snugly, starting with the heel strap.
4. Straps may be adjusted and/or cut as desired.

CLEANING AND CARE

The brace can be hand washed using mild detergent and cold water. Do not wash in washing machine. Do not use bleach.

Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not place in dryer or near heater.

REPORT A SERIOUS INCIDENT

Important notice to users and/or patients established in Europe:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Össur Americas
27051 Towne Centre Drive
Foothill Ranch, CA 92610, USA
Tel: +1 (949) 382 3883
Tel: +1 800 233 6263
ossurusa@ossur.com

Össur UK Ltd
Unit No 1, S:Park
Hamilton Road
Stockport SK1 2AE, UK
Tel: +44 (0) 8450 065 065
ossuruk@ossur.com

Össur Europe BV
De Schakel 70
5651 GH Eindhoven
The Netherlands
Tel: +800 3539 3668
Tel: +31 499 462840
info-europe@ossur.com

Össur Deutschland GmbH
Augustinusstrasse 11A
50226 Frechen
Deutschland
Tel: +49 (0) 2234 6039 102
info-deutschland@ossur.com

Össur Canada
2150 – 6900 Graybar Road
Richmond, BC
V6W 0A5 , Canada
Tel: +1 604 241 8152

Össur Nordic
Box 770
191 27 Sollentuna, Sweden
Tel: +46 1818 2200
info@ossur.com

Össur Iberia S.L.U
Calle Caléndula, 93 -
Miniparc III
Edificio E, Despacho M18
28109 El Soto de la Moraleja,
Alcobendas
Madrid – España
Tel: 00 800 3539 3668
orders.spain@ossur.com
orders.portugal@ossur.com

Össur Europe BV – Italy
Via Baroaldi, 29
40054 Budrio, Italy
Tel: +39 05169 20852
orders.italy@ossur.com

Össur APAC
2F, W16 B
No. 1801 Hongmei Road
200233, Shanghai, China
Tel: +86 21 6127 1707
asia@ossur.com

Össur Australia
26 Ross Street,
North Parramatta
NSW 2151 Australia
Tel: +61 2 88382800
infosydney@ossur.com

Össur South Africa
Unit 4 & 5
3 on London
Brackengate Business Park
Brackenfell
7560 Cape Town
South Africa
Tel: +27 0860 888 123
infosa@ossur.com



Össur hf.
Grjótháls 1-5
110 Reykjavík
Iceland



www.ossur.com

©Copyright Össur 2020

IFU 1506_001

Rev. 1

 **ÖSSUR**[®]
LIFE WITHOUT LIMITATIONS

Instructions for Use

Canvas Rocker Bottom Cast Shoe

Vinyl Rocker Bottom Cast Shoe