ENGLISH

Medical Device

INDICATIONS FOR USE

For arm conditions that may benefit from supporting the arm, such as:

- Post trauma and pre- or post-operative
- To support a cast or a splint

CONTRAINDICATIONS

None known.

WARNINGS AND PRECAUTIONS

- This product is for single patient use only.
- If any undesirable side effects occur with the use of this product, immediately contact a healthcare professional.
- If there is a change or loss in device functionality or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

FITTING INSTRUCTIONS

Putting the sling on:

1. Open the sling.
2. Place injured arm into the sling. Place elbow deep into the pocket of the sling at a 90° angle when possible. Ensure that the open end of sling supports the hand and fingers.
3. Place strap behind the neck and over the opposite shoulder of the injured arm. Position strap and adjust padding on shoulder for comfort.
4. Adjust length of strap using buckle to give proper support. Adjustments may be necessary as swelling patterns change.

CLEANING AND CARE

The sling can be hand washed using mild detergent and cold water. Do not wash in washing machine. Do not use bleach.

Lay sling out flat on a towel and air dry. Do not squeeze or wring out the sling. Do not place in dryer or near heater.

REPORT A SERIOUS INCIDENT

Important notice to users and/or patients established in Europe:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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