ENGLISH

MD Medical Device

PRODUCT DESCRIPTION

Universal Elastic Rib Belt is a brace that provides compression and support to injured rib area.

INDICATIONS FOR USE

Fractures and strains to the rib cage.

CONTRAINDICATIONS

None known.

WARNINGS AND PRECAUTIONS

- This product is for single patient use only.
- If any undesirable side effects occur with the use of this product, immediately contact a healthcare professional.
- If there is a change or loss in device functionality or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

CAUTION: Contains natural rubber **latex** which can cause allergic reactions.

FITTING INSTRUCTIONS

1. Secure support around rib cage holding the belt with both hands. Align to center of body. Wrap around rib cage securing with contact closure. Contact closure attaches in front of the belt.
2. When securing contact closure, ensure the belt fits tightly enough to provide even, constant pressure without having any impact on the patient's breathing.
3. For Female Rib Belts, contour fits just below the bust line.

CLEANING AND CARE

The brace can be hand washed using mild detergent and cold water. Do not wash in washing machine. Do not use bleach.

Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not place in dryer or near heater.

REPORT A SERIOUS INCIDENT

Important notice to users and/or patients established in Europe:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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©Copyright Össur 2020 IFU 0571 1495_001