


Instructions for Use

EXOFORM[®] DORSAL NIGHT
SPLINT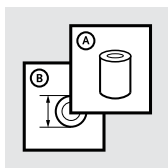

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	6
FR	Notice d'utilisation	8
ES	Instrucciones para el uso	10
IT	Istruzioni per l'uso	12
NO	Bruksanvisning	14
DA	Brugsanvisning	16
SV	Bruksanvisning	18
EL	Οδηγίες Χρήσης	20
FI	Käyttöohjeet	22
NL	Gebruiksaanwijzing	24
PT	Instruções de Utilização	26
JA	取扱説明書	28
ZH	中文说明书	30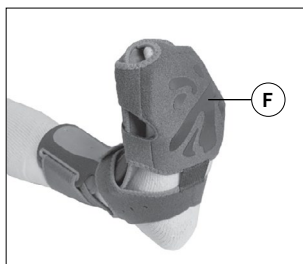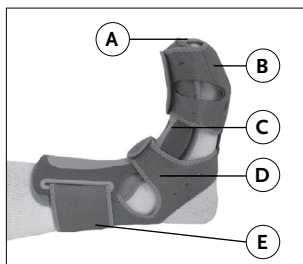

ENGLISH

USEFUL TERMINOLOGY

- A. Toe Strap
- B. Foot Strapping
- C. Shell
- D. Heel Strapping
- E. Calf Strapping
- F. Anti-slip sole

APPLICATION AND REMOVAL

1. Loosen the calf strapping within the slots on the shell and unfasten the heel strapping to provide enough room to pass the foot into and through the softgood.
2. Slide the foot into the brace until the toes reach the end of the softgood as indicated by the toe strap. The toes should not extend past the toe strap.
3. Secure the brace to the foot by first adjusting the foot strapping as necessary being careful to maintain a gentle and comfortable stretch of the foot . Do not over-tighten the foot strapping.
4. Position the shell against the shin and apply the calf strapping to secure the brace in a comfortable manner.
5. Fasten the heel strapping being sure to center the heel of the foot into the heel strapping opening as illustrated. Do not over- tighten the heel strapping.

ADJUSTMENTS

This product has been pre-shaped to provide the optimal degree of stretch to the foot. In situations where a more customized level of stretch of the foot is required, the Exoform Night Splint shell can be modified by hand to either increase or decrease the angle and degree of flexion of the foot.

When modifying the angle of the shell to increase or decrease the stretch of the foot, the product should be held and bent in such a way as to create a bending point at the mid section of the shell in the area of the heel strapping. The shell is not intended to be repeatedly bent and modified as this will adversely affect product integrity.

USE AND CARE

- This product is used to treat Plantar Fasciitis, Achilles Tendonitis, Drop Foot, and Post Static Pain.
- Be careful to not over-tighten the strapping.
- For an optimal level of stretch to the foot, the toes should be positioned to the end of the toe strap within the softgood.
- The shell angle should be adjusted and modified only in special circumstances and only by a medical professional. Repeated modification of the shell angle will compromise the shell integrity.
- This product may be cleaned at any time using a mild detergent and cold water. Do not use bleach or strong detergents. Hand wash and air dry this product. Do not wash the product in a washing machine or in a dryer.

CAUTION AND WARNING

- The patient should be instructed to inspect their foot, ankle, and the product before and after each use.
- This product is not intended for prolonged walking or weight bearing. Any ambulation in the product should be kept to a minimum.
- Do not use this product if plantar flexion contracture is greater than 20 degrees.
- This product is not designed to treat uncontrollable, severe plantar flexion spasticity.
- Patients with diabetes and or neuropathy of lower extremities should not use this product.

WARRANTY

This warranty covers all defects in materials and workmanship of your new Ossur product. The warranty covers all materials, including straps, sleeves, liners, and pads for six months from the date of original purchase.

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- A. Zehenriemen
- B. Fußriemen
- C. Schale
- D. Fersenriemen
- E. Wadenriemen
- F. Antirutschsohle

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