

## BRACING & SUPPORTS CATALOGUE





# ÖSSUR BRACING & SUPPORTS CATALOGUE

## TABLE OF CONTENTS

ORDERING INFORMATION .....	10
INTRODUCTION TO SOLUTIONS.....	12
SPINE .....	15
MIAMI J® .....	17
MIAMI J® SELECT .....	19
MIAMI JTO® THORACIC EXTENSION .....	20
OCCIAN® BACK .....	21
MIAMI Jr® .....	22
PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR .....	23
PHILADELPHIA® TRACHEOTOMY COLLAR .....	24
NECLOC® .....	25
NECLOC® KIDS .....	26
MIAMI LSO® .....	27
MIAMI TLSO 464 .....	29
MIAMI TLSO 456 .....	31
MIAMI LUMBAR® POSTEO .....	33
MP45 SPINAL HYPEREXTENSION BRACE .....	34
PAPOOSE® .....	35
FORMFIT® PRO BACK .....	36
FORMFIT® BACK SUPPORT AIR .....	37
FORMFIT® BACK SUPPORT .....	38
DUAL MATERNITY BELT .....	39
HIP .....	41
REBOUND® HIP .....	43
UNLOADER® HIP .....	45
UPPER EXTREMITY .....	47
FORMFIT® SHOULDER BRACE .....	49
PREMIUM SHOULDER IMMOBILISER .....	50
BERGSCHULTZ UNIVERSAL ARM SLING .....	50
HIGH ARM SLING .....	51
PAEDIATRIC UNIVERSAL ARM SLING .....	51
FORMFIT® PRO ELBOW .....	52
REBOUND® POST-OP ELBOW .....	53
FORMFIT® TENNIS ELBOW .....	55
FORMFIT® PRO WRIST .....	56
FORMFIT® WRIST / FORMFIT® WRIST & FOREARM .....	57
SOFTPRO® FUNCTIONAL RESTING WHFO .....	58
SOFTPRO® GRIP WHFO .....	59
FORMFIT® UNIVERSAL WRIST & THUMB .....	60
FORMFIT® THUMB SPICA .....	61
WRIST / THUMB BRACE .....	62
ELASTIC THUMB SPICA .....	62
SPECTRA® WRIST BRACE .....	63

# ÖSSUR BRACING & SUPPORTS CATALOGUE

<b>KNEE</b>	<b>65</b>
UNLOADER ONE® X	67
UNLOADER ONE® X CUSTOM	71
UNLOADER ONE®	74
UNLOADER ONE® CUSTOM	77
UNLOADER ONE® PLUS	78
REBOUND® CARTILAGE	80
CTI® OA	82
REBOUND® PCL	84
REBOUND® ACL	88
REBOUND® DUAL BASIC AND REBOUND® DUAL BASIC ST	92
FORMFIT® OA EASE	94
CTI® OTS	96
CTI® CUSTOM	99
PARADIGM®	102
FLEX™	103
C180™ ROCKET	105
REBOUND® POST-OP KNEE	106
FORMFIT POST-OP KNEE	107
REBOUND® KNEE BRACE	108
REBOUND® KNEE IMMOBILIZER UNIVERSAL	109
FORMFIT® PRO KNEE	110
FORMFIT® TRACKER	112
FORMFIT® PRO KNEE FLITE	113
FORMFIT® HINGED LATERAL J	114
FORMFIT® KNEE HINGED	115
FORMFIT® KNEE MCL	116
FORMFIT® KNEE ROM BRACE	117
NEOPRENE HINGED KNEE SUPPORT	118
NEOPRENE KNEE SUPPORT WITH STABILISED PATELLA	118
NEOPRENE WRAPAROUND HINGED KNEE SUPPORT	119
NEOPRENE KNEE SLEEVE	120
MEASUREMENT FORM CTI® CUSTOM BRACE	121
MEASUREMENT FORM UNLOADER ONE® CUSTOM / REBOUND® ACL & PCL	124
<b>FOOT AND ANKLE</b>	<b>125</b>
REBOUND® DIABETIC WALKER	127
DH OFFLOADING WALKER®	129
REBOUND® AIR WALKER	130
FORMFIT® WALKER AIR	132
FORMFIT® WALKER	133
PAEDIATRIC WALKER	134
AFO DYNAMIC	135
AFO LIGHT	137
PROLITE AFO	138
EXTRA STRONG PROLITE	138
REBOUND FOOT-UP®	139
FOOT-UP®	140



# ÖSSUR BRACING & SUPPORTS CATALOGUE

---

FORMFIT® PRO ANKLE .....	141
FORMFIT® ANKLE BRACE .....	143
FORMFIT® ANKLE WITH SPEEDLACE .....	144
FORMFIT® ANKLE STIRRUPS .....	145
EXOFORM® DORSAL NIGHT SPLINT .....	146
FORMFIT® NIGHT SPLINT .....	147
DYNAPRO® AFO .....	148
SOFTPRO® GAIT TRAINER AFO .....	149
SOFTPRO® IN-BED AFO .....	150
DH OFFLOADING POST-OP STYLE SHOE .....	151
MESH TOP POST-OP SHOE .....	152
SOFT TOP POST-OP SHOE .....	152
CANVAS ROCKER BOTTOM CAST SHOE .....	153
<b>CASTING &amp; TRACTION .....</b>	<b>155</b>
TECHFORM® PREMIUM .....	157
NOTES .....	158



## WELCOME TO THE ÖSSUR® UK BRACING AND SUPPORTS PRODUCT CATALOGUE 2021

---

Össur's diverse range of bracing and supports products offer expert treatment for a range of conditions. Combining scientifically advanced products with high standards of quality assurance and customer care, our destination is Life Without Limitations. Össur's extensive orthotics range offers some of the most effective orthopaedic solutions available today.

Your business is important to us. We are committed to offering products and services that will enhance your operation, from highly innovative products backed by exceptional service and educational support, to more common items that represent great value for money.

We hope this catalogue will guide you through our comprehensive range of products.

Should you need any assistance, please do not hesitate to contact our Customer Care Team on 03450 065 065, email us at [ukinfo@ossur.com](mailto:ukinfo@ossur.com) or visit [www.ossur.com](http://www.ossur.com).

Yours sincerely



Stewart Hogg, Sales Director, Össur UK Ltd.

## CUSTOMER CARE TEAM

Our Customer Care team really understand the UK healthcare industry and the NHS in particular – we've been working and changing with it for years.

Tel: [03450 065 065](tel:03450065065)

Enquiries: [ukinfo@ossur.com](mailto:ukinfo@ossur.com)

Orders: [ossuruk@ossur.com](mailto:ossuruk@ossur.com)



If you would like to know more about consolidated ordering you can contact our friendly and experienced Customer Care team or your Area Sales Manager who will help set you up if you like the concept.

Our sales team are well trained and are there to help you in any way they can – training staff, fitting products, solving logistical problems and demonstrating new technology.

As a global company we are able to innovate and advance our industry, investing heavily in research and development, we bring a continuous stream of new products, new ideas and materials to our customers in the UK.

These innovations can save cost, some as alternative treatments to surgery and others by reducing inpatient stay.

We're a pretty friendly bunch who work to a set of values that guide our actions, rather than a rule book.

### HONESTY

Honest with our customers  
and honest with each other.

### FRUGALITY

Use resources responsibly,  
ours and the environment's.

### COURAGE

Do the right thing and  
not the easy thing.

Össur UK are working towards implementation of GS1 as part of the NHS eProcurement Strategy.

NATIONAL SALES MANAGER

Tel: +44 (0)796 800 5658

SCOTLAND &  
NORTHERN IRELAND  
Tel: +44 (0)755 117 0897

NORTH WEST &  
REPUBLIC OF IRELAND  
Tel: +44 (0)796 895 0286

NORTH EAST &  
SOUTH YORKSHIRE  
Tel: +44 (0)788 181 2771

MANCHESTER &  
M62 CORRIDOR  
Tel: +44 (0)795 871 7373

MIDLANDS &  
SOUTH WALES  
Tel: +44 (0)782 795 0195

SOUTH WEST  
Tel: +44 (0)7824 371 434

NORTH EAST LONDON &  
EAST ANGLIA  
Tel: +44 (0)758 421 6572

SOUTH CENTRAL  
Tel: +44 (0)779 510 5748

## ORDERING INFORMATION

---

- Please quote your account number when placing an order
- Please quote model numbers and any other specification required when ordering
- Please quote your purchase order number
- Orders placed by telephone then faxed, posted or emailed should be clearly marked with 'CONFIRMATION ORDER' to avoid risk of duplication
- Össur® participate in the NHS PURCHASING CARD SCHEME and can accept orders / payment by this method

### RESPONSE TO ORDERS

We are proud of our service to our customers and maintain large stocks of products to expedite delivery. Orders for stock items are usually processed and despatched the same day.

### METHOD OF DISPATCH

Your order will be dispatched by the most economical means available, with regard to weight / volume and the degree of urgency. Unless otherwise instructed, orders will be sent by standard 2 - 3 day delivery. Timed deliveries are available on request. Please contact Customer Care for pricing.

- Next working day before 9:00am\*
- Next working day before 10:30am\*
- Next working day before 12:00 noon\*
- Next working day before 5pm\*

*\* where available at an additional cost*

### CUSTOM PRODUCTS

Össur® UK Ltd., through our own manufacturing facility, have the expertise and resources to create one-off custom requirements for products such as the Unloader One Custom, Rebound PCL Custom, Rebound ACL Custom, CTi Custom. Where this is the case, please allow 10 working days for delivery.

### INCORRECT DELIVERY

On receipt of the goods the customer is required to check the consignment. In the unlikely event that the type or quantity of goods is incorrect then you are requested to notify our Customer Care Department within 7 working days of the date of delivery. After 7 working days Össur® UK Ltd cannot accept responsibility for an alleged incorrect delivery.

### RETURN OF GOODS POLICY

You can return your unwanted products to us free of charge for up to 365 days after you purchased them. Authorisation must be issued by Össur® UK Ltd prior to goods being returned, please contact Customer Care for a return reference number and ensure sections 1-5 of Return Advice Note (on the back of the dispatch note) are fully completed. Soiled, custom manufactured, modified or obsolete goods may not be returned. Goods accepted for return must be in a resalable condition, guaranteed infection free and in the original packaging. Össur® UK Ltd reserves the right to charge delivery costs and / or re-stocking fee of up to 20% on returned goods.

### 30-DAY MONEY BACK GUARANTEE ON THE UNLOADER ONE AND UNLOADER ONE X

A product that is risk-free; if the product doesn't work for a patient, you get a full refund.

### ÖSSUR NO QUIBBLES GUARANTEE.

At Össur we know that accidents can happen. To put your mind at ease and to avoid unexpected costs, we are pleased to announce that the Unloader One is covered by our Össur Care, No Quibbles Guarantee.

This coverage runs concurrent with the original 2 year warranty as applicable and covers accidental damage during the normal working process of the brace and in accordance with the safe usage guidelines provided in the product manuals.

### TERMS AND CONDITIONS OF SALE

Our full Terms and Conditions of Sale can be provided upon request.

The product range and availability may change during the period that the current catalogue runs.

The returns policy does not apply to custom made or modified goods. Statutory rights are not affected. Please check with Customer Care at time of sale if in doubt.

Telephone Orders: 03450 065 065

Facsimile Orders: 0330 808 3926

E-Mail: [ossuruk@ossur.com](mailto:ossuruk@ossur.com) for orders, [ukinfo@ossur.com](mailto:ukinfo@ossur.com) for enquiries

### SINGLE PATIENT USE

It is a condition of Sale that any orthopaedic product purchased from Össur® UK Ltd is restricted to 'Single use Only' by the originally-fitted patient in order to protect the care provider and the patient against potentially adverse consequences of infectious disease transmission or material instability resulting from previous use by the original wearer. Any expressed or implied warranties are voided if the orthosis is re-used for another patient. Any orthosis purchased from Össur® UK Ltd must be prescribed and applied by a qualified clinician to ensure that it meets the needs of the particular patient and accomplishes the desired results.







## ÖSSUR OA SOLUTIONS

### Enhance quality of life and mobility for people living with osteoarthritis.

These product solutions have been built on decades of experience and are designed to enhance the quality of life of people living with osteoarthritis by reducing pain and improving mobility. Through innovation leadership, OA expertise and partnerships with medical professionals, we offer comprehensive, proven and effective treatment options.

## UNLOADER®

The Össur Unloader® brand name has become synonymous with osteoarthritis bracing, representing OA Solutions' technology as the industry standard, with more than 15 clinical studies demonstrating its efficacy.

Össur strives to maintain its position as the innovation leader for OA Solutions by continuing to develop simple, yet effective clinically proven treatment solutions that increase people's quality of life.





## For people recovering from fractures, ligament injuries or in need of post-operative treatment solutions.

These product solutions have been built on decades of experience and are designed to enhance the healing process of bone and soft tissue injuries. Through innovation leadership, proven biomechanical approaches, and partnerships with leading medical professionals, we strive to offer comprehensive and effective treatment options.

Össur Injury Solutions are globally recognised, primarily under the brand names CTi®, Rebound® and Miami®, all of which are developed to assist healing and increase people's quality of life.



Our expertise in Injury Solutions has driven us to take the next step forward by developing indication-specific products which are designed to optimise healing while maintaining function and mobility. These Functional Healing products incorporate Össur proprietary technology and are designed to enhance clinically relevant outcomes.



FUNCTIONAL HEALING –  
WHERE MOBILITY MEANS RECOVERY.







## TABLE OF CONTENTS

---

SPINE .....	15
MIAMI J® .....	17
MIAMI J® SELECT .....	19
MIAMI JTO® THORACIC EXTENSION .....	20
OCCIAN® BACK .....	21
MIAMI Jr® .....	22
PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR .....	23
PHILADELPHIA® TRACHEOTOMY COLLAR .....	24
NECLOC® .....	25
NECLOC® KIDS .....	26
MIAMI LSO® .....	27
MIAMI TLSO 464 .....	29
MIAMI TLSO 456 .....	31
MIAMI LUMBAR® POSTEO .....	33
MP45 SPINAL HYPEREXTENSION BRACE .....	34
PAPOOSE® .....	35
FORMFIT® PRO BACK .....	36
FORMFIT® BACK SUPPORT AIR .....	37
FORMFIT® BACK SUPPORT .....	38
DUAL MATERNITY BELT .....	39

## MIAMI J®

Known as the superior c-spine immobiliser through multiple independent studies, the Miami J's patented design meets the specialised needs of individuals with cervical injuries. Bioengineered to minimise pressure points in key known areas: Chin, occiput, trapezius, and clavicle. Miami J offers features that enhance compliance, while providing immobilisation, inhibition of skin deterioration, and patient comfort.



## INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients


## FEATURES AND BENEFITS

- Sorbatex™ padding is antibacterial and clinically proven to inhibit microbial growth, which enhances comfort and helps assure patient compliance
- Comfort adjustment buttons for occiput and sternal pad is removable when supine, lying and swallowing
- MRI, CT and X-ray lucent, MR compatible

## REQUIRED MEASUREMENTS

- Sizing based on 6 phenotypes to ensure correct fit and optimum immobilisation

Phenotype-driven sizing system makes size selection simple while drastically reducing margin of error in fitting a patient. Each collar has a bioengineered front with a specific back designed to work together for superior immobilisation. For paediatric sizing, please see Miami Jr in this catalogue.

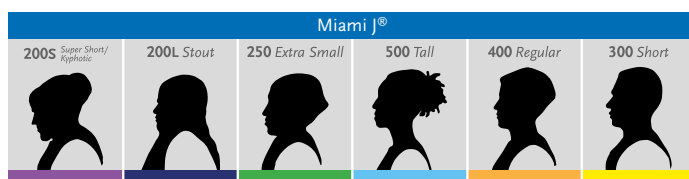

# MIAMI J<sup>®</sup>

## ORDERING INFORMATION

### MIAMI J

Size	Collar set*	Collar	Collar Back	Collar Front
Super Short	MJR-200S	MJ-200S	MJ-400BK	MJ-200SFRT
Stout	MJR-200L	MJ-200L	MJ-200LBK	MJ-200LFRT
XS	MJR-250	MJ-250	MJ-250BK	MJ-250FRT
Short	MJR-300	MJ-300	MJ-300BK	MJ-300FRT
Regular	MJR-400	MJ-400	MJ-400BK	MJ-400FRT
Tall	MJR-500	MJ-500	MJ-500BK	MJ-500FRT
Bariatric	MJR-200B	MJ-200B		

\*Collar Sets come with one additional set of replacement pads.

Miami J Front and back panels may be interchanged to accommodate patient anatomy, e.g. a Short front may be used with a Tall back. Choose collar size based on patient neck height, not patient body size. Super Short collars are packaged with regular back panels

## ACCESSORIES

### REPLACEMENT PADS

Size	Part#
Super Short	MJP-100
Stout	MJP-200L
XS	MJP-250
Short	MJP-100
Regular	MJP-100
Tall	MJP-100
Bariatric	MJP-200B



## MIAMI J® SELECT

The Miami J Select cervical collar by Össur offers the range of height adjustability in the market with a single, universal design that replicates the comfort, immobilisation and quality you expect from a Miami J product. Featuring a patented, first-to-market locking mechanism to encourage patient compliance and anti-microbial Sorbatex padding, Miami J Select is designed to support successful patient outcomes.



### INDICATIONS FOR USE

- Situations requiring gross immobilisation of the cervical spine, these may include:
- Pre and post c-spine surgery
- C-spine precaution for trauma patients

### FEATURES AND BENEFITS

- Easy height adjustability with a tamper-proof locking mechanism – ‘Left to Lock’
- Anti-microbial & biocompatible Sorbatex™ padding prevents bacterial build-up
- Universal design that doesn’t compromise immobilisation or comfort
- 7 precise fitting sizes with opportunity to fine-tune
- Miami J® inspired chin tray
- One-piece, anatomically inspired collar back with Flex Edge® designed to reduce occipital pressure
- Easy to don and doff with patented Reproducible Fit option
- Intuitive blue Patient Touch-Points to send patients home with confidence – Blue is You™
- Simplified comfort adjustments with integrated and patented Sternal-Relief Dial
- Large tracheal opening – improved function for ICU patients
- Easy to clean and replace multi-coloured padding
- X-ray and CT lucent, MR-safe
- Compatible with Occian™ Back & Miami JTO® accessories

### ORDERING INFORMATION

#### COLLAR SET

Description	Part#
Collar set	MJSR-101

#### MIAMI J SELECT COLLAR

Description	Part#
Miami J Select Collar	MJS-101

### OTHER ACCESSORIES

Description	Part#
Miami J Select Replacement Pads	MJSP-101
Miami J Select Tabs Reproducible Fit Tabs (10 pack)	MJST-101

## MIAMI JTO® THORACIC EXTENSION

The Miami JTO Thoracic Extension is designed in conjunction with leading spine surgeons, orthotists and nurses. The orthosis is used exclusively with all Miami J® collars when even greater stability at the cervical junctions is indicated.

Miami JTO Thoracic Extension is a versatile, tool-free device with a patented chin strut, providing more control than a collar alone can provide, designed to treat cervical and high-thoracic injuries.



### INDICATIONS FOR USE

- Low c-spine and high t-spine precaution for trauma patients
- Immobilisation for pre and post low c-spine and high t-spine surgery
- Other conditions requiring gross immobilisation of the low-cervical and/or high-thoracic-spine
- Conditions requiring gross immobilisation of the low cervical and/or high-thoracic-spine

### FEATURES AND BENEFITS

- Patented design with 'floating back' permits maximum immobilisation without the risk of compensatory motion
- Snap on and off without tools
- Sorbatex™ II padding is antibacterial and clinically proven to inhibit microbial growth, which enhances comfort and helps assure patient compliance
- MRI, CT and X-ray lucent

### REQUIRED MEASUREMENTS

- Universal, one size

### ORDERING INFORMATION

#### MIAMI JTO THORACIC EXTENSION

Description	Part#
Miami JTO	MT-900
Miami JTO with replacement softgoods	MTR-900

Compatible with all sizes of the Miami J®, Miami J® Select and the P3 Miami Jr®. Miami J collar is not included with Miami JTO Thoracic Extension. Miami J collar needs to be purchased also

### ACCESSORIES

Description	Part#
Replacement softgoods*	MTP-950

\*Contains 2 x 4" Shoulder Strap Pads / 4 x 8" Waist Strap Pads / 4 x Snap Rivet Sets

## OCCIAN® BACK

The Occian Back is designed exclusively for the specialised needs of the multi-system trauma patient in the ICU. Universally sized, specifically engineered and designed with advanced, Intuitech™ pressure-relieving memory foam, the Occian Back helps caregivers manage c-spine patients who are at high risk of occipital breakdown.



### INDICATIONS FOR USE

- Patients not awake and alert within 24 - 48 hours after admission
- Patients with pre-existing skin injuries and/or lacerations to the head/neck
- Burn patients who must wear a collar
- Elective surgery patients desiring night time post-operative comfort

### FEATURES AND BENEFITS

- Intuitech pressure relieving memory foam
- Maintains c-spine stability
- Provides unique pressure-relieving comfort by replacing the back section of a Miami J Collar.
- Easy to clean
- MRI, CT and X-ray lucent

### REQUIRED MEASUREMENTS

- Universal, one size

### ORDERING INFORMATION

Description	Part#
Occian Back	ACB-100
Occian Back Junior	ACB-200

Compatible with all sizes of the Miami J®, Miami J® Advanced, Miami J® Select and the Miami Jr®. Collar front panel not included.

## MIAMI Jr®

Miami Jr is engineered to fit the unique anatomies of patients aged 12 and under. The Miami Jr line, which spans the growth cycle, is sized according to age. Össur is the only company that has a paediatric line of collars based on the anatomical differences between adults and children. We realise that children are not just “little adults” - they require innovative design.



## INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- pre and post c-spine surgery
- c-spine precaution for trauma patients

## FEATURES AND BENEFITS

- Designed exclusively for children 12 years and under
- Sorbatex™ II pads make the collar “skin friendly” and protect skin integrity during extended wear
- MRI, CT and X-ray lucent, MR compatible

## REQUIRED MEASUREMENTS

- Age

## ORDERING INFORMATION

### MIAMI JR

Size	Collar set*	Collar
0 - 6 months	MJR-P0	MJ-P0
6 - 24 months	MJR-P1	MJ-P1
2 - 6 years	MJR-P2	MJ-P2
6 - 12 years	MJR-P3	MJ-P3

\*Collar sets come with one additional set of replacement pads.

## ACCESSORIES

### REPLACEMENT PADS

Size	Part#
0 - 6 months	MJP-P0
6 - 24 months	MJP-P1
2 - 6 years	MJP-P2
6 - 12 years	MJP-P3

## PHILADELPHIA® ADJUSTABLE TRACHEOTOMY COLLAR

Philadelphia tracheotomy collars have been an industry standard for decades. Philadelphia Adjustable is constructed from the same high-quality materials as the original, and combines convenient size adjustability with immobilisation and comfort.

### INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients

### FEATURES AND BENEFITS

- Circumference adjustable and height adjustable with permanent lock
- Anatomically shaped Plastazote® foam leaving clearance for cervical veins
- Well ventilated and water-resistant for bathing and hydro-therapy
- MRI, CT and X-ray lucent, MR compatible

### REQUIRED MEASUREMENTS

- Universal, one size



## ORDERING INFORMATION

Description	Part#
Philadelphia Adjustable Tracheotomy Collar	PHP-AA

## PHILADELPHIA® TRACHEOTOMY COLLAR

The Philadelphia Tracheotomy Collar is a hospital and rehabilitation collar that is designed to maintain neutral alignment of the cervical spine after injury to the neck. The collar contains a large tracheal opening, allows carotid pulse monitoring and emergency tracheotomies.

### INDICATIONS FOR USE

- C-spine precaution for trauma patients
- Immobilisation for pre c-spine surgery
- Other conditions requiring gross immobilisation of the mid-cervical spine

### FEATURES AND BENEFITS

- Anterior and posterior rigid plastic reinforcement supports limit movement
- Total cervical arch support maintains neutral alignment of the cervical spine
- Water-resistant for bathing and hydro-therapy
- Easily customised for individual requirements
- Does not contain natural rubber latex, non-toxic and hypoallergenic Plastazote® foam reduces skin irritation



### REQUIRED MEASUREMENTS

- Neck circumference and height from tip of chin to sternal notch

## ORDERING INFORMATION

Size	Neck circumference	6cm height	8cm height	11cm height	13cm height
XS	< 25cm	PHP-T2XS			
S	25 - 33cm	PHP-T2S	PHP-T3S	PHP-T4S	PHP-T5S
M	33 - 41cm	PHP-T2M	PHP-T3M	PHP-T4M	PHP-T5M
L	41 - 48cm	PHP-T2L	PHP-T3L	PHP-T4L	PHP-T5L
XL	> 48cm	PHP-T2XL	PHP-T3XL	PHP-T4XL	PHP-T5XL

Size	Neck circumference	Part#
Infant	15 - 20cm	PHP-TI
Paediatric	20 - 28cm	PHP-TP

## ACCESSORIES

### PHILADELPHIA COOLMAX COTTON LINER

Size	Neck circumference	Part#
Child	20 - 28cm	PHP-LCP
S	25 - 33cm	PHP-LCS
M	33 - 41cm	PHP-LCM
L	41 - 48cm	PHP-LCL

### PHILADELPHIA TRACHEOTOMY ATTACHMENTS

Size	Stabiliser
Adult	PHP-OS

## NECLOC®

### INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients

### FEATURES AND BENEFITS

- Ergonomic two-piece clamshell design and retaining strap secure flexion during application
- Five unique phenotype-driven sizes make it easy to select the correct size
- Minimal manipulation of patient during application
- X-ray and CT lucent. MR safe




### REQUIRED MEASUREMENTS

- Sizing based on 5 phenotypes to ensure correct fit and optimum immobilisation

## SIZING

- Phenotype-driven sizing system makes size selection simple while drastically reducing margin of error in fitting a patient. Each collar has bioengineered size specific front and back sections designed to work together for superior immobilisation. For paediatric sizing, please see NecLoc Kids.

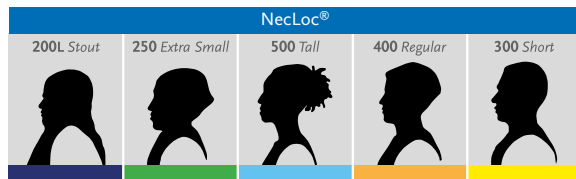

## ORDERING INFORMATION

Size	Part#
Stout	NL-200EL
XS	NL-250E
S	NL-300E
M	NL-400E
L	NL-500E



## NECLOC® KIDS

### INDICATIONS FOR USE

Situations requiring gross immobilisation of the cervical spine, these may include:

- Pre and post c-spine surgery
- C-spine precaution for trauma patients

### FEATURES AND BENEFITS

- Ergonomic two-piece clamshell design and retaining strap secure flexion during application
- Three paediatric sizes ensure optimal airway and cervical spine immobilisation
- Minimal manipulation of patient during application to children
- Meets the unique developing anatomical requirements of children
- X-ray and CT lucent. MR Safe



### REQUIRED MEASUREMENTS

- Age

### ORDERING INFORMATION

Age	Part#
0 - 2 years	NL-P1
2 - 6 years	NL-P2
6 - 12 years	NL-P3

## MIAMI LSO®

Miami LSO is a spinal orthosis designed for pain relief, protection of injured ligaments or muscles and post-surgical immobilisation. It provides low-profile, anatomical immobilisation, easy fit and wear, options for modular support and an advanced pulley system with a length-adjustable pulley cord.



### INDICATIONS FOR USE

Conditions requiring gross immobilisation of the trunk in the lumbar region such as:

- Stable non-displaced spinal fractures (L1-L5)
- Spinal Stenosis
- Herniated Disks
- Degenerative Spinal Pathologies
- Spondylolysis
- Spondylolisthesis

### FEATURES AND BENEFITS

- Slim, breathable material
- Proven Miami double-pulley system with length-adjustable pulley cord
- Intuitive fitting without trimming with fold-over belt arms & Precise-Size™ landmarks
- Optional fit tool for perfect fitting directly on the patient
- Universal belt sizing accommodates waists up to 178cm
- New Universal Posterior panel to accommodate more patients with less inventory
- Contoured posterior panels sized S-XL for a precise, anatomical fit
- Quick-Fit pocket enclosure and blue highlights for patient ease of use

### REQUIRED MEASUREMENTS

- Waist circumference

# MIAMI LSO®

## ORDERING INFORMATION

### MIAMI LSO BELT

Size	Waist Circumference	Part#
6" Universal (Slim)	66-127cm	MLSO-1E
8" Universal (Standard)	66-127cm	MLSO-2E



### OPTIONAL BELT EXTENDERS (2 PCS)

Size	Waist Circumference	Part#
6" (Slim) Belt Extension	127-177cm	MLSO-1EXT
8" (Standard) Belt Extension	127-177cm	MLSO-2EXT

### STANDARD POSTERIOR PANEL

Size	Part#
Standard Posterior Panel	MLSO-P56



### POSTERIOR PANEL WITH LATERAL SUPPORTS

Size	Waist Circumference	Part#
S	66-86cm	MLSO-P12
M	86-96cm	MLSO-P22
L	96-127cm	MLSO-P32
XL	127-178 cm	MLSO-P42



\*Non-lateral and non-lordotic panels available on request.

### ANTERIOR PANEL

Size	Part#
6" Panel (Slim)	MLSO-PA6
8" Panel (Standard)	MLSO-PA8



### QUICK FIT TOOL

Size	Part#
Quick fit tool	B-040100000



## MIAMI TLSO 464

The Miami TLSO from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to-use pre-assembled Miami TLSO brace offers three configurations in one.

**Compliance:** Easily applied in over the shoulder or underarm configuration, donning and doffing is further simplified with the conveniently located quick release buckles.

**Adjustability:** The universal belt designs allows for setting the belt length on the patient for a more accurate sizing experience.

**Comfort:** Truly shaped to fit the anatomy, the anatomically contoured design better immobilises the spine while maximising patient comfort.

**Ease of Use:** The Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



## INDICATIONS FOR USE

Conditions requiring gross immobilisation of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilisation
- Spinal support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolithesis
- Spondylolysis

## CONTRAINDICATIONS

- Unstable, displaced fractures

## FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/doffing
- Fully assembled for fitting with ease
- Accommodates heights 155-193 cm
- Posterior thoracic extension option with ergonomic axillary strapping
- Step-down to an LSO as one progresses through rehabilitation
- One-touch telescoping anterior-thoracic extension option with single or double-hinge
- The convenience of a one-size Universal or individualise with a uniquely-sized anatomical design
- Improve comfort and immobilisation with multiple strapping configurations and removable anchors

## REQUIRED MEASUREMENTS

- Waist circumference

## MIAMI TLSO 464

### ORDERING INFORMATION

#### ATE/PTE, LORDOTIC

Size	Part#	Waist circumference
Universal	MTLSO-2351	66 - 177cm
S	MTLSO-2111	66 - 86cm
M	MTLSO-2121	86 - 107cm
L	MTLSO-2131	107 - 127cm
XL	MTLSO-2141	127 - 177cm

\*Non-lateral and non-lordotic available on request

\*Belt extensions are required for waist circumferences larger than 127cm

#### ATE/PTE, LORDOTIC, SLIM

Size	Part#	Waist circumference
Universal	MTLSO-1351	66 - 177
S	MTLSO-1111	66 - 86
M	MTLSO-1121	86 - 107
L	MTLSO-1131	107 - 127
XL	MTLSO-1141	127 - 177

\*Belt extensions are required for waist circumferences larger than 127cm

### OTHER ACCESSORIES

Description	Part#
5MM Hex Long Wrench	MTLSO-0004
Axillary strap accessory kit	MTLSO-0003
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-167cm. Fits size XL of Miami LSO.)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-167cm. Fits size XL of Miami LSO.)	MLSO-1EXT

## MIAMI TLSO 456

The Miami TLSO 456 from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to-use pre-assembled Miami TLSO brace offers the convenience of a one-size Universal or a uniquely-sized anatomical design.

**Compliance:** Blue is You™ patient touch-points help send patients home with confidence.

**Adjustability:** The universal belt design allows for setting the belt length on the patient for a more accurate sizing experience. Heat moldable posterior panels allow a customised fit, while the modular design allows step-down support to an LSO.

**Comfort:** Featuring the Miami LSO™ Advanced Compression pulley system with lumbar compression you can feel, and an anatomically contoured design that better immobilises the spine while maximising patient comfort.

**Ease of Use:** Easily applied with over-the-shoulder straps, the Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



## INDICATIONS FOR USE

Conditions requiring gross immobilisation of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilisation
- Spinal support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolithesis
- Spondylolysis

## CONTRAINDICATIONS

- Unstable, displaced fractures

## FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/doffing (when compared to Miami Lumbar TLSO)
- Fully assembled for fitting with ease
- Accommodates heights 155-193cm
- Step-down to an LSO as one progresses through rehabilitation
- The convenience of a one-size Universal or individualise with a uniquely-sized anatomical design

## REQUIRED MEASUREMENTS

- Waist circumference

## MIAMI TLSO 456

### ORDERING INFORMATION

#### PTE LORDOTIC

Size	Part#	Waist circumference
Universal	MTLSO-2352	66 - 177cm
S	MTLSO-2112	66 - 86cm
M	MTLSO-2122	86 - 107cm
L	MTLSO-2132	107 - 127cm
XL	MTLSO-2142	127 - 177cm

\*Belt extensions are required for waist circumferences larger than 127cm

#### PTE LORDOTIC, SLIM

Size	Part#	Waist circumference
Universal	MTLSO-1352	66 - 177cm
S	MTLSO-1112	66 - 86cm
M	MTLSO-1122	86 - 107cm
L	MTLSO-1132	107 - 127cm
XL	MTLSO-1142	127 - 177cm

\*Non-lateral and non-lordotic available on request.

\*Belt extensions are required for waist circumferences larger than 127cm

### OTHER ACCESSORIES

Description	Part#
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-167cm. Fits size XL of Miami LSO)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-167cm. Fits size XL of Miami LSO)	MLSO-1EXT



## MIAMI LUMBAR® POSTEO

Designed with patient comfort and compliance in mind, the Miami Lumbar Posteo provides functional relief of vertebral fracture pain, and facilitates return to mobility. The lumbar compression system offers both pain relief via active vertebral offloading and better grip of patient's hips for greater ease of use and compliance. The Miami Lumbar Posteo includes an innovative approach to ease of use and consistent donning.



### INDICATIONS FOR USE

- Back pain caused by stable vertebral compression fractures (VCF)
- Post-operative immobilisation

### FEATURES AND BENEFITS

- Active offloading via thoracic extension and lumbar compression
- Integrated pulley system for customised fit
- Trimmable axillary straps with adjustable tension for optimal stabilisation
- Cold-moldable aluminum PTE to fit different anatomies
- Extra cushioning around the aluminum strut for minimised skin pressure

### REQUIRED MEASUREMENTS

- Back length measured from the top of the sacrum up to C7 vertebrae, following the spinal curvature

### SIZING

Please note that the Miami Lumbar Posteo includes a universal lumbar belt which can be trimmed down to the appropriate size.

### ORDERING INFORMATION

#### MIAMI LUMBAR POSTEO

Size	Back Length	Part#
XS	37 - 44cm	LMB-POSTEO-XS
S	44 - 51cm	LMB-POSTEO-SM
M	49 - 56cm	LMB-POSTEO-MD
L	55 - 61cm	LMB-POSTEO-LG

## MP45 SPINAL HYPEREXTENSION BRACE

### INDICATIONS

- Post-operative
- Post-trauma

### FEATURES AND BENEFITS

- Lightweight aluminium construction
- Spring-controlled 'floating' pelvic band continually adjusts as the patient changes from a standing to a sitting position - effectively eliminating the problem of upward brace movement without compromising brace function
- Padded pelvic band may be locked in position
- New fastening system and adjustment slots on the sternal component allow simple and convenient height and width adjustment without component removal
- Angle of the sternal component may also be adjusted without bending to provide optimum positioning against the sternum
- Extended sizing choice allows simple off-the-shelf fitting for a wide variety of patient shapes and sizes



### REQUIRED MEASUREMENTS

- Hip circumference

### ORDERING INFORMATION

Size	Hip Circumference	Min - max length	Part#
S	60 - 75cm	40 - 44cm	MP45/S
M	79 - 90cm	43 - 47cm	MP45/M
L	90 - 105cm	47 - 51cm	MP45/L
L/Short	90 - 105cm	43 - 47cm	MP45/LS
XL	105 - 115cm	50 - 54cm	MP45/XL
XL/Short	105 - 115cm	47 - 51cm	MP45/XLS

## PAPOOSE®

Papoose is an innovative design for newborns up to 3 months (Broselow Grey). This true cervical spine immobiliser cradles the baby's body, securely while gently immobilising the head and spine in the proper anatomical airway and spinal alignment to help achieve optimal patient outcome while allowing for continued intimacy between caregiver and child.



### INDICATIONS FOR USE

- C-Spine precaution for infants
- Immobilisation for pre and post c-spine surgery
- Other conditions requiring gross immobilisation of the infant cervical spine

### FEATURES AND BENEFITS

- X-ray & CT Lucent. MRI Safe

### REQUIRED MEASUREMENTS

- Universal, one size

### ORDERING INFORMATION

#### PAPOOSE

Description	Part#
Papoose without front collar	PO-100
Papoose without front with replacement pads	POR-100
Papoose with front collar	PA-100
Papoose with front with replacement pads	PAR-100

### ACCESSORIES

#### REPLACEMENT PADS

Description	Part#
Papoose replacement pads (with front)	PAP-100
Papoose replacement pads (without front)	POP-100

## FORMFIT® PRO BACK

Össur Formfit® Pro Back provides relief for patients suffering from pains and strains in the lower back. Manufactured with our dynamic, 3D-Motion Tech™ knitting technology, our belts feature superior moisture wicking capabilities and are highly breathable. The patented back panel design utilises our StayBilizer™ technology to provide optimum shape and fit, while conveniently placed finger loops make donning easy. The SmartPad™ provides a targeted support and cushioning and can be removed if desired.



MotionTech™



CoolVent™



StayBilizer™



SmartPad™



### INDICATIONS FOR USE

- For lower back conditions that may benefit from compression and gross motion restriction, such as:
- Lower back pain (Lumbago)
- Lumbosacral sprain/strain
- Degenerative joint/disc disease

### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit
- Removable lumbar SmartPad™ for targeted support and cushioning
- Lightweight, malleable aluminum stays are shaped to the anatomy of the lower back

### REQUIRED MEASUREMENTS

- Measure waist circumference

### ORDERING INFORMATION

Size	Part#	Waist circumference
XS	FP500001	70 - 80cm
S	FP500002	80 - 90cm
M	FP500003	90 - 100cm
L	FP500004	100 - 115cm
XL	FP500005	115 - 130cm



## FORMFIT® BACK SUPPORT AIR

The Form Fit Back Support with Air provides compression and support for those suffering from lower back pain. The integrated pneumatic system is intuitive and conveniently located so that adjustments can be made as demands on the body change throughout the day. The double wrap design improves compression, while durable finger loops make applying the brace easy. Made from breathable materials, the Form Fit Back Support provides compression and support for those suffering from lower back pain. The double wrap design improves compression, while the durable finger loops make applying the brace easy.



### INDICATIONS FOR USE

Lower back conditions that may benefit from gross motion restriction, such as;

- Muscular low back pain
- Lumbosacral strains
- Chronic low back pain
- Sciatica

### FEATURES AND BENEFITS

- Self-contained pneumatic system is low-profile and easy to use
- Mesh panels and ventilation holes increase breathability and enhance comfort
- Overlapping double pull strap design provides additional support to the lumbar and abdominal areas
- Flexible stays provide optimum support in the lumbar and abdominal areas
- Gripper strips prevent migration
- Adjustable air bladder provides a custom fit and offers focused compression in the lumbar area

### REQUIRED MEASUREMENTS

- Waist circumference

## ORDERING INFORMATION

Size	Waist circumference	Standard
S	63 - 76cm	209133
M	76 - 86cm	209135
L	86 - 97cm	209137
XL	97 - 107cm	209138
XXL	107 - 142cm	209139

## ACCESSORIES

Description	Part#
Gel pad	GB28

## FORMFIT® BACK SUPPORT

Made from breathable materials, the Formfit Back Support provides compression and support for those suffering from lower back pain. The double wrap design improves compression, while the durable finger loops make applying the brace easy

### INDICATIONS FOR USE

- Muscular low back pain
- Lumbosacral strains
- Chronic low back pain
- Sciatica

### FEATURES AND BENEFITS

- Anti-migration system reduces slippage
- Double wrap design improves compression
- Finger loops for easy application
- Hot/Cold Gel pad available separately

### REQUIRED MEASUREMENTS

- Waist circumference



## ORDERING INFORMATION

Size	Waist Circumference	Standard
S	25-30"	209143
M	30-34"	209145
L	34-38"	209147
XL	38-42"	209148
XXL	42-56"	209149
XXXL	56-60"	209151

## ACCESSORIES

Description	Part#
Gel pad	GB28



## DUAL MATERNITY BELT

The Dual Maternity Band provides support to the sacroiliac and symphysis pubis to ease sacroiliac, supra pubic or low back pain.

### INDICATIONS FOR USE

- Sacroiliac support
- Low back pain
- Supra pubic support

### FEATURES AND BENEFITS

- The 3-touch-and-close strap configuration allows the patient to obtain optimum fit through a range of strap positions
- The strong, lightweight elastic gives excellent support without being restrictive

### REQUIRED MEASUREMENTS

- Circumference around hips and under bump



## ORDERING INFORMATION

Hip circumference	Part#
90 - 96cm	331/MS
96 - 102cm	332/MS
102 - 108cm	333/MS
108 - 116cm	334/MS
116 - 126cm	335/MS
126 - 136cm	336/MS
136 - 146cm	337/MS
146 - 156cm	338/MS



HIP

---



# TABLE OF CONTENTS

---

HIP .....	41
REBOUND® HIP .....	43
UNLOADER® HIP .....	45



## REBOUND® HIP

Rebound Hip delivers gentle controlling forces and effective functional ROM restriction in extension / flexion and abduction / adduction to promote optimal post-surgical outcomes. The low profile, lightweight design supports patient comfort and compliance. The universal frame and softgood are easy to stock, assemble and quickly fit.



### INDICATIONS FOR USE

Hip conditions that could benefit from motion restriction such as:

- Postoperative after (total) hip arthroplasty
- Postoperative after revision of (total) hip arthroplasty
- Postoperative hip arthroscopy
- Post-op following FAI arthroscopic surgery
- Post-op following gluteus medius repair surgery
- Post-op following hamstring repair surgery
- Inoperable hip abnormalities
- Preventively when there is risk of hip dislocation
- Conventional treatment after dislocation/reduction

### FEATURES AND BENEFITS

- Low-profile, lightweight design that supports patient comfort and compliance
- Quick-release pegs easily adjust the desired range-of-motion needed for the patient's optimal rehab
- Quick-fit with sliding mechanism on frame to adjust for patient's height
- Optional, adjustable ROM hinge enables adduction/abduction from 0° to 10°, 20° or 30°
- OR kit offers patient protection of ankle and perineum skin during intraoperative traction on a surgical table

### REQUIRED MEASUREMENTS

- Circumference at iliac crest height
- Length from greater trochanter to mid patella

### ORDERING INFORMATION

#### REBOUND HIP

Description	Part#
Rebound Hip	98013XRB
Rebound Hip with adjustable hinge	98012XRB

X = 0 (Preset to left) or 5 (Preset to right)

\*Frame: Universal; Softgood: Universal fitting to 122cm hip circumference

# REBOUND® HIP

## ACCESSORIES

Description	Part#
Rebound Hip alligator kit	980501
Rebound Hip v2 softgood kit	980601
Rebound Hip v2 belt only kit	980701
OR Kit	980301
Belt Extender (Fitting 122 - 170cm hip circumference)	980121RB
Rotation Strap	RHK-100024
Fitting Tool	B-040100000

## UNLOADER® HIP

Unloader Hip is designed to improve activities of daily living by optimising load dispersion for patients suffering from mild and moderate osteoarthritis of the hip, thus contributing to hip stability and improved mobility.

Unloader Hip delivers proprioceptive support during the gait cycle by providing compression with a proven pulley system design. If required it can decrease the internal rotation and adduction moment with its unique and clinically proven Dynamic Rotation Straps™.



### INDICATIONS FOR USE

Conditions benefitting from hip repositioning and/or compression around the greater trochanters, this may include:

- Hip osteoarthritis (OA)
- Femoroacetabular impingement (FAI)
- Labrum tears

### FEATURES AND BENEFITS

- The Dynamic Rotation Straps with SmartDosing dial allows individual adjustment of the external rotation force, to achieve optimum pain relief for each patient
- Easy donning and doffing thanks to pocket wrap closure system and zipper
- Pulley system for improved stability
- Premium quality Lycra® for breathability and a snug fit


### REQUIRED MEASUREMENTS

- Measure the hip circumference at the anterior superior iliac-spine.

### ORDERING INFORMATION

#### UNLOADER HIP

Size	Hip Circumference	Woman	Men
S	80 – 88cm	B-465350030	B-465350031
M	88 – 96cm	B-465350032	B-465350033
L	96 – 104cm	B-465350034	B-465350035
XL	104 – 114cm	B-465350036	B-465350037
XXL	114 – 124cm	B-465350038	B-465350039

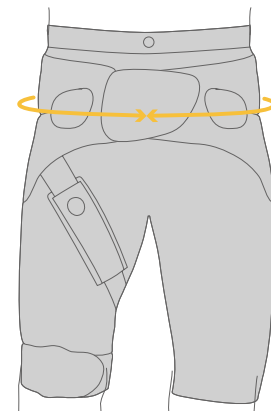



# UNLOADER® HIP

## ACCESSORIES/ OPTIONS

### REPLACEMENT TROUSERS

Size	Woman	Men
S	B-465350077	B-465350078
M	B-465350079	B-465350080
L	B-465350081	B-465350082
XL	B-465350083	B-465350084
XXL	B-465350085	B-465350086

### OTHER ACCESSORIES

Description	Part#
Rotation strap	B-465350064
Washbag Kit	WB000001
Trochanter pad	B-465350066
Thigh strap S	B-465350067
Thigh strap M	B-465350068
Thigh strap L	B-465350069
Thigh strap XL	B-465350070
Thigh strap XXL	B-465350071
Bilateral kit* S	B-465350072
Bilateral kit* M	B-465350073
Bilateral kit* L	B-465350074
Bilateral kit* XL	B-465350075
Bilateral kit* XXL	B-465350076

\*The bilateral kit consists of a rotation control strap, a thigh strap and a trochanter pad

## UPPER EXTREMITY

---



## TABLE OF CONTENTS

UPPER EXTREMITY .....	47
FORMFIT® SHOULDER BRACE .....	49
PREMIUM SHOULDER IMMOBILISER .....	50
BERGSCHULTZ UNIVERSAL ARM SLING .....	50
HIGH ARM SLING .....	51
PAEDIATRIC UNIVERSAL ARM SLING .....	51
FORMFIT® PRO ELBOW .....	52
REBOUND® POST-OP ELBOW .....	53
FORMFIT® TENNIS ELBOW .....	55
FORMFIT® PRO WRIST .....	56
FORMFIT® WRIST / FORMFIT® WRIST & FOREARM .....	57
SOFTPRO® FUNCTIONAL RESTING WHFO .....	58
SOFTPRO® GRIP WHFO .....	59
FORMFIT® UNIVERSAL WRIST & THUMB .....	60
FORMFIT® THUMB SPICA .....	61
WRIST / THUMB BRACE .....	62
ELASTIC THUMB SPICA .....	62
SPECTRA® WRIST BRACE .....	63

## FORMFIT® SHOULDER BRACE

The Formfit Shoulder Brace is designed to provide immobilisation following shoulder injury/surgery. With its streamlined fit, sporty look and breathability it offers excellent comfort and ease of use. Same as its predecessor, the SmartSling®, the Formfit Shoulder Brace also provides abduction of the shoulder (when the abduction pillow is attached), or external rotation of the shoulder (when the external rotation pillow is attached).

Optional axilla pillow helps distribute weight over a larger surface area.

### INDICATIONS FOR USE

- Immobilisation for rotator cuff repairs
- Bankart repairs
- Capsular shifts
- Glenohumeral dislocations/subluxations
- Soft tissue repairs/strains
- Shoulder instabilities/injuries
- Post-surgical rehabilitation

### FEATURES AND BENEFITS

- Breathable material
- Quick-release buckles
- Offloading strap
- Extra padding
- Universal thumb loop

### REQUIRED MEASUREMENTS

- Olecranon to end of closed fist



Formfit Shoulder Brace

Axilla Pillow

## ORDERING INFORMATION

### FORMFIT SHOULDER BRACE

Size	Length	Part
S	up to 29cm	SSS-SO
M	29 - 40cm	SSR-SO
L	40 - 44.5cm	SSL-SO

### FORMFIT SHOULDER BRACE WITH ABDUCTION

Size	Length	Part
S	up to 29cm	SSS
M	29 - 40cm	SSR
L	40 - 44.5cm	SSL

### FORMFIT SHOULDER BRACE WITH EXTERNAL ROTATION

Size	Length	Part
S	up to 29cm	SSS-ER
M	29 - 40cm	SSR-ER
L	40 - 44.5cm	SSL-ER

Description	Part
Axilla Pillow	SSAXP

## PREMIUM SHOULDER IMMOBILISER

### INDICATIONS FOR USE

For shoulder conditions that may benefit from immobilising the shoulder, such as:


- Pre or post-operative
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

### FEATURES AND BENEFITS

- Effectively controls abduction and rotation
- Humerus cuff, wrist cuff and shoulder strap may be placed in desired position for optimum fit/comfort
- One size for reduced stockholding
- Fits sizes S-XL

### REQUIRED MEASUREMENTS

- Universal, one size (for upper waist diameter up to 106cm)

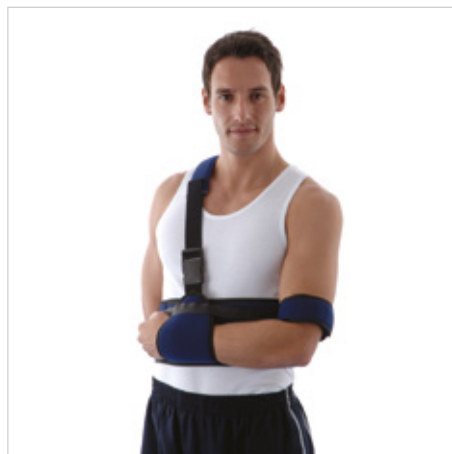

### ORDERING INFORMATION

Description	Part#
Premium Shoulder Immobiliser	0380
Premium Shoulder Immobiliser XL	0380XL

## BERGSCHULTZ UNIVERSAL ARM SLING

### INDICATIONS FOR USE

- Post-trauma
- Post-operative

### FEATURES AND BENEFITS

- Designed by Jim Bergschultz. R.O.T.
- Blue heavy denim material with contact closure

### REQUIRED MEASUREMENTS

- Universal, one size



### ORDERING INFORMATION

Description	Part#
Bergschulz Universal Arm Sling	204070



## HIGH ARM SLING

### INDICATIONS FOR USE

- Post upper limb trauma
- Post upper limb surgery
- Hand surgery

### FEATURES AND BENEFITS

- Maintains upper arm in raised position
- Breathable nylon/foam laminate construction
- Easy to apply with touch and close fastening
- Waist strap included



### ORDERING INFORMATION

Description	Part#
High Arm Sling	0860

## PAEDIATRIC UNIVERSAL ARM SLING

### INDICATIONS FOR USE

- Post upper limb trauma
- Post upper limb surgery

### FEATURES AND BENEFITS

- Prong buckle closure correct fit without slippage

### REQUIRED MEASUREMENTS

- Universal, one size



### ORDERING INFORMATION

Description	Part#
Paediatric Universal Arm Sling Packed in boxes of 6	204020

## FORMFIT® PRO ELBOW

Formfit® Pro Elbow provides relief from elbow pain. This 3D compressive elbow sleeve is manufactured using our MotionTech™ technology and features two integrated epicondyle SmartPads™ for more targeted support and cushioning. The RadialStrap™ provides additional compression and support.



MotionTech™



CoolVent™



SmartPad™



FoldLock™

### INDICATIONS FOR USE

- For elbow conditions that may benefit from compression, such as:
- Medial and lateral epicondylitis
- Elbow pain and swelling


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### FEATURES AND BENEFITS

- 3D MotionTech™ for compression and dynamic fit keep
- Epicondyle SmartPads™ for targeted support and cushioning
- Patented FoldLock™ system for easier and more secure application
- RadialStrap™ for increased compression and support

### REQUIRED MEASUREMENTS

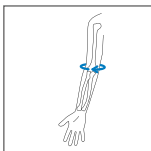

Measure arm circumference exactly at the elbow. Elbow in full extension.

### ORDERING INFORMATION

Size	Elbow circumference	Part#
XS	17 - 20cm	FP300001
S	20 - 23cm	FP300002
M	23 - 26cm	FP300003
L	26 - 29cm	FP300004
XL	29 - 32cm	FP300005





## REBOUND® POST-OP ELBOW

The Rebound® Post-Op Elbow brace by Össur combines intuitive features for fitters and users with a design that aims to improve the post-op experience. The industry's only elbow immobiliser that combines a drop-lock hinge and four telescoping paddles, individual post-op protocols are easily catered for. The Rebound Post-Op Elbow, lightest of its kind, promotes user comfort with straps that stay in place, anatomical left and right fit with mouldable stability cuffs and blue patient touch-points for easy education.



### INDICATIONS FOR USE

Indications requiring elbow immobilisation and/or range of movement restriction. This may include:

- Stable fractures of the elbow, distal humerus, proximal radius or ulna
- Tendon and ligament injuries or repairs
- Collateral ligament reconstructions
- Elbow hyperextension
- Triceps tendon repair
- Chronic elbow injuries
- Tennis elbow release

### FEATURES AND BENEFITS

- Designed to be the most lightweight brace on the market
- Telescoping paddles relocate to individualise length and keep incision sites clear
- Easy education for users with intuitive patient touchpoints, including Blue is You™ buckles and numbered strapping system
- Drop-Lock hinge enables secure range of movement with bump-resistant pull adjustment stops and optional ROM lock-outs
- Mouldable stability cuffs designed to be individually customised for improved fit and support
- Strap guides maintain strap location for added security and comfort
- Universal fit for both left and right elbows

### REQUIRED MEASUREMENTS

- Universal, one size

## REBOUND® POST-OP ELBOW

### ORDERING INFORMATION

Side	Part#
Left	B-8391680
Right	B-8391580

### ACCESSORIES

Description	Part#
Elbow Wrist Control Kit	B-8390003
Elbow ROM Locks Kit	B-8390005

### REPLACEMENT PARTS

Description	Part#
Elbow Liner Kit	B-8390002
Elbow Sling Kit	B-8390004
Strap Tab Kit	B-8390006
Adhesive Hook Sticker Kit	B-8390007
Anti-Migration Pad Kit	B-8390008
Elbow Strap Kit	B-8390009

## FORMFIT® TENNIS ELBOW

Formfit Tennis Elbow is made with a patented, ergonomically-shaped plastic. There are two blue cushions with ribbed surface on each side that reduce traction on the tendons. The durable design and materials enable everyday use for both work and athletic activities.

### INDICATIONS FOR USE

- Tendonitis of the elbow
- Prevention during work, daily life and sport
- Epicondylitis

### FEATURES AND BENEFITS

- Adjustable strap
- Patented ergonomically-shaped plastic eliminates tourniquet effect associated with traditional designs.
- Twin blue cushions provide counterforce, reducing traction on the tendons and thus inflammation
- Ribbed surfaces reduce vibration to the tendon, increasing comfort
- Durable design and materials enable everyday use for both work and athletic activities

### REQUIRED MEASUREMENTS

- Circumference of the elbow



## ORDERING INFORMATION

Size	Circumference	Part#
S	22-26cm	B-247000002
M	27-32cm	B-247000003
L	33-38cm	B-247000004

## FORMFIT® PRO WRIST

Formfit Pro Wrist is constructed using our superior signature 3D MotionTech knit, and ideal for injuries that may benefit from compression. Malleable aluminum stays can be adjusted to conform to an individual's anatomy. CoolVent™ technology in the basal thumb area enhances breathability and comfort.



MotionTech™



CoolVent™



StayBilizer™



FoldLock™



### INDICATIONS FOR USE

- For wrist conditions that may benefit from compression, such as:
- Rheumatoid Arthritis (RA) and other chronic pain conditions
- Wrist sprain and strains
- Overuse and pain conditions
- Wrist tendonitis


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit has been proven to be superior to the leading brands at moisture wicking and breathability, keeping patients cool during wear
- Malleable aluminum stay for a customised fit
- CoolVent™ technology in thumb base area for extra breathability and comfort

### REQUIRED MEASUREMENTS

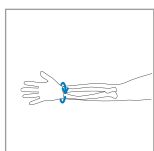

Measure wrist circumference at the level of the radial styloid with the hand outstretched. Wrist in 0 degrees of flexion.

### ORDERING INFORMATION

Size	Wrist circumference	Left	Right
XS	14 - 15cm	FP400201	FP400101
S	15 - 16cm	FP400202	FP400102
M	16 - 17cm	FP400203	FP400103
L	17 - 18cm	FP400204	FP400104
XL	18 - 19cm	FP400205	FP400105
XXL	19 - 20cm	FP400206	FP400106



## FORMFIT® WRIST / FORMFIT® WRIST & FOREARM

Formfit Wrist, and Formfit Wrist & Forearm, provides immobilisation and protection for patients suffering from tendonitis, select soft tissue injuries – including mild to moderate sprains and strains, as well as for post-cast healing. The lightweight, breathable material and contour fit make this comfortable product an excellent choice.

### INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other selected soft tissue injuries

### FEATURES AND BENEFITS

- Breathable material
- Contoured fit
- Contact closure straps
- Customisable stays

### REQUIRED MEASUREMENTS

- Wrist circumference



## ORDERING INFORMATION

### FORMFIT WRIST & FOREARM

Size	Wrist Circumference	8"		10"	
		Left	Right	Left	Right
XS	13 - 16cm	317082	317072	617082	617072
S	16 - 18.5cm	317083	317073	617083	617073
M	18.5 - 20.5cm	317085	317075	617085	617075
L	20.5 - 23.5cm	317087	317077	617087	617077
XL	23.5 - 25.5cm	317088	317078	617088	617078

## ACCESSORIES

### GEL PAD FOR HOT/COLD THERAPY

Description	Left	Right
Form Fit Wrist (8" version)*	GB20L	GB20R
Form Fit Wrist & Forearm (10" version)	GB22L	GB22R



\* Fits XS/S. For M-XL, please use Form Forearm 10" part numbers



## SOFTPRO® FUNCTIONAL RESTING WHFO

The SoftPro® Functional Wrist/Hand/Finger Orthosis (WHFO) is clinically indicated for providing support and protection of a weak or injured wrist, hand, fingers, including thumb. The “bend to fit” endoskeleton frame can be hand shaped to accommodate the desired angle of support for the wrist, hand, fingers, and thumb. The endoskeleton frame can be modified in the sagittal plane as well as medially, laterally, and rotationally to accommodate most angles of the wrist/hand/and fingers without the use of tools. Each endoskeleton frame is closed cell foam. The endoskeleton frame is a closed cell foam, covered by a laminated foam/cloth cover that can be laundered as necessary.



### INDICATIONS FOR USE

- Support and protection of a weak or injured wrist, hand, fingers, including thumb

### FEATURES AND BENEFITS

- Extra long wrist strap holds the wrist in the proper position and applies a gentle stretch
- Maintains/supports the wrist/hand/fingers in a comfortable functional position
- “Bend to Fit” – Easily adjust to correct alignment & support joints without the use of a heat gun

### REQUIRED MEASUREMENTS

- Forearm-fingertip length
- Hand width at metacarpals

## ORDERING INFORMATION

Part#	Description	Width of MP Joints	Length
SP31FFR-C-L	SoftPro Rest Child Left	4.5 - 5cm	18cm
SP31FFR-C-R	SoftPro Rest Child Right	4.5 - 5cm	18cm
SP31FFR-Y-L	SoftPro Rest Youth Left	5 - 6.5cm	20cm
SP31FFR-Y-R	SoftPro Rest Youth Right	5 - 6.5cm	20cm
SP31FFR-S-L	SoftPro Rest Small Left	6.5 - 7.5cm	23cm
SP31FFR-S-R	SoftPro Rest Small Right	6.5 - 7.5cm	23cm
SP31FFR-M-L	SoftPro Rest Medium Left	7.5 - 9cm	25cm
SP31FFR-M-R	SoftPro Rest Medium Right	7.5 - 9cm	25cm
SP31FFR-L-L	SoftPro Rest Large Left	9 - 10cm	28cm
SP31FFR-L-R	SoftPro Rest Large Right	9 - 10cm	28cm

## SOFTPRO® GRIP WHFO

The SoftPro Grip WHFO “bend to fit” endoskeleton frame can be hand shaped to accommodate the desired angle of support for the wrist, hand, fingers, and thumb. The endoskeleton frame can be modified in the sagittal plane as well as medially, laterally, and rotationally to accommodate most angles of the wrist/hand/and fingers without the use of tools. Each endoskeleton frame is closed cell foam. The endoskeleton frame is a closed cell foam, covered by a laminated foam/ cloth cover that can be laundered as necessary.

### INDICATIONS FOR USE

- Provides necessary support and protection of a weak or injured wrist and/or hand and/or fingers, including the thumb

### FEATURES AND BENEFITS

- Two incrementally larger palmar rolls included to gradually increase extension of the MCP and PIP joints
- Maintains/supports the wrist/hand/fingers in a comfortable position
- “Bend to Fit” – Easily adjust to correct alignment & support joints without the use of a heat gun

### REQUIRED MEASUREMENTS

- Forearm-fingertip length
- Hand width at metacarpals



## ORDERING INFORMATION

Part#	Description	Width of MP Joints	Length
SP31FG-C-L	SoftPro Grip Child Left	4.5 - 5cm	18cm
SP31FG-C-R	SoftPro Grip Child Right	4.5 - 5cm	18cm
SP31FG-Y-L	SoftPro Grip Youth Left	5 - 6.5cm	20cm
SP31FG-Y-R	SoftPro Grip Youth Right	5 - 6.5cm	20cm
SP31FG-S-L	SoftPro Grip Small Left	6.5 - 7.5cm	23cm
SP31FG-S-R	SoftPro Grip Small Right	6.5 - 7.5cm	23cm
SP31FG-M-L	SoftPro Grip Medium Left	7.5 - 9cm	25cm
SP31FG-M-R	SoftPro Grip Medium Right	7.5 - 9cm	25cm
SP31FG-L-L	SoftPro Grip Large Left	9 - 10cm	28cm
SP31FG-L-R	SoftPro Grip Large Right	9 - 10cm	28cm



## FORMFIT® UNIVERSAL WRIST & THUMB

Formfit Universal Wrist and Thumb products are designed to restrict motion of the wrist and the thumb. Providing immobilisation and protection from select soft tissue injuries, mild to moderate strains and sprains, including gamekeeper's thumb and post cast healing. The lightweight, universal design also reduces inventory needs.

### INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Post-cast healing
- Other selected soft tissue injuries

### FEATURES AND BENEFITS

- Breathable material
- Quick-Pull close strap
- Positional dorsal stay
- Customisable thumb stay
- Malleable palmar stay

### REQUIRED MEASUREMENTS

- Wrist circumference



## ORDERING INFORMATION

### FORMFIT WRIST

Size	Wrist Circumference	Left	Right
Paediatric	3.5 - 5"	B-252603301	B-252603300
8" Universal	4.5 - 9.5"	B-252603109	B-252503109
8" XL	7.5 - 11"	B-252603106	B-252603105

### FORMFIT WRIST & FOREARM UNIVERSAL

Size	Wrist Circumference	Left	Right
10" Universal	4.5 - 9.5"	B-252603210	B-252603209

### FORMFIT THUMB SPICA UNIVERSAL

Size	Wrist Circumference	Left	Right
Paediatric	3.5 - 5"	B-253603301	B-253603300
8" Universal	4.5 - 9.5"	B-253603109	B-253503109
8" XL	7.5 - 11"	B-253603106	B-253603105

## FORMFIT® THUMB SPICA

The Formfit Thumb Spica is designed to restrict wrist and thumb motion. It provides immobilisation and protection for patients suffering from tendonitis (e.g. DeQuervain's Tendonitis), select soft tissue injuries – including mild to moderate sprains and strains and Gamekeeper's Thumb –, as well as for post-cast healing. The lightweight, breathable material and contour fit make this comfortable product an excellent choice.

### INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
  - e.g., DeQuervain's Tendonitis
- Post-cast healing
- Other selected soft tissue injuries
  - e.g., Gamekeeper's Thumb

### FEATURES AND BENEFITS

- Breathable material
- Contoured fit
- Contact closure straps
- Customisable stays

### REQUIRED MEASUREMENTS

- Wrist circumference



## ORDERING INFORMATION

Size	Wrist Circumference	Left	Right
XS	13 - 16cm	3020	3120
S	16 - 18.5cm	3030	3130
M	18.5 - 20.5cm	3050	3150
L	20.5 - 23.5cm	3070	3170
XL	23.5 - 25.5cm	3080	3180

## ACCESSORIES

### GEL PAD FOR HOT/COLD THERAPY

Description	Left	Right
Form Fit Wrist (8" version)*	GB20L	GB20R
Form Fit Wrist & Forearm (10" version)	GB22L	GB22R



\* Fits XS/S. For M-XL, please use Form Forearm 10" part numbers

## WRIST / THUMB BRACE

A simple yet effective wrist brace offering superior comfort.

### INDICATIONS FOR USE

- Stabilising hyper-or hypotonic wrist/thumb flexors
- Wrist / Thumb over use injury
- Wrist / Thumb trauma

### FEATURES AND BENEFITS

- Simple touch-and-close fastenings
- Full length malleable aluminium thumb stay

### REQUIRED MEASUREMENTS

- Wrist circumference



## ORDERING INFORMATION

		Elastic		Neoprene	
Size	Wrist circumference	Left	Right	Left	Right
XS	11 - 13cm	150/ET	155/ET	160/NT	165/NT
S	13 - 15cm	151/ET	156/ET	161/NT	166/NT
M	15 - 17cm	152/ET	157/ET	162/NT	167/NT
L	17 - 19cm	153/ET	158/ET	163/NT	168/NT
XL	19 - 21cm	154/ET	159/ET	164/NT	169/NT

## ELASTIC THUMB SPICA

The Elastic Thumb Spica is designed to protect the thumb from unwanted motion whilst allowing full range of motion of the fingers. Made from elastic, Thumb Spica is simple and very easy to fit. The thumb spica is designed to restrict movement of the thumb post injury, such as a broken thumb, which aids in the recuperation process. It also offers moderate and flexible control following CMC ligament sprains or strains.

### KEY BENEFITS:

Constructed from elastic, simple design with spiral thumb stay, restricts movement of first MCP and CMC joints, touch-and-close fastenings

### INDICATIONS FOR USE

- CMC ligament sprains or strains requiring moderate, flexible control



## ORDERING INFORMATION

Size	Wrist circumference	Left	Right
S	13 - 17cm	191/TS	196/TS
M	17 - 21cm	192/TS	197/TS
L	21 - 25cm	193/TS	198/TS
XL	25 - 29cm	194/TS	199/TS

## SPECTRA® WRIST BRACE

The Spectra wrist brace is designed to immobilise the wrist at a neutral angle. Its single-pull lacing closure makes it easier to apply than a conventional wrist brace, meeting the needs of patients with arthritis and other hand dexterity issues.

### INDICATIONS FOR USE

- Mild to moderate sprains and strains
- Tendonitis
- Carpal tunnel and other repetitive stress injuries
- Post cast healing

### FEATURES AND BENEFITS

- Wicking liner moves moisture away from the skin for increased comfort
- Single-pull lace closure enables easy application
- Adjustable webspace strap allows size adjustment for increased comfort
- Removable/adjustable palmar stay provides adjustability of immobilisation angle
- Suede soft good material is durable and breathable

### REQUIRED MEASUREMENTS

- Wrist circumference



## ORDERING INFORMATION

Size	Wrist circumference	Left	Right
XS	12 - 14cm	6602210	6602315
S	14 - 16cm	6602422	6602535
M	16 - 18cm	6602624	6602734
L	18 - 20cm	6602755	6602793
XL	20 - 22cm	6602826	6602932





KNEE





## TABLE OF CONTENTS

KNEE .....	65
UNLOADER ONE® X .....	67
UNLOADER ONE® X CUSTOM .....	71
UNLOADER ONE® .....	74
UNLOADER ONE® CUSTOM .....	77
UNLOADER ONE® PLUS .....	78
REBOUND® CARTILAGE .....	80
CTI® OA .....	82
REBOUND® PCL .....	84
REBOUND® ACL .....	88
REBOUND® DUAL BASIC AND REBOUND® DUAL BASIC ST .....	92
FORMFIT® OA EASE .....	94
CTI® OTS .....	96
CTI® CUSTOM .....	99
PARADIGM® .....	102
FLEX™ .....	103
C180™ ROCKET .....	105
REBOUND® POST-OP KNEE .....	106
FORMFIT POST-OP KNEE .....	107
REBOUND® KNEE BRACE .....	108
REBOUND® KNEE IMMOBILIZER UNIVERSAL .....	109
FORMFIT® PRO KNEE .....	110
FORMFIT® TRACKER .....	112
FORMFIT® PRO KNEE FLITE .....	113
FORMFIT® HINGED LATERAL J .....	114
FORMFIT® KNEE HINGED .....	115
FORMFIT® KNEE MCL .....	116
FORMFIT® KNEE ROM BRACE .....	117
NEOPRENE HINGED KNEE SUPPORT .....	118
NEOPRENE KNEE SUPPORT WITH STABILISED PATELLA .....	118
NEOPRENE WRAPAROUND HINGED KNEE SUPPORT .....	119
NEOPRENE KNEE SLEEVE .....	120
MEASUREMENT FORM CTi® CUSTOM BRACE .....	121
MEASUREMENT FORM UNLOADER ONE® CUSTOM / REBOUND® ACL & PCL .....	124



## UNLOADER ONE® X

Unloader One X improves mobility by providing pain relief from knee osteoarthritis and degenerative meniscal tears. It features Össur's unique patented and clinically proven 3-point Leverage System that generates unicompartmental unloading of the knee joint.



### INDICATIONS FOR USE


- Moderate to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Other unicompartmental knee conditions that may benefit from unloading, such as:
  - Articular cartilage defect repair
  - Meniscal repair
  - Avascular necrosis
  - Condylar bone marrow lesions
  - Tibial plateau fractures

### FEATURES AND BENEFITS

- 3-Point Leverage System consisting of two Dynamic Force System (DFS) Straps, rigid shells and a single upright hinge contribute to a unicompartmental unloading of the knee joint.
- The unloading is effective only when needed, no condyle pressure during sitting or swing phase.
- A slender perforated frame with soft edges, contours easily to individual anatomies and remains discreet whether worn over or under clothing.
- Skin friendly liners and pads as well as a smooth inner surface of shells reduce probability of pressure points.
- The gastroc strap is positioned over thickest part of calf to help prevent migration.
- Tool-free SmartDosing® dials enable the patient to be in control of their pain level by fine-tuning the strap tension.
- Colour coded touchpoints and Quick-Fit buckles increase user-friendliness.
- Alignment guide on hinge facilitates correct positioning on the leg.
- Entire brace is machine washable (washbag included with product).
- Easily adaptable to individual needs and anatomical shapes.
- Field serviceable.

### REQUIRED MEASUREMENTS

Size	Calf circumference 15cm below mid patella
XS-M	25 - 43cm
L-XL	43 - 62cm

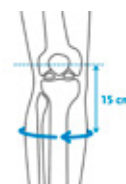

Measure calf circumference  
15cm below mid-patella.  
Leg in extension.


# UNLOADER ONE® X

## ORDERING INFORMATION

### SHORT

Size	Right Medial	Left Medial	Right Lateral	Left Lateral
XS-M	UOX1518713	UOX1618713	UOX1528713	UOX1628713
L-XL	UOX1518715	UOX1618715	UOX1528715	UOX1628715


Recommended for patients with an inseam measurement <85cm. Hinge is placed on the side of the affected compartment.

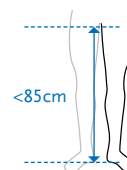

### STANDARD

Size	Right Medial	Left Medial	Right Lateral	Left Lateral
XS-M	UOX0518713	UOX0618713	UOX0528713	UOX0628713
L-XL	UOX0518715	UOX0618715	UOX0528715	UOX0628715

Recommended for patients with an inseam measurement >85cm. Hinge is placed on the side of the affected compartment.

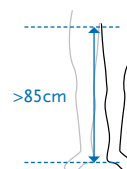

## ACCESSORIES

### EASE-OF-USE KIT

Description	Part#
Kit (Sundial & Buckle finger loop)	UOX800001

### DOESKIN THIGH LINER, GREY

Right Medial, Left Lateral	Right Lateral, Left Medial
UOX526101	UOX525101

### SENSIL THIGH LINER, GREY

Right Medial, Left Lateral	Right Lateral, Left Medial
UOX526201	UOX525201

### DOESKIN THIGH EXTENSION LINER, GREY

Right Medial, Left Lateral	Right Lateral, Left Medial
UOX546101	UOX545101

### SENSIL THIGH EXTENSION LINER, GREY

Right Medial, Left Lateral	Right Lateral, Left Medial
UOX546201	UOX545201

## UNLOADER ONE® X

### ÖSSUR ONE SLEEVE, GREY

Size	Part#
XXS	OSO60011
XS	OSO60012
S	OSO60013
M	OSO60014
L	OSO60015
XL	OSO60016
XXL	OSO60017
XXXL	OSO60018

Description	Part#
Strap wrap set (calf+thigh), grey	UOX700094
Antimigration pillar grey LL/RM	UOX700025
Antimigration pillar grey RL/LM	UOX700045
Hinge pad, grey	UOX700011
Extension stop kit	B-301040047
Sensil strap pad, grey (10 pcs)	UOX700193
Sundial	UOX1200001

## REPLACEMENT PARTS

Description	Part#
Popliteal pad, grey short	UOX700082
Popliteal pad, grey	UOX700092
Alligator kit, grey - 2x DFS alligators with overmold - 1 thigh strap alligator - 1 calf strap alligator	UOX900002
SmartDosing kit, grey (1pc)	UOX1000002

### UPPER DFS AND CALF STRAP WITH BUCKLE, GREY

Right Medial, Left Lateral	Right Lateral, Left Medial
UOX406101	UOX405101

### LOWER DFS AND THIGH STRAP WITH BUCKLE, GREY

Right Medial, Left Lateral	Right Lateral, Left Medial
UOX406102	UOX405102

# UNLOADER ONE® X

## REPLACEMENT PARTS

### SENSIL CALF LINER, GREY

Size	Right Medial, Left Lateral	Right Lateral, Left Medial
XS-M	UOX516152	UOX515152
L-XL	UOX516162	UOX515162

### ACTIVEGRIP CALF LINER, GREY

Size	Right Medial, Left Lateral	Right Lateral, Left Medial
XS-M	UOX516153	UOX515153
L-XL	UOX516163	UOX515163

### DOESKIN CALF LINER, GREY

Size	Right Medial, Left Lateral	Right Lateral, Left Medial
XS-M	UOX516151	UOX515151
L-XL	UOX516161	UOX515161

#### Note:

#### SIZE SELECTION GUIDANCE CALF LINERS

Due to a recent upgrade of the calf size dimensions of medium sized braces, please consider the following when ordering replacement liners:

- If calf liner size XS-M is indicated in thigh shell inscription, order size XS-M

- If no such inscription is indicated in thigh shell, order size L-XL

## UNLOADER ONE® X CUSTOM

The Unloader One X Custom improves patient mobility by providing relief from knee osteoarthritis (OA) and degenerative meniscal tears.

Its patented 3-Point Leverage System builds on decades of patient and practitioner experience and is designed with ease-of-use and effortless wear in mind.

The Unloader One X Custom is clinically proven to provide pain relief and functional improvements for unicompartmental OA of the knee and degenerative meniscal tears and is custom manufactured to fit your patient's unique leg contour.



### INDICATIONS FOR USE

- Moderate to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Other knee conditions that may benefit from unicompartmental unloading

### FEATURES AND BENEFITS

- Relieves Pain - 3-Point Leverage System with Dynamic Force System™ straps, rigid shells, and a single upright hinge biomechanically unloads the affected knee compartment
- Easy-to-Wear - Slender frame with soft edges easily contours to your leg and remains discreet whether worn under or over clothing
- Comfortable - Skin-friendly liner feels great against the skin and reduces pressure points.  
Anti-Migration Technology - Strap design captures the thickest part of the calf to reduce brace migration during activity
- Control - SmartDosing® dials put you in control of your pain by allowing you to fine-tune strap tension
- Easy Care - Machine washable with convenient washbag included

### SMARTMEASURE®


A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required.

### CUSTOM STANDARD COLOURS

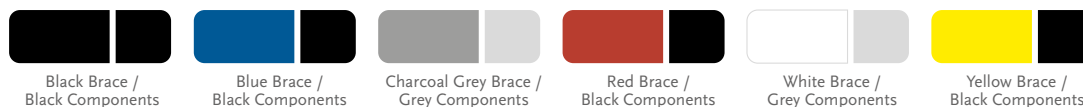

Actual colours may vary.

### ORDERING INFORMATION

Orientation	GlobalSkuCode
Left	UOX0L
Right	UOX0R

Standard and short sizes available

# UNLOADER ONE® X CUSTOM

## ACCESSORIES

### FLEXION/EXTENSION STOP KIT

Colour	GlobalSkuCode
Black	B-301040047

### DOESKIN CALF LINER

Orientation	Size	GlobalSkuCode - GREY	GlobalSkuCode - BLACK
Right Medial, Left Lateral	XS-S	UOX516151	UOX516951
Right Medial, Left Lateral	M-XL	UOX516161	UOX516961
Right Lateral/Left Medial	XS-S	UOX515151	UOX515951
Right Lateral/Left Medial	M-XL	UOX515161	UOX515961

### ACTIVEGRIP CALF LINER

Orientation	Size	GlobalSkuCode - GREY	GlobalSkuCode - BLACK
Right Medial, Left Lateral	XS-S	UOX516153	UOX516953
Right Medial, Left Lateral	M-XL	UOX516163	UOX516963
Right Lateral/Left Medial	XS-S	UOX515153	UOX515953
Right Lateral/Left Medial	M-XL	UOX515163	UOX515963

### SENSIL STRAP PAD (1 PC)

GlobalSkuCode - GREY	GlobalSkuCode - BLACK
UON0182	UON0181

### SENSIL STRAP PAD (10 PCS)

GlobalSkuCode - GREY	GlobalSkuCode - BLACK
UOX700193	UOX70113

### EASE-OF-USE KIT

GlobalSkuCode
UOX800001

Includes: Sundial and buckle finger loops

### THIGH EXTENSION LINER

Orientation	GlobalSkuCode - GREY	GlobalSkuCode - BLACK
Right Medial, Left Lateral	UOX546101	UOX546901
Right Lateral/Left Medial	UOX545101	UOX545901

### SLEEVE

Size	GlobalSkuCode - BLACK - Unloader One X	GlobalSkuCode - BLACK - Össur
XS	UOX60001	B-250150101
S	UOX60002	B-250150102
M	UOX60003	B-250150103
L	UOX60004	B-250150104
XL	UOX60005	B-250150105
XXL	UOX60006	B-250150106

Unloader One X Sleeve: 78% Cotton/14% Spandex/8% Polyester; Össur Sleeve: 80% Nylon/20% Spandex

### SUNDIAL

GlobalSkuCode
UOX1200001

# UNLOADER ONE® X CUSTOM

## REPLACEMENT PARTS

### SENSIL CALF LINER

Orientation	Size	GlobalSkuCode - GREY	GlobalSkuCode - BLACK
Right Medial, Left Lateral	XS-S	UOX516152	UOX516952
Right Medial, Left Lateral	M-XL	UOX516162	UOX516962
Right Lateral/Left Medial	XS-S	UOX515152	UOX515952
Right Lateral/Left Medial	M-XL	UOX515162	UOX515962

### DOESKIN THIGH LINER

Orientation	GlobalSkuCode - GREY	GlobalSkuCode - BLACK
Right Medial, Left Lateral	UOX526101	UOX526901
Right Lateral/Left Medial	UOX525101	UOX525901

### HINGE PAD

GlobalSkuCode - GREY	GlobalSkuCode - BLACK
UOX700011	UOX700091

### POPLITEAL PAD

GlobalSkuCode - GREY	GlobalSkuCode - BLACK
UOX700092	UOX700012

### ALLIGATOR CLIPS

GlobalSkuCode - GREY	GlobalSkuCode - BLACK
UOX900002	UOX900001

Includes 4 Alligator Clips: 2DFS, 1 thigh and 1 calf

### UPPER DFS AND CALF STRAP WITH BUCKLE

Orientation	GlobalSkuCode - GREY	GlobalSkuCode - BLACK
Right Medial, Left Lateral	UOX406101	UOX406901
Right Lateral/Left Medial	UOX405101	UOX405901

Includes: Calf & DFS strap with buckle

### LOWER DFS AND THIGH STRAP WITH BUCKLE

Orientation	GlobalSkuCode - GREY	GlobalSkuCode - BLACK
Right Medial, Left Lateral	UOX406102	UOX406902
Right Lateral/Left Medial	UOX405102	UOX405902

Includes: Thigh & DFS strap with buckle

### SMARTDOSING DIAL

GlobalSkuCode - GREY	GlobalSkuCode - BLACK
UOX1000002	UOX1000001



# UNLOADER ONE®

Unloader One® is designed to relieve osteoarthritis knee pain and wearing it may help improve mobility. It features Össur's unique, patented 3-point leverage system, which combines a single hinge with dual Dynamic Force System (DFS) straps to provide unloading of the knee joint. Clinical studies demonstrate that this system may relieve osteoarthritis knee pain and improve function and quality of life while being worn. Colour-coded Quick Fit buckles make donning and doffing simple, while flexible shells and breathable liners are designed to improve comfort and reduce migration, thus helping to ensure compliance.



## INDICATIONS FOR USE

- Moderate to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Other knee conditions that may benefit from unicompartmental unloading, such as Articular cartilage defect repair, Avascular necrosis, Tibial plateau fracture, Bone marrow lesions (bone bruises)



## FEATURES AND BENEFITS

- Dual Dynamic Force Straps® designed to reduce pressure on sensitive bony surfaces of the knee, resulting in reduction of pain
- Lightweight frame, perforated for flexibility and ventilation, with flexible, low profile upright that contours to patient anatomy and allows use underneath clothing
- Tool-free SmartDosing® dial enables fine-tuned pain management and can be adjusted even when wearing the brace under clothing
- Colour-coded quick fit buckles enable easy application and removal
- Proprietary Sensil® silicone liners designed to reduce migration
- Available in standard and short version, for a perfect fit on shorter legs, too

## REQUIRED MEASUREMENTS

- Circumference measurement: 15cm below mid-patella

Size	Measurements
S	31 - 35.5cm
M	35.5 - 42.5cm
L	42.5 - 51cm
XL	51 - 61.5cm

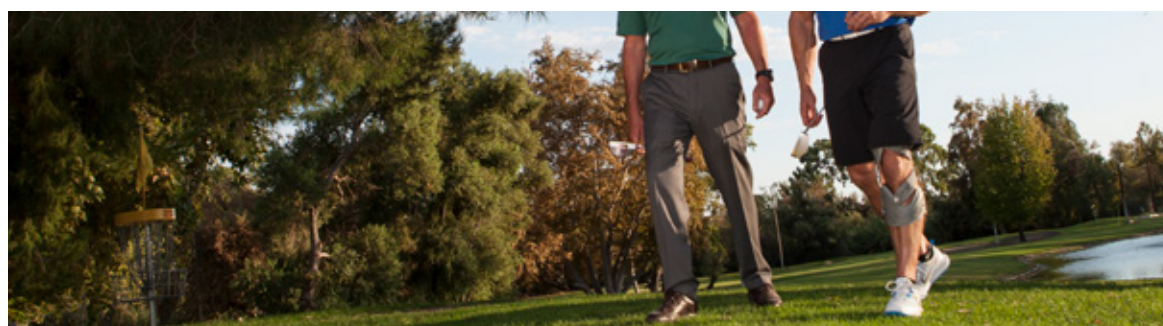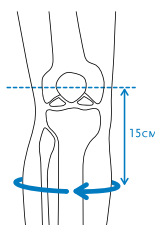

# UNLOADER ONE®

## ORDERING INFORMATION

### UNLOADER ONE WITH SMARTDOSING

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
S	31 - 35.5cm	B-240618712	B-240518712	B-240628712	B-240528712
M	35.5 - 42.5cm	B-240618713	B-240518713	B-240628713	B-240528713
L	42.5 - 51cm	B-240618714	B-240518714	B-240628714	B-240528714
XL	51 - 61.5cm	B-240618715	B-240518715	B-240628715	B-240528715

### UNLOADER ONE SHORT WITH SMARTDOSING

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
XS	25 - 31cm	B-241618711	B-241518711	B-241628711	B-241528711
S	31 - 35.5cm	B-241618712	B-241518712	B-241628712	B-241528712
M	35.5 - 42.5cm	B-241618713	B-241518713	B-241628713	B-241528713
L	42.5 - 51cm	B-241618714	B-241518714	B-241628714	B-241528714
XL	51 - 61.5cm	B-241618715	B-241518715	B-241628715	B-241528715

### UNLOADER ONE WITH RATCHET

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
S	31 - 35.5cm	B-240619712	B-240519712	B-240629712	B-240529712
M	35.5 - 42.5cm	B-240619713	B-240519713	B-240629713	B-240529713
L	42.5 - 51cm	B-240619714	B-240519714	B-240629714	B-240529714
XL	51 - 61.5cm	B-240619715	B-240519715	B-240629715	B-240529715

### UNLOADER ONE SHORT WITH RATCHET

Size	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
XS	25 - 31cm	B-241619711	B-241519711	B-241629711	B-241529711
S	31 - 35.5cm	B-241619712	B-241519712	B-241629712	B-241529712
M	35.5 - 42.5cm	B-241619713	B-241519713	B-241629713	B-241529713
L	42.5 - 51cm	B-241619714	B-241519714	B-241629714	B-241529714
XL	51 - 61.5cm	B-241619715	B-241519715	B-241629715	B-241529715

Standard version is recommended for patients above 1.65m  
Short version is recommended for patients under 1.65m tall.  
Hinge is placed on the side of the affected compartment

## ACCESSORIES

### ACCESSORY KITS

	SMARTDOSING		RATCHET	
Description	RM/LL	RL/LM	RM/LL	RL/LM
Strap & Strap Pad & Buckle Clasp Kit	B-705277003	B-705278003	B-701217008	B-701218008
Strap & Strap Pad Kit	B-705270003		B-701210006	
Strap Kit	B-701270013		B-701210004	
Strap Kit Short	B-701240012			
Pad Kit	B-702210003		B-702210003	

# UNLOADER ONE®

## ACCESSORIES

### ACCESSORY KITS

Description	Right medial/Left lateral	Right lateral/Left medial
Buckle Clasp Kit	B-708217003	B-708218003
Alligator Clip Kit (included in all strap kits)	B-704210003	
Extension Stop Kit	B-301040046	
Twist-Fit Hook & Loop Attachment Kit	B-721240003	

### UNDERSLEEVE

Size	Thigh Circumference	Part#
XS	25.4 - 30.9cm	B-250150201
S	30.9 - 35.6cm	B-250150202
M	35.6 - 42.5cm	B-250150203
L	42.5 - 50.8cm	B-250150204
XL	50.8 - 61cm	B-250150205
XXL	61 - 81.3cm	B-250150206

When using an undersleeve we advice to use the suspension strap kit to minimise migration



### SENSIL LINERS

	Tibia		Femur	
Size	RM/LL*	RL/LM**	RM/LL*	RL/LM**
XS	B-419700311	B-419800311	B-409700321	B-409800321
S	B-419700312	B-419800312	B-409700322	B-409800322
M	B-419700313	B-419800313	B-409700323	B-409800323
L	B-419700314	B-419800314	B-409700324	B-409800324
XL	B-419700315	B-419800315	B-409700325	B-409800325

\* Right medial/Left lateral / \*\* Right lateral/Left medial



### DOESKIN LINERS

	Tibia		Femur	
Size	RM/LL*	RL/LM**	RM/LL*	RL/LM**
XS	B-424703721	B-424803721	B-412701721	B-412801721
S	B-424703722	B-424803722	B-412701722	B-412801722
M	B-424703723	B-424803723	B-412701723	B-412801723
L	B-424703724	B-424803724	B-412701724	B-412801724
XL	B-424703725	B-424803725	B-412701725	B-412801725

When using a doeskin tibia liner we advice to use a suspension strap kit to minimise migration / \* Right medial/Left lateral / \*\* Right lateral/Left medial



### SENSIL AMS (ANTI-MIGRATION SYSTEM) WRAPS

Size	Right medial/Left lateral	Right lateral/Left medial
XS	B-715217031	B-715218031
S	B-715217032	B-715218032
M	B-715217033	B-715218033
L	B-715217034	B-715218034
XL	B-715217035	B-715218035



### SUSPENSION STRAP KIT

Description	Part#
Suspension Strap Kit	UNL00212



# UNLOADER ONE® CUSTOM

## ORDERING INFORMATION

### UNLOADER ONE CUSTOM WITH SMARTDOSING

Colour	Thigh circumference	Calf circumference	Left	Right
Black	34.3 - 73.5cm	25 - 61.5cm	B-115600112	B-115500112
Grey	34.3 - 73.5cm	25 - 61.5cm	B-115601712	B-115501712

### UNLOADER ONE CUSTOM

Colour	Thigh circumference	Calf circumference	Left	Right
Black	34.3 - 73.5cm	25 - 61.5cm	B-115600111	B-115500111
Grey	34.3 - 73.5cm	25 - 61.5cm	B-115601711	B-115501711

\*Please allow 10 working days for delivery



### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace, including the Unloader One. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



### DIGITAL MEASUREMENT SYSTEM (DMS)

This system requires minimal physical measurements and two digital photos, which can then be sent to customer service. This system is a quick, easy and reliable method of measuring for an Unloader One® Custom without the need for training.

\*For custom accessories, please contact customer care on 03450 065 065.

### CUSTOM STANDARD COLOURS

Red	Pink	Silver
Orange	Black	White
Yellow	Navy Blue	Champagne
Lime Green	Ocean Blue	Charcoal

## UNLOADER ONE® PLUS

Many sufferers of OA knee pain are overweight. For these patients, containing excess soft tissue is essential for comfortable and effective bracing. That's why our new Unloader One Plus model features a wider thigh strap than the standard version, plus a gastroc strap that helps secure the brace in place. And because bigger legs require more support, we included a sturdier upright with an Adjustable Dynamic Joint, enabling additional leverage for optimal pain management. Multiple, peer-reviewed clinical and biomechanical studies have proven the efficacy of this technology.



### INDICATIONS FOR USE

- Moderate to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Other knee conditions that may benefit from unicompartmental unloading such as: Articular cartilage defect repair, Avascular necrosis, Tibial plateau fracture, Bone marrow lesions (bone bruises)

### FEATURES AND BENEFITS

- 3-Point Leverage System provides clinically relevant pain relief of affected compartment
- Single SmartDosing dial allows fine-tuned pain management
- Helps keep patients mobile during daily activities
- Discreet hinge and sleeve design ensure a snug fit underneath clothing
- Lightweight and easy to use

### REQUIRED MEASUREMENTS

- Thigh circumferences of 53.3cm to 73.7cm and calf circumferences of 38.1cm to 50.8cm

# UNLOADER ONE® PLUS

## ORDERING INFORMATION

### UNLOADER ONE PLUS WITH RATCHET

Size	Thigh Circumference	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
L	53 - 64cm	36 - 43cm	B-245610004	B-245510004	B-245620004	B-245520004
XL	64 - 81cm	43 - 51cm	B-245610005	B-245510005	B-245620005	B-245520005

### UNLOADER ONE PLUS OTS, LONG VERSION

Size	Thigh Circumference	Calf circumference	Left medial	Right medial	Left lateral	Right lateral
L	53 - 64cm	36 - 43cm	B-244610004	B-244510004	B-244620004	B-244520004
XL	64 - 81cm	43 - 51cm	B-244610005	B-244510005	B-244620005	B-244520005

Standard version is recommended for patients between 1.65 and 1.92 metres tall.  
Long version is recommended for patients over 1.92 metres tall.

## ACCESSORIES

### SENSIL AMS (ANTI-MIGRATION SYSTEM) WRAPS

Size	Right medial/Left lateral	Right lateral/Left medial
M	B-715237033	B-715238033
L	B-715237034	B-715238034
XL	B-715237035	B-715238035

### LINERS

	Tibia, silicone		Femur, doeskin	
Size	Right medial/Left lateral	Right lateral/Left medial	Right medial/Left lateral	Right lateral/Left medial
M	B-413700313	B-413800313	B-414700323	B-414800323
L	B-413700314	B-413800314	B-414700324	B-414800324
XL	B-413700314	B-413800315	B-414700325	B-414800325
XXL			B-414700326	B-414800326

### ACCESSORY KITS

Description	Right medial/Left lateral	Right lateral/Left medial
Strap & Strap Pad & Buckle Clasp Kit	B-701237008	B-701238008
Strap & Strap Pad Kit	B-701230006	
Strap Kit	B-701230004	
Pad Kit	B-702230003	
Extra Pads	B-702230013	
Buckle Clasp Kit	B-708237003	B-708238003
Alligator Clip Kit (included in all strap kits)	B-704210003	
Gastroc Kit	B-705230001	
Elastic Thigh Strap 8"	B-701237083	B-701238083
Elastic Thigh Strap 10"	B-701237103	B-701238103

## REBOUND® CARTILAGE

Cartilage and meniscal injuries usually lead to significant limitations of knee function and knee pain. As surgery cannot be done on the day of diagnosis, prehabilitation is indicated to limit knee pain and keep the patient functional until surgery is scheduled. In addition, post-operatively, the success of meniscal tear and articular cartilage repair is dependent upon patient compliance during the rehabilitation process and the protocol is based on restricted weight bearing combined with gradual increase in activity. Many rehabilitation protocols suggest partial weight-bearing as the standard of care for successful outcomes. However, patient compliance to partial weight-bearing instructions with crutches has been shown to be inconsistent.

Rebound Cartilage is a protective functional solution designed for prehabilitation / pre-surgical management and rehabilitation of cartilage injuries and meniscal lesions. By protecting the healing tissue from overload, it fits into accelerated rehab protocols.

FUNCTIONAL HEALING - WHERE MOBILITY MEANS RECOVERY



### INDICATIONS FOR USE

- Articular cartilage defect repair needing unicompartmental load reduction or range of movement restriction
- Cartilage knee trauma benefiting from unloading or movement restriction
- Meniscal repair
- Avascular necrosis
- Condylar bone marrow lesions (bone bruises)
- Tibial plateau fractures
- May also be beneficial for other unicompartmental knee conditions that require unloading

### FEATURES AND BENEFITS

- Dual Cartilage Protection Straps™ (CPS) maintain joint unloading in flexion for protection of the cartilage during the healing process
- Breathable wrap around sleeve integrated with Sensil® silicone coated calf liners and doeskin thigh liners provide increased comfort while ensuring adequate suspension
- Colour coded Quick Fit buckles enable easy application and removal
- Optional modular Flexion Control kit when needed per prescribed rehab protocol for range of movement restriction.

### REQUIRED MEASUREMENTS

- Circumference measurement: 15cm below mid-patella





# REBOUND® CARTILAGE

## ORDERING INFORMATION

### REBOUND CARTILAGE

Size	Circumference	Left medial	Right medial	Left lateral	Right lateral
S	31 - 35.5cm	B-254610002	B-254510002	B-254620002	B-254520002
M	35.5 - 42.5cm	B-254610003	B-254510003	B-254620003	B-254520003
L	42.5 - 51cm	B-254610004	B-254510004	B-254620004	B-254520004
XL	51 - 61.5cm	B-254610005	B-254510005	B-254620005	B-254520005

### REBOUND CARTILAGE - WITH FLEXION CONTROL

Size	Circumference	Left medial	Right medial	Left lateral	Right lateral
S	31 - 35.5cm	B-254810002	B-254710002	B-254820002	B-254720002
M	35.5 - 42.5cm	B-254810003	B-254710003	B-254820003	B-254720003
L	42.5 - 51cm	B-254810004	B-254710004	B-254820004	B-254720004
XL	51 - 61.5cm	B-254810005	B-254710005	B-254820005	B-254720005

Note: Circumference limitation: Thigh circumferences smaller than 34.3cm or larger than 73.5cm. Calf circumferences smaller than 25cm or larger than 61.5cm.

## ACCESSORIES

### WRAPAROUND SLEEVE (WITH INTEGRATED SILICONE CALF LINER AND DOESKIN THIGH LINER)

Size	Right medial/Left lateral	Right lateral/Left medial
S	B-254900072	B-254900082
M	B-254900073	B-254900083
L	B-254900074	B-254900084
XL	B-254900075	B-254900085
Optional sleeveless liner kit	Right medial/Left lateral	Right lateral/Left medial
Silicone calf liner, doeskin thigh liner, strap pads	B-254900070	B-254900080
Doeskin calf and thigh liners, suspension strap kit, strap pads	B-427705000	B-427705001

### STRAP KITS

Size	Complete strap kit (incl. 4 straps)	CPS strap kit (incl. 2 straps)
S	B-701290002	B-700290002
M	B-701290003	B-700290003
L	B-701290004	B-700290004
XL	B-701290005	B-700290005

### UNIVERSAL FLEXION CONTROL KIT

Description	Part#
Left medial	B-705296100
Right medial	B-705295100
Left lateral	B-705296200
Right lateral	B-705295200

### OTHER ACCESSORIES

Description	Part#
Flexion Stop Kit	B-729000000
Twist-Fit Hook & Loop Attachment Kit	B-705290000

## CTi® OA

CTi braces provide the ultimate combination of stabilisation and protection of the knee joint. CTi uses Accutrak® hinges with extension stops to glide with the knee and breathable liners coated with bio-inert Sensil® silicone to ensure the CTi remains properly positioned on the leg, for optimal functional benefit. Allowing for varus or valgus correction, the custom made CTi OA is an excellent, dual hinged hybrid OA and ligament knee brace for your more active patients.



### INDICATIONS FOR USE

- For knee conditions that may benefit from increased A-P and M-L stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities
- Also treats mild to severe unicompartmental osteoarthritis

### FEATURES AND BENEFITS

- Reversible field-serviceable d-rings and trimmable straps facilitate fast adjustments and customised fit
- Corrosion resistant materials, ideal for water sports
- Anatomically-correct Accutrak® hinges with extension stops
- Proprietary Sensil® silicone liners designed to reduce migration

### REQUIRED MEASUREMENTS

- Custom-to-measurement
- 5 point knee measurement (Smartmeasure)



## CTi® OA

### ORDERING INFORMATION

#### CTi OA

Description	Left	Right
CTi OA	B-116630010	B-116530010

For accessories see CTi Custom.



#### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace, including the CTi OA. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



#### DIGITAL MEASUREMENT SYSTEM (DMS)

This system simply requires minimal physical measurements and two digital photos, which can then be sent to customer service. This system is a quick, easy and reliable method of measuring for a CTi OA without the need for training.

Please contact customer care for a free DMS Kit. Item number - 89100

## REBOUND® PCL

Unlike the anterior cruciate ligament (ACL), the PCL has the intrinsic ability to heal and regain functionality after injury. Physiologically correct positioning of the PCL during conservative or post-surgical rehabilitation is important, as the PCL experiences variable tensile forces during knee flexion, and healing of the PCL in an elongated position can lead to chronic instability and disability. Insufficient healing after a PCL rupture leads to significant pathologies, such as chronic anterior knee pain or knee osteoarthritis.

The Rebound PCL applies a physiologically correct dynamic force to the PCL during rehabilitation. The dynamic load on the tibia is generated by applying an anterior directed dynamic force on the calf area, and opposing counterforces on the anterior aspect of the leg.

**FUNCTIONAL HEALING - WHERE MOBILITY MEANS RECOVERY**




### INDICATIONS FOR USE

- Non-surgical treatment of PCL ruptures

Post-surgical rehabilitation for:

- Partial PCL tears
- PCL reconstruction/augmentation
- PCL revision

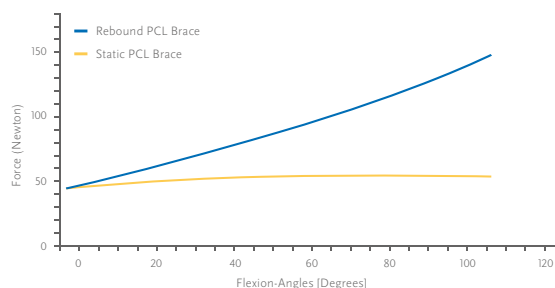

LaPrade RF, Smith SD, Wilson KJ, Wijdicks CA. Quantification of functional brace forces for posterior cruciate ligament injuries on the knee joint: an in vivo investigation. Knee Surg Sports Traumatol Arthrosc [Internet]. 2014 Aug 22 [cited 2014 Dec 11]; Available from: <http://www.ncbi.nlm.nih.gov/pubmed/25145947>

### FEATURES AND BENEFITS

- The Dynamic Tension System™ (DTS) is designed to facilitate healing by increasing the load on the tibia as the knee goes through flexion, and providing an anterior drawer force necessary to decrease load on the healing posterior cruciate ligament (PCL).
- Three shear knob options permit customisation of the biochemical correct posterior tibial load for each patient, and allow for adjustment, e.g. in case of swelling.
- A polycentric hinge with extension and flexion stops allows for range of motion adjustment. Lightweight brace with proprietary Sensil® silicone calf liner and doeskin thigh liner to reduce migration, while providing comfort to the user.
- Posterior frame for optimised support, with open anterior for ease of donning/doffing.

### REQUIRED MEASUREMENTS

- Caliper measurement: Medio-laterally at joint width in a standard, weight-bearing position
- Calf circumference 15 cm below mid patella
- Thigh circumference 15 cm above mid patella

# REBOUND® PCL

## ORDERING INFORMATION

### REBOUND PCL

Size	Caliper M/L*	Calf Circ.**	Thigh Circ.***	Left	Right
S	8.9 - 10.2cm	29 - 34.5cm	42 - 45cm	B-124501161	B-124501158
M	10.2 - 11.4cm	34.5 - 38cm	45 - 51cm	B-124501162	B-124501159
L	11.4 - 12.7cm	38 - 42cm	51 - 58.5cm	B-124501163	B-124501160
XL	12.7 - 14.0 cm	42 - 45.7 cm	58.5 - 66.7cm	B-124501155XL	B-124501154XL

\* M/L measurement: In a standard, weight bearing position - Using a Caliper: M/L at joint width, Using a Constant Force Caliper: M/L at knee axis line

\*\* Calf circumference measurement: 15cm below knee centre

\*\*\* Thigh circumference measurement: 15cm above knee centre

### DOESKIN THIGH LINER

Size	Left	Right
XS	PCL60031	PCL60011
S	PCL60032	PCL60012
M	PCL60033	PCL60013
L	PCL60034	PCL60014
XL	PCL60035	PCL60015
XXL	PCL60036	PCL60016

### DOESKIN CALF LINER

Size	Left	Right
XS	PCL60171	PCL60071
S	PCL60172	PCL60072
M	PCL60173	PCL60073
L	PCL60174	PCL60074
XL	PCL60175	PCL60075
XXL	PCL60176	PCL60077

### SILICONE CALF LINER

Size	Left	Right
XS	PCL60121	PCL60021
S	PCL60122	PCL60022
M	PCL60123	PCL60023
L	PCL60124	PCL60024
XL	PCL60125	PCL60025
XXL	PCL60126	PCL60026

### STANDARD THIGH LINER KIT

Size	Left	Right
XS	B-124501170	B-124501164
S	B-124501171	B-124501165
M	B-124501172	B-124501166
L	B-124501173	B-124501167
XL	B-124501174	B-124501168
XXL	B-124501175	B-124501169

Includes: Doeskin thigh liner, doeskin femoral cuff liner (softgood on lower thigh strap), doeskin pre-femoral liner (softgood on upper thigh strap), hook and loop (4x round coins, 8x rec. tabs)

## REBOUND® PCL

### ORDERING INFORMATION

#### STANDARD CALF LINER KIT

Size	Left	Right
XS	B-124601176	B-124501176
S	B-124601177	B-124501177
M	B-124601178	B-124501178
L	B-124601179	B-124501179
XL	B-124601180	B-124501180
XXL	B-124601181	B-124501181

Includes: Silicone calf liner/AMS, doeskin tibia cuff liner (softgood on lower calf strap), hook and loop (2x round coins, 4x rec. tabs)

#### DOESKIN CALF LINER KIT

Size	Left	Right
XS	B-124601182	B-124501182
S	B-124601183	B-124501183
M	B-124601184	B-124501184
L	B-124601185	B-124501185
XL	B-124601186	B-124501186
XXL	B-124601187	B-124501187

Includes: Doeskin calf liner/AMS, doeskin tibia cuff liner (softgood on lower calf strap), tibia strap pad, hook and loop (2x round coins, 4x rec. tabs)



#### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



#### DIGITAL MEASUREMENT SYSTEM (DMS)

This system simply requires minimal physical measurements and two digital photos, which can then be sent to customer service. This system is a quick, easy and reliable method of measuring for a custom knee brace without the need for training.

Please contact customer care for a free DMS Kit. Item number - 89100

#### REBOUND PCL CUSTOM

Description	Part#
Rebound PCL Custom Left	B-124501155
Rebound PCL Custom Right	B-124501154

\*Please allow 10 working days for delivery

#### CUSTOM STANDARD COLOURS

Red	Pink	Silver
Orange	Black	White
Yellow	Navy Blue	Champagne
Lime Green	Ocean Blue	Charcoal

# REBOUND® PCL

## ACCESSORIES

### STRAPS

Size	Part#
S - L	PCL00061
XL - XXL	PCL00062

### OTHER ACCESSORIES

Description	Part#
Extension Stop Kit	B-705130042
Flexion Stop Kit	B-705130043
Shear Knob Kit	B-465350087
Condyle Pad Kit	B-705130046



Shear Knob Kit



## REBOUND® ACL

The Rebound ACL brace is an evolution in bracing, designed to comfortably apply an anatomically correct, dynamic load to the anterior cruciate ligament (ACL). As an element in either conservative or surgical management, the Rebound ACL brace complements clinical protocols and is conducive to an ideal biomechanical joint environment as a foundation for long-term successful patient outcomes. The innovative technology in the Rebound® Dynamic Tension System (DTS) offers individualised adjustment options to reduce strain on the healing ACL.




FUNCTIONAL HEALING – WHERE MOBILITY MEANS RECOVERY

### INDICATIONS FOR USE

Conditions benefiting from an agonistic force that varies throughout flexion in proportion to ACL tension applied to the thigh and calf in the sagittal plane. These may include:

- Anterior cruciate ligament (ACL) injury
- Reconstruction
- Non-surgical treatment

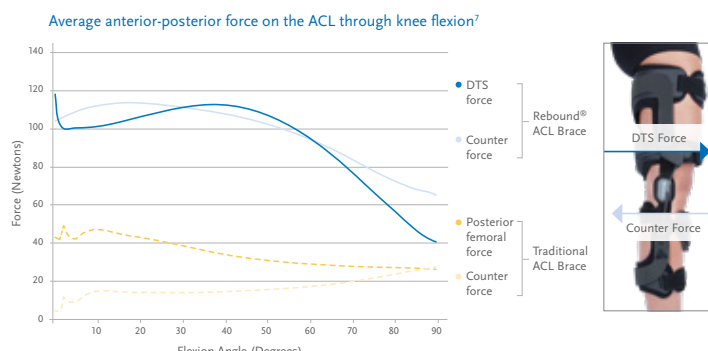

### FEATURES AND BENEFITS

- Dynamic technology for knee stability, with a Dynamic Tension System™ that applies an agonistic, dynamic force to the ACL that is flexion-angle dependent (LaPrade et al., 2017) and designed to reduce ACL deficiency (positive anterior-drawer test), promoting knee stability.
- Specific load adaptation according to the patient's individual anatomy and rehabilitation requirements. Clinical and biomechanical studies indicate that the Rebound ACL brace reduces strain on the ACL during activity and significantly reduces strain on the meniscus in ACL-deficient knees (Tomescu et al., 2017). Biocompatible, hi-tech materials, including Blue is You™ patient touchpoints and ActiveGrip™ liners, promote suspension and comfort.
- Lightweight, aircraft-grade aluminium frame
- Available in sizes S-XL
- Flexion & extension range-of-motion lock-outs.
- Posterior frame optimised for fit and support, with open anterior for ease of donning/doffing Clinically-tested comfort, validated by wearers and with load mapping data collected during rehabilitation activities, the Rebound® Dynamic Tension System (DTS) provides a comfortable load to the posterior thigh and anterior tibia.

# REBOUND® ACL

## REQUIRED MEASUREMENTS



Measure knee caliper medio-laterally at joint width in standing, weight-bearing position.



Measure calf circumference 15 cm below mid-patella. Leg in extension.



Measure thigh circumference 15 cm above mid-patella. Leg in extension.

## ORDERING INFORMATION

### STANDARD

Size	Left	Right	Caliper	Calf circumference 15 cm below mid patella	Thigh circumference 15 cm above mid patella
SM	B-124601191	B-124501191	8.9 - 10.2cm	29.2 - 34.3cm	41.9 - 47.0cm
MD	B-124601192	B-124501192	10.2 - 11.4cm	34.3 - 39.4cm	47.0 - 52.1cm
LG	B-124601193	B-124501193	11.4 - 12.7cm	39.4 - 44.5cm	52.1 - 57.2cm
XL	B-124601194	B-124501194	12.7 - 14.0cm	44.5 - 49.5cm	57.2 - 62.2cm

Recommended for patients 1.56m and taller

### REPLACEMENT PARTS

Description	Part#
Extension stop kit (0°, 5°, 10°, 20°, 30°, 40°)	B-124702035
Flexion stop kit (0°, 15°, 25°, 35°, 45°, 55°, 65°, 75°, 90°, 105°)	B-124702034
Shear Knob Kit	B-705130051
SM/MD Strap Kit	B-124702030
LG/XL Strap Kit	B-124702031
Condyle Pad Kit	B-701090009
Tibial Pad	B-124702032
Tibial Hex Pad	B-124702033

# REBOUND® ACL

## ACCESSORIES

### ANTI-MIGRATION WRAP

SIZE	RIGHT	LEFT
S/M	B-124501195	B-124601195
S/M HEX	B-124501196	B-124601196
L/XL	B-124501197	B-124601197
L/XL HEX	B-124501198	B-124601198

### UPPER DOESKIN LINER KIT

SIZE	RIGHT	LEFT
S	B-124702004	B-124702000
M	B-124702005	B-124702001
L	B-124702006	B-124702002
XL	B-124702007	B-124702003

### LOWER ACTIVEGRIP LINER KIT

SIZE	RIGHT	LEFT
S	B-124702014	B-124702010
M	B-124702015	B-124702011
L	B-124702016	B-124702012
XL	B-124702017	B-124702013

### LOWER DOESKIN LINER KIT

SIZE	RIGHT	LEFT
S	B-124702024	B-124702020
M	B-124702025	B-124702021
L	B-124702026	B-124702022
XL	B-124702027	B-124702023

# REBOUND® ACL

## ORDERING INFORMATION FOR REBOUND ACL CUSTOM

### REBOUND ACL CUSTOM

Part#	Type	Min / Max Caliper	Min / Max Circ. above	Min / Max Circ. below	Measuring
B-124701000	Right	8.9 - 14cm	41.9 - 62.2cm	29.2 - 49.5cm	SmartMeasure, Cast
B-124701001	Left				

Caliper = M/L width at knee center  
Circumference = 15cm above / below mid-patella  
Ideal for patients 1.56m and taller

### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



Description	Part#
SmartMeasure Measuring Kit	CMS0000
Constant Force Caliper	B-312000000

Scans, CMS (CTI only), CCS and casting also accepted. Knee M/L required.

## REQUIRED MEASUREMENTS

- Custom made to measure ( Smartmeasure, 3D-Scan or patient cast)

## REBOUND® DUAL BASIC AND REBOUND® DUAL BASIC ST

The Rebound DUAL Basic and Rebound DUAL Basic ST offer the value and simplicity you need to navigate the complex coverage landscape. Within the versatility of the Rebound DUAL collection, Össur offers high quality solutions from a single source, to help users transition to a successful recovery without compromise.



Rebound DUAL Basic

Rebound DUAL Basic ST  
(Step-Through)

### INDICATIONS FOR USE

The Rebound DUAL Basic is indicated for combined and non-combined instabilities of the:

- Anterior cruciate ligament (ACL)
- Medial collateral ligament (MCL)
- Lateral collateral ligament (LCL)
- Posterior cruciate ligament (PCL)
- (Only Rebound DUAL Basic) Acculign version only: Conditions requiring unicompartmental load control

### FEATURES AND BENEFITS

- (Only Rebound DUAL Basic ST) Step-Through Design: advantage in comfort & fitting with ease
- (Only Rebound DUAL Basic) Multi-purpose length-adjustable upper frame accommodates the user's height, helping to ensure the perfect fit and enabling adjustment for varus/valgus alignment
- Össur Hinge alignment guide provides easy & accurate re-fitting for users, ensuring correct placement of the brace relative to the knee
- Lightweight aircraft aluminium frame keeps a low profile
- Doeskin liners for comfort & no-slip Activegrip™ strap pad
- Trimmable straps with tactile strap tabs
- Low-profile polycentric hinge with stop options for stability & range-of-motion restriction
- Step-through or Anterior frame design
- Suitable for low-high impact and sliding sports
- Optional PCL kit accessory for multi-ligament support
- Available in pre-fabricated sizes XS-3XL

### REQUIRED MEASUREMENTS

- Knee caliper
- Thigh circumference 15 cm above mid patella

# REBOUND® DUAL BASIC AND REBOUND® DUAL BASIC ST

## ORDERING INFORMATION

**X** = 5 (right)  
 6 (left)  
**Y** = 1 (Rebound DUAL Basic)  
 2 (Rebound DUAL Basic ST)

Description	Size	Caliper	Thigh Circumference	Part #
Rebound DUAL Basic or Rebound DUAL Basic ST*	XS	7.6 - 8.9cm	38.1 - 41.9cm	B-242 <b>XY</b> 0001B
	S	8.9 - 10.2cm	41.9 - 47cm	B-242 <b>XY</b> 0002B
	M	10.2 - 11.4cm	47 - 52.1cm	B-242 <b>XY</b> 0003B
	L	11.4 - 12.7cm	52.1 - 57.2cm	B-242 <b>XY</b> 0004B
	XL	12.7 - 14cm	57.2 - 62.2cm	B-242 <b>XY</b> 0005B
	XXL	14 - 15.2cm	62.2 - 69.9cm	B-242 <b>XY</b> 0006B
	3XL	15.2 - 16.5cm	69.9 - 76cm	B-242 <b>XY</b> 0007B

## REPLACEMENT LINER KITS

Measuring	Basic Liner Part #	Basic ST Liner Part #
XS	B-701090130	B-701090140
S	B-701090131	B-701090141
M	B-701090132	B-701090142
L	B-701090133	B-701090143
XL	B-701090134	B-701090144
XXL	B-701090135	B-701090145
3XL	B-701090136	B-701090146

## REPLACEMENT PARTS

Size	Part #
RDUAL Basic Strap Kit XS-L	B-701090137
RDUAL Basic Strap Kit XL-3XL	B-701090138
RDUAL Basic ST Strap Kit XS-L	B-701090147
RDUAL Basic ST Strap Kit XL-3XL	B-701090148
Condyle Pad Kit	B-701090009

## ACCESSORIES

Size	Part #
RDUAL Basic ROM Stop Kit	B-701090109
RDUAL Basic PCL Strap Kit XS-L	B-701090057
RDUAL Basic PCL Strap Kit XL-3XL	B-701090058
RDUAL Basic ST PCL Strap Kit XS-L	B-701090065
RDUAL Basic ST PCL Strap Kit XL-3XL	B-701090066

\* Rebound DUAL Basic ST is 35.5cm

## FORMFIT® OA EASE

The Formfit OA Ease is a single upright brace with a Dynamic Force System™ strap designed to relieve pain from mild-to-moderate unicompartmental knee osteoarthritis (OA). Its wraparound design is easy to apply and remove using simple rubber pull-tabs, colour-coded guide dots, and strap tension indicator lines. This low-profile knee brace is comfortable, easy to use, and adapts to a variety of leg anatomies. The Formfit OA Ease can also be machine washed for effortless care.



### INDICATIONS FOR USE

Take control of mild-to-moderate OA pain with the Formfit OA Ease, an easy-to-wear knee brace with a simple wraparound design that can be applied and removed with ease.

- Mild-to-moderate unicompartmental knee osteoarthritis and/or degenerative meniscal tears
- Other conditions that may benefit from unicompartmental unloading

### FEATURES AND BENEFITS

- Easy Wraparound Design - Simple rubber pull-tabs, colour-coded guide dots and a convenient hinge alignment guide make application and removal simple
- Relieves OA Pain - 3-Point Leverage System with Dynamic Force System™ strap and shells unload the affected knee compartment
- Comfortable - Lightweight and breathable fabric feels great against the skin and can be machine washed for effortless care
- Discreet – Low-profile brace with a slim hinge can be worn under clothes
- Control – Fine-tune strap tension for individual pain relief, with marked dosing indicator lines
- Extension kits and liners are available to further enhance fit and comfort

### REQUIRED MEASUREMENTS



Measure calf circumference 15cm below mid-patella. Leg in extension.



Measure thigh circumference 15cm above mid-patella. Leg in extension.



# FORMFIT® OA EASE

## ORDERING INFORMATION

Size	Calf circ. 15cm below mid patella	Thigh circ. 15cm above mid patella	Right Medial	Left Medial	Right Lateral	Left Lateral
XS	28 - 32cm	36 - 41cm	FOA0518711	FOA0618711	FOA0528711	FOA0628711
S*	32 - 36cm	41 - 46cm	FOA0518712	FOA0618712	FOA0528712	FOA0628712
M	36 - 41cm	46 - 53cm	FOA0518713	FOA0618713	FOA0528713	FOA0628713
L*	41 - 46cm	53 - 60cm	FOA0518714	FOA0618714	FOA0528714	FOA0628714
XL	46 - 51cm	60 - 67cm	FOA0518715	FOA0618715	FOA0528715	FOA0628715
XXL*	51 - 56cm	67 - 75cm	FOA0518716	FOA0618716	FOA0528716	FOA0628716

\*All sizes are available but for consolidated stock option this size can be achieved by combining the smaller size with accessories below.

## ACCESSORIES

Description	Part#
ActiveGrip Calf Liner Kit 2MM	FOA000101
ActiveGrip Calf Liner Kit 6MM	FOA000102
Doeskin Thigh Liner Kit 2MM	FOA000201
Doeskin Thigh Liner Kit 6MM	FOA000202
Small Extension Kit 2 Pieces	FOA000301
Small Extension Kit 10 Pieces	FOA000302
Large Extension Kit XS-SM (Right Medial/Left Lateral)	FOA000401
Large Extension Kit MD-LG (Right Medial/Left Lateral)	FOA000402
Large Extension Kit XL-XXXL (Right Medial/Left Lateral)	FOA000403
Large Extension Kit XS-SM (Right Lateral/Left Medial)	FOA000404
Large Extension Kit MD-LG (Right Lateral/Left Medial)	FOA000405
Large Extension Kit XL-XXXL (Right Lateral/Left Medial)	FOA000406
Wash Bag	WB000001
Suspension Strap	OAW00007
Flexion/Extension Stop Kit	B-301040047

## REPLACEMENT PARTS

Description	Part#
Alligator Clip (Right Medial/Left Lateral)	FOW00501
Alligator Clip (Right Lateral/Left Medial)	FOW00502
Hinge Pad	B-304120101

## CTi® OTS

The proven technology of the CTi knee brace is available with off-the-shelf convenience, providing adjustability and an excellent out-of-the-box fit while maintaining the level of support and protection that you expect from a CTi knee brace.

Outfitted with anatomically-correct Accutrak® hinges and flexible subshells and d-rings, the rock-solid CTi carbon frame provides a rigid exoskeleton that stabilises the knee joint. Breathable liners coated with bio-inert Össur Sensil® Silicone ensure CTi OTS remains properly positioned on the leg, for optimal functional benefit.



### INDICATIONS FOR USE


- ACL, MCL, LCL, PCL, rotary and combined instabilities

### FEATURES AND BENEFITS

- Anatomically-correct Accutrak hinges with extension stops to glide with the knee
- Flexible subshell and Adjustable Tibial Strap capture the tibia and allow adjustments in hinge depth for a more customised fit
- Breathable liners coated with bio-inert Össur Sensil Silicone to reduce migration and skin irritation
- Total Support System™ for ACL, MCL, LCL, PCL, rotary and combined instabilities
- Corrosion resistant materials, ideal for water sports
- Available in 2 models: Standard model for moderate to high impact levels, Pro Sport model for the impact levels

### REQUIRED MEASUREMENTS

- Caliper measurement: Medio-laterally at joint width in a standard, weight-bearing position
- Circumference: Just below the distal border of the patella

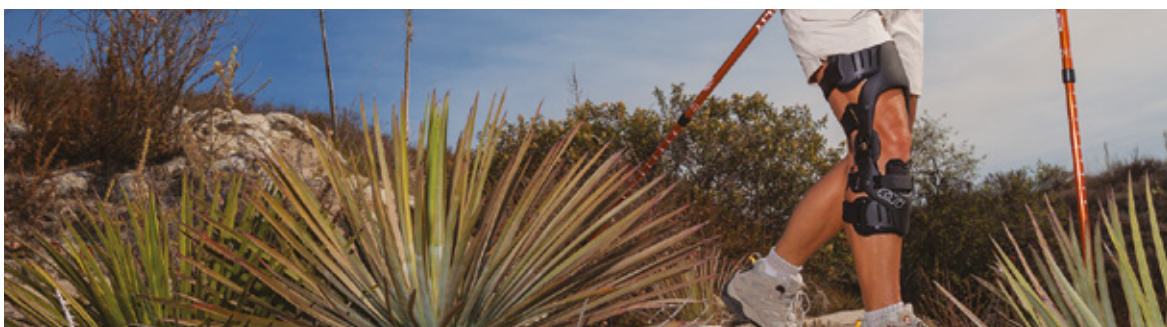

# CTI® OTS

## ORDERING INFORMATION

### CTI

Size	Caliper	Circumference	Standard Left	Standard Right
S	8.9 - 10.2cm	28 - 34cm	B-238600112	B-238500112
M	10.2 - 11.4cm	34 - 38cm	B-238600113	B-238500113
L	11.4 - 12.7cm	38 - 41cm	B-238600114	B-238500114
XL	12.7 - 14.0cm	41 - 44cm	B-238600115	B-238500115
XXL	14.0 - 15.2cm	44 - 48cm	B-238600116	B-238500116

Size	Pro Sport Left	Pro Sport Right	CTi PCL left	CTi PCL right
S	B-239600112	B-239500112	S, B-238640112	S, B-238540112
M	B-239600113	B-239500113	M, B-238640113	M, B-238540113
L	B-239600114	B-239500114	L, B-238640114	L, B-238540114
XL	B-239600115	B-239500115	XL, B-238640115	XL, B-238540115
XXL	B-239600116	B-239500116	XXL, B-238640116	XXL, B-238540116

Note: M/L measurements are the preferred and most accurate form of measurement for the CTi. Circumferential measurements are approximate.

## ACCESSORIES

### LINERS

Size	Silicone tibia cuff	Silicone calf	Silicone thigh	Lycra thigh
M	B-304030030	B-425580002	B-425571002	B-425570002
L		B-425580004	B-425571004	B-425570004
XL, XXL		B-425580005	B-425571004	B-425570004

### STRAPS KITS

Size	Part#
S, M	B-701140103
L, XL, XXL	B-701140104

### PCL KITS

Size	Part#
S, M	B-705000132
L, XL, XXL	B-705000235

### STRAP PADS

Size	Part#
S	B-304010057
M, L	B-304010058
XL, XXL	B-304010059

### AMS (ANTI-MIGRATION SYSTEM) WRAPS

Description	Part#
Silicone AMS Wrap S, M	B-715260002
Silicone AMS Wrap L, XL, XXL	B-715260004
Neoprene AMS Wrap S, M	B-410000001
Neoprene AMS Wrap L, XL	B-410000002

# CTI® OTS

## ACCESSORIES

### MX GEAR



Description	Part#
Patella Cup	B-711130011
Gear Guards	B-705000003
MX Kit S (patella cup, gear guards, undersleeve, stickers)	B-711140002
MX Kit M (patella cup, gear guards, undersleeve, stickers)	B-711140003
MX Kit L (patella cup, gear guards, undersleeve, stickers)	B-711140004
MX Kit XL (patella cup, gear guards, undersleeve, stickers)	B-711140005
MX Kit XXL (patella cup, gear guards, Sleeve with Mesh, stickers)	B-711140006

### SLEEVES

Circumference 15cm above mid patella	Softsleeve	Undersleeve	Oversleeve	Size	Circumference 15cm above mid patella	Sleeve with Mesh
< 38cm	20643	20251	20051	XS	< 43 cm	SLV60011
38 - 45cm	20644	20252	20052	S	43 - 46 cm	SLV60012
45 - 49.5cm	20644	20253	20053	M	46 - 51 cm	SLV60013
49.5 - 54.5cm	20645	20254	20054	L	51 - 54 cm	SLV60014
54.5 - 62cm	20645	20255	20055	XL	54 - 56 cm	SLV60015
62 - 71cm	20646	20256	20056	XXL	56 - 61 cm	SLV60016
71 - 86cm	20647	20257	20057			

### OTHER ACCESSORIES

No.	Description	Part#
1	CTi D-Ring Kit Standard 2x 1" and 2x 1.5" d-rings	B-704132600
2	CTi D-Ring Kit 2" 2x 2" d-rings	B-704132602
3	CTi Condyle Pad Kit	B-704130150
4	Extension Stop Kit	22130
5	Flexion Stop Kit	19254

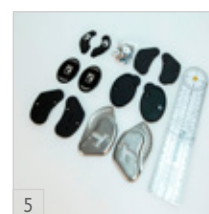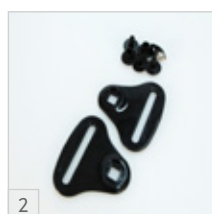

## CTi® CUSTOM

CTi braces provide the ultimate combination of stabilisation and protection of the knee joint. This truly custom made brace, with rock-solid CTi carbon frame, uses Accutrac® hinges with extension stops to glide with the knee and breathable liners coated with bio-inert Sensil® silicone to ensure the CTi remains properly positioned on the leg, for optimal functional benefit. CTi custom braces are available in different models and can be further customised with a variety of options, colours, designs and accessories.



### INDICATIONS FOR USE

For knee conditions that may benefit from increased AP and ML stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities

### FEATURES AND BENEFITS

- Reversible field-serviceable d-rings and trimmable straps facilitate fast adjustments and customised fit (optional)
- Corrosion resistant materials, ideal for water sports
- Anatomically-correct Accutrac® hinges with extension stops
- Proprietary Sensil® silicone liners designed to reduce migration
- Available in 3 different models: Standard model for moderate to high impact levels, Pro Sport model for the impact levels and Vapor model for low to moderate impact levels

### REQUIRED MEASUREMENTS

- Custom made to measure (CMS, Smartmeasure, 3D-Scan or patient cast)


#### CUSTOM STANDARD COLOURS



#### CUSTOM PAINTED DESIGNS



No charge for custom standard colours. Additional fees required for other custom colours, custom painted designs and custom printed graphics.

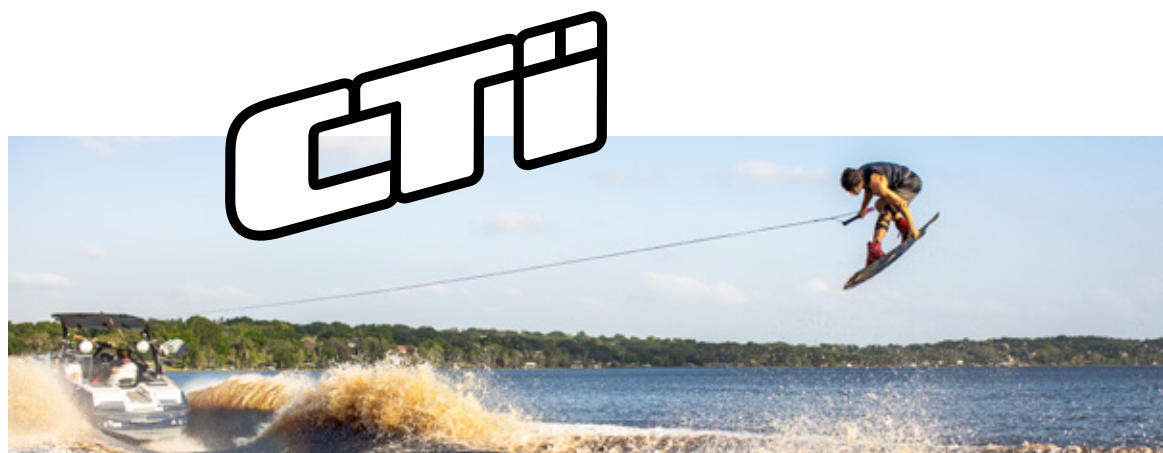

# CTI® CUSTOM

## ORDERING INFORMATION

### CTI CUSTOM

Description	Left	Right
Standard	B-116600010	B-116500010
Pro Sport	B-117600010	B-117500010
Vapor	B-118600010	B-118500010

\*Please allow 10 working days for delivery



### SMARTMEASURE®

A digital measuring system for custom knee braces on the iPhone or iPad. The most intuitive, efficient and accurate way to measure for any Össur custom knee brace, including the CTi. Utilises a section-based, step-by-step approach to guide you through the entire brace selection, measuring and order process.



### DIGITAL MEASUREMENT SYSTEM (DMS)

This system simply requires minimal physical measurements and two digital photos, which can then be sent to customer service. This system is a quick, easy and reliable method of measuring for a CTi without the need for training.

Please contact customer care for a free DMS Kit. Item number - 89100

## ACCESSORIES

### CALF LINERS

Description	Part#
CTi Custom liner	B-411500115

### STRAPS KITS

Size	Part#
S, M	B-701130104
L, XL, XXL	B-701130108

### PCL KITS

Size	Part#
S, M	B-705000132
L, XL, XXL	B-705000235



## CTI® CUSTOM

### ACCESSORIES

#### STRAP PADS

Size	Part#
S	B-304010057
M, L	B-304010058
XL, XXL	B-304010059



Strap pads

#### AMS (ANTI-MIGRATION SYSTEM) WRAPS

Description	Part#
AMS Wrap EVA	22095
AMS Wrap Neo	22094
AMS Wrap Neo XL	22152



AMS Wrap

#### MX GEAR

Description	Part#
Patella Cup	B-711130011
Gear Guards	B-705000003
MX Kit S (patella cup, gear guards, undersleeve, stickers)	B-711140002
MX Kit M (patella cup, gear guards, undersleeve, stickers)	B-711140003
MX Kit L (patella cup, gear guards, undersleeve, stickers)	B-711140004
MX Kit XL (patella cup, gear guards, undersleeve, stickers)	B-711140005
MX Kit XXL (patella cup, gear guards, Sleeve with Mesh, stickers)	B-711140006



#### SLEEVES

Size	Circumference 15cm above mid patella	Softsleeve	Undersleeve	Oversleeve
S	< 38cm	20643	20251	20051
M	38 - 45cm	20644	20252	20052
L	45 - 49.5cm	20644	20253	20053
XL	49.5 - 54.5cm	20645	20254	20054
XXL	54.5 - 62cm	20645	20255	20055
XXXL	62 - 71cm	20646	20256	20056
XXXXL	71 - 86cm	20647	20257	20057

Size	Circumference 15cm above mid patella	Sleeve with Mesh
XS	< 43 cm	SLV60011
S	43 - 46 cm	SLV60012
M	46 - 51 cm	SLV60013
L	51 - 54 cm	SLV60014
XL	54 - 56 cm	SLV60015
XXL	56 - 61 cm	SLV60016

#### OTHER ACCESSORIES

Description	Part#
CTi D-Ring Kit Standard 2x 1" and 2x 1.5" d-rings	B-704132600
CTi D-Ring Kit 2" 2x 2" d-rings	B-704132602
CTi Condyle Pad Kit	B-704130150
Extension Stop Kit	22130
Flexion Stop Kit	19254



## PARADIGM®

- ACL, MCL, LCL, PCL, rotational and combined instabilities
- Protection and stabilisation of surgical grafts
- Recommended for low to moderate activity

### FEATURES AND BENEFITS

- Carbon fibre TriaX® shells with flexible subshell provides support for low to moderate activity
- Hard-anodised, polycentric hinges
- Adjustable tibial strap captures the tibia and allows adjustments in hinge depth for a more customised fit
- ROM kit included, enabling post surgical use
- Large, 'open window' frame design for comfort and compliance
- Lay on design for easy fitting



### REQUIRED MEASUREMENTS

- Circumference measurement: 15 cm above mid-patella

## ORDERING INFORMATION

Size	Circumference	Left	Right
XS	34.3 - 39.4cm	B-210600001	B-210500001
S	39.4 - 44.5 cm	B-210600002	B-210500002
M	44.5 - 52.1 cm	B-210600003	B-210500003
L	52.1 - 59.7 cm	B-210600004	B-210500004
XL	59.7 - 67.3 cm	B-210600005	B-210500005
XXL	67.3 - 74.9 cm	B-210600006	B-210500006

## ACCESSORIES

### PARADIGM

Size	Strap and Pad	Liner Set Right	Liner Set Left
XS	B-701060001	B-703065001	B-703066001
S	B-701060002	B-703065002	B-703066002
M	B-701060003	B-703065003	B-703066003
L	B-701060004	B-703065004	B-703066004
XL	B-701060005	B-703065005	B-703066005
XXL	B-701060006	B-703065006	B-703066006

### FLEXION / EXTENSION STOP KIT

Description	Item No.
Flexion / Extension Stop Kit	B-705000504

## FLEX™

Flex is a post-op/functional rehab solution offering more versatility in both shape and function than most other options. It's heat-moldable, carbon composite frame accommodates oedema and/or changes in leg shape during recovery from surgery, while its Accutrac® range-of-motion hinges provide flexion and extension control. Gel Fit™ condylar pads improve suspension and are comfortable for tender knees recovering from trauma.

### INDICATIONS FOR USE

- ACL, MCL, LCL, PCL, rotary and functional support
- Long-term use for post-op, rehab and functional support
- Recommended for low to moderate activity

### FEATURES AND BENEFITS

- Accutrac® hinges with easy to use flexion/extension stops
- Heat-mouldable, carbon composite frame accommodates oedema and/or changes in leg shape during rehabilitation
- Gel Fit™ Condylar pad for improved suspension and comfort
- Total Support System™ for ACL, MCL, LCL, PCL, rotary and combined instabilities
- Adjustable Tibial Strap captured the tibia and allows adjustments in hinge depth for a customised fit
- Large, "open window" frame design accommodates post-op dressing and oedema
- Corrosion resistant materials, ideal for use in water



### REQUIRED MEASUREMENTS

- Caliper measurement: Medio-laterally at joint width in a standard, weight-bearing position
- Circumference: Just below the distal border of the patella

## ORDERING INFORMATION

			Standard		PCL	
Size	Caliper	Circumference	Left	Right	Left	Right
S	8.9 - 10.2cm	27.9 - 32.1cm	FLEX-SL	FLEX-SR	FLEXPC-L-SL	FLEXPC-L-SR
M	10.2 - 11.4cm	32.1 - 36.2cm	FLEX-ML	FLEX-MR	FLEXPC-L-ML	FLEXPC-L-MR
L	11.4 - 12.4cm	36.2 - 39.1cm	FLEX-LL	FLEX-LR	FLEXPC-L-LL	FLEXPC-L-LR
XL	12.4 - 13.3cm	39.1 - 42.2cm	FLEX-XLL	FLEX-XLR	FLEXPC-L-XLL	FLEXPC-L-XLR

## ACCESSORIES

### REFURB KIT\*

Size	Gel Left	Gel Right	Neoprene Left	Neoprene Right
S	F/O-SETLS	F/O-SETRS	F/O-SETLSNEO	F/O-SETRSNEO
M	F/O-SETLM	F/O-SETRM	F/O-SETLMNEO	F/O-SETRMNEO
L	F/O-SETLL	F/O-SETRL	F/O-SETLLNEO	F/O-SETRLNEO
XL	F/O-SETLXL	F/O-SETRXL	F/O-SETLXLNEO	F/O-SETRXLNEO

\*incl. liners, straps, strap pads, condyle pads

## FLEX™

### LINERS

Size	Tibia Left	Tibia Right	Femur Left	Femur Right
S	44048	44049	44046	44047
M	44040	44041	44042	44043
L	44044	44045	44038	44039
XL	44052	44053	44050	44051

### ACCESSORIES

#### STRAPS AND STRAP PAD KITS

Size	Part#
S	44071
M	44069
L	44070
XL	44068

#### OTHER ACCESSORIES

Description	Part#
Flexion / Extension Stop Kit	44035
Flexion Stop Kit (10,20,30°)	B-305070010
Hinge Cover Kit	44083
Condyle Pad Gel (1/4in)	22183
Condyle Pad Kit Neoprene (1/8in + 1/16in + cover)	44066
Rigid Condyle Pad Left	44027
Rigid Condyle Pad Right	44028

## C180™ ROCKET

With a condensed size and Accutrac™ hinges that are modified to more accurately track young knees, the lightweight C180 Rocket provides support for children with ligament instabilities.

### INDICATIONS FOR USE

- For knee conditions that may benefit from increased AP and ML stability around the knee, such as: ACL, MCL, LCL, PCL, rotary and combined instabilities

### FEATURES AND BENEFITS

- Anatomically-correct Accutrac® hinges with easy-to-use extension stops
- Semi-rigid frame consisting of woven carbon fibre infused with acrylic resin, is heat-mouldable for customisation

### REQUIRED MEASUREMENTS

- Caliper measurement: Medio-laterally at joint width in a standard, weight-bearing position
- Circumference: Just below the distal border of the patella



## ORDERING INFORMATION

Size	Caliper	Circumference	Left	Right
S	7.6 - 8.9cm	24.1 - 27.9cm	C180KSL	C180KSR
L	8.3 - 9.5cm	26 - 29.8cm	C180KLL	C180KLR
XL	8.9 - 10.2cm	27.9 - 32.1cm	C1802XSL	C1802XSR

## ACCESSORIES

Description	Part#
Motocross kit S / L	10833
Motocross kit XL	B-705130072
Patella Cup S / L	25029
Patella Cup XL	B-711130000

## REBOUND® POST-OP KNEE

The simple, versatile post-operative knee bracing solution that is easy for everyone. Whether adjusting ROM, setting the drop lock, or altering the brace length, you'll appreciate the simplicity of the Rebound Post-Op knee brace.



### INDICATIONS FOR USE

- ACL, PCL, MCL, LCL repairs/injuries
- Meniscal repairs
- Tibial plateau fractures
- Patellar tendon repairs
- Osteochondral repairs
- Condylar fractures
- Sprains/strains of the knee
- HTOs (High Tibial Osteotomy)

### FEATURES AND BENEFITS

- Tool-free design and multi-purpose Smart-Fit™ packaging
- Available in universal, left or right contour and paediatric
- Telescoping frame from 47cm-68cm (Adult) and 39.5cm - 54cm (paediatric)
- OneTouch™ easy to adjust ROM hinge with lock-out capability
- Colour-coding and numbered frame for patient education
- Quick-fit, gripping buckles for easy application

### REQUIRED MEASUREMENTS

- Circumference: 15cm above mid-patella

### ORDERING INFORMATION

#### REBOUND POST-OP

Description	Thigh circumference	Cool	Full foam
Paediatric	30.5 - 63.5cm	B-8381470	
Universal, Left contoured	41 - 81cm	B-8381680	B-8382680
Universal, Right contoured	41 - 81cm	B-8381580	B-8382580
Universal	41 - 81cm	B-8381480	B-8382480
Universal XL	41 - 96,5cm	B-8381490	B-8382490

#### ACCESSORIES

Description	Part#	Description	Part#
ROM stop locks	B-8380005	Standard buckle & pad kit	B-8381096
Alligator hook & loop kit	B-8380006	AMS Kit Paediatric	B-8381093
Adhesive Hook & Loop kit Paediatric	B-8380008	AMS Kit Regular	B-8381094
Adhesive Hook & Loop kit universal / universal XL	B-8380007	AMS Kit XL	B-8381095

#### REPLACEMENT SOFTGOODS

Description	Cool	Full foam
Paediatric	B-8381072	
Universal*	B-8381082	B-8382082
Universal XL	B-8381092	B-8382092

\*For universal, left contoured, right contoured

## FORMFIT POST-OP KNEE

### INDICATIONS FOR USE

- Post-op/post-injury ROM control or immobilisation
- Indicated for ligament and fracture-related surgeries and when controlled range of motion is necessary

### FEATURES AND BENEFITS

- Breakaway struts for easy, tool free length adjustment
- OneTouch™ easy to adjust ROM hinge with lock-out capability from -10° to 30° (10° incremental)
- Extension control from -10° to 90° (10° incremental)
- Flexion control from 0° to 120° (10° incremental)
- Available in cool or foam models, 3 lengths, and two sizes (Regular and XL)



### REQUIRED MEASUREMENTS



Measure thigh circumference  
15 cm above mid-patella.  
Leg in extension.

Size	Thigh circumference 15 cm above mid patella
XL	69 - 89cm
Regular	0 - 69cm

### ORDERING INFORMATION

Size	Thigh circumference	Cool	Full Foam
Knee PO 61cm	UP TO 69cm	FPOK403100	FPOK403200
Knee PO XL 61cm	69 - 89cm	FPOK403102	FPOK403202
Knee PO 71cm	UP TO 69cm	FPOK403101	FPOK403201
Knee PO XL 71cm	69 - 89cm	FPOK403103	FPOK403203
Knee PO 41cm	69cm	-	FPOK403300
Knee PO XL 41cm	69 - 89cm	-	FPOK403301

### SOFT GOODS

Description	Cool	Full Foam
Replacement Softgood, Regular	403175	403275
Replacement Softgood, XL	403176	403277
Replacement Softgood, Regular Long	403276	403278

### REPLACEMENT SOFTGOODS

Size	
ROM stop locks (set of 20 lock clips)	B-8380005
Strap Reduction Kit Cool	403199
Strap Reduction Kit Full Foam	403299

## REBOUND® KNEE BRACE

With six variations to choose from the Rebound hinged knee brace product line is a comprehensive portfolio of high-quality braces designed to serve the needs of a broad range of patients.

### INDICATIONS FOR USE

- ROM Hinge Models: Mild to moderate ACL and / or PCL, MCL and LCL instabilities, sprains and strains
- Non-ROM Hinge Models: Mild medial / lateral knee instabilities

### FEATURES AND BENEFITS

- Highly breathable aerospace material that reduces heat retention and improves comfort
- Low profile hinges with extension and flexion stop kit options (ROM version) and malleable arms to provide a custom fit and knee movement control
- Contoured pre-tensed inset fabric that reduces irritation and material bulging in the popliteal area as well as inventory
- Removable straps allow application from right or left side and strap pull tab allow for consistent donning by patient
- Universal buttresses are included with all sleeve versions for customisable patella support

### REQUIRED MEASUREMENTS

- Circumference: 15cm above mid-patella



## ORDERING INFORMATION

### REBOUND ROM KNEE

Size	Circumference	Sleeve Long	Sleeve Short	Wrap Long	Wrap Short
XS	29 - 35cm	708152	708052	707152	707052
S	35 - 41cm	708153	708053	707153	707053
M	41 - 46cm	708155	708055	707155	707055
L	46 - 52cm	708157	708057	707157	707057
XL	52 - 57cm	708158	708058	707158	707058
XXL	57 - 63cm	708159	708059	707159	707059
XXXL	63 - 72cm			707160	707060

### REBOUND KNEE

Size	Circumference	Sleeve Short	Wrap Short
XS	29 - 35cm	704052	703052
S	35 - 41cm	704053	703053
M	41 - 46cm	704055	703055
L	46 - 52cm	704057	703057
XL	52 - 57cm	704058	703058
XXL	57 - 63cm	704059	703059
XXXL	63 - 72cm		703060



## REBOUND® KNEE IMMOBILIZER UNIVERSAL

Rebound® Knee Immobilizer Universal features a universal 3-panel design that can be adjusted to accommodate most leg circumferences. With an MR\* Conditional rating, patients can be safely scanned in an MR system, while maintaining immobilisation.

\*See IFU for details.



### INDICATIONS FOR USE

- Emergency immobilisation
- Pre- and post-surgical knee immobilisation
- Traumatic knee injuries

### FEATURES AND BENEFITS

- Adjustable medial/lateral stays and rigid posterior stays provide motion restriction of the knee
- Universal 3-piece design simplifies inventory needs
- MR\* Conditional: safely scan patients in an MR system while maintaining immobilisation
- Min thigh circumference: 26cm
- Max thigh circumference: 67.31cm

### ORDERING INFORMATION

Length	Part#
30 cm	210012
35 cm	210014
40 cm	210016
45 cm	210018
50 cm	210020
55 cm	210022
60 cm	210024

### ACCESSORIES

Description	Part#
Hot/Cold Therapy Gel Pad	GB7
PCL Pad	B-705000036



PCL Pad



## FORMFIT® PRO KNEE

Össur Formfit® Pro Knee is a compressive knee sleeve that features a SmartPad™ buttress for targeted support and cushioning. Our patented design realigns the patella, suitable for conditions such as Patella Tendonitis (Jumper's Knee) and Chondromalacia Patella. The dynamic, 3-D MotionTech™ knit is supportive, meets certified medical grade compression requirements and has been proven to be superior to the leading brands at moisture wicking and breathability, keeping your knee cool during wear. Lightweight, removable spiral stays ensure optimum shape and fit, while distinct patella graphics make donning intuitive.



MotionTech™



CoolVent™



StayBilizer™



SmartPad™

### INDICATIONS FOR USE

For knee conditions that may benefit from compression, such as;

- Patella tendonitis
- Chondromalacia Patella
- Mild sprains and strains
- Knee pain
- Feeling of instability


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.


### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit with superior moisture wicking and breathability, keeping your knee cool during wear
- Certified medical grade compression (20-30 mmHg)
- Lightweight CoolVent™ knit in popliteal area for added breathability and comfort
- Patented SmartPad™ for targeted support and cushioning
- Patella graphics for intuitive placement
- Patented FoldLock™ system for easier and more secure application
- Lightweight, removable spiral steel stays

### REQUIRED MEASUREMENTS

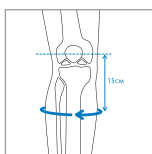

Measure calf circumference  
15 cm below mid-patella.  
Leg in extension.

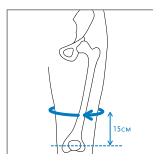

Measure thigh circumference  
15 cm above mid-patella.  
Leg in extension.

Size	Calf circumference 15 cm below mid patella	Thigh circumference 15 cm above mid patella
XS	26 - 31cm	36 - 41cm
S	31 - 36cm	41 - 46cm
M	36 - 41cm	46 - 51cm
L	41 - 46cm	51 - 56cm
XL	46 - 50cm	56 - 60cm

# FORMFIT® PRO KNEE

## ORDERING INFORMATION

### BLACK

Size	Part#
XS	FP100001
S	FP100002
M	FP100003
L	FP100004
XL	FP100005

### BLUE

Size	Part#
XS	FP101001
S	FP101002
M	FP101003
L	FP101004
XL	FP101005



## FORMFIT® TRACKER

The Formfit® Tracker with micro-adjustable ReflexWing™ buttress realigns the knee cap, providing dynamic lateral support for common knee cap conditions. Its FlyWeight™ fabric delivers lightweight breathability and a low-profile design that feels great against the skin and minimises migration. PowerLock™ straps work in tandem with heat mouldable CustomFit™ hinges to provide a customisable fit for your patient.

### INDICATIONS FOR USE

For knee conditions that may benefit from realignment of the knee cap and/or patients that would benefit from an added lateral support of the knee cap, such as:


- lateral knee cap subluxation or dislocation
- lateral maltracking

### FEATURES AND BENEFITS

- CustomFit™ hinges for customised fit and support
- Anti-Migration Tech™ minimises migration
- PowerLock™ straps for secure lateral support
- ReflexWing™ buttress for comfortable knee cap realignment
- FlyWeight™ fabric for soft feel and breathability
- Durable and machine washable



## REQUIRED MEASUREMENTS & ORDERING INFORMATION

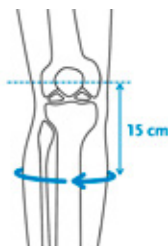

Calf circumference 15 cm below mid patella



Thigh circumference 15 cm above mid patella

Size	Calf circumference 15 cm below mid patella	Thigh circumference 15 cm above mid patella	Left	Right
XS	25 - 30cm	33 - 39cm	719162	719152
S	30 - 35cm	39 - 47cm	719163	719153
M	35 - 40cm	47 - 53cm	719164	719154
L	40 - 46cm	53 - 60cm	719165	719155
XL	46 - 51cm	60 - 67cm	719166	719156
XXL	51 - 56cm	67 - 75cm	719167	719157
3XL	56 - 61cm	75 - 81cm	719168	719158

## ACCESSORIES

Description	Part#
ReflexWing™ Buttress	719100

## FORMFIT® PRO KNEE FLITE

### INDICATIONS FOR USE

For knee conditions that may benefit from compression, such as:

- Mild sprains and strains
- Knee pain
- Feeling of instability


### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit
- CoolVent™ knit in popliteal area for added breathability
- Patented FoldLock™ system for easy and secure application

Lightweight, removeable spiral steel stays



### REQUIRED MEASUREMENTS

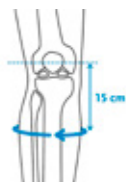

Measure calf circumference  
15 cm below mid-patella.  
Leg in extension.



Measure thigh circumference  
15 cm above mid-patella.  
Leg in extension.

Size	Calf circumference 15 cm below mid patella	Thigh circumference 15 cm above mid patella
XS	26 - 31cm	36 - 41cm
S	31 - 36cm	41 - 46cm
M	36 - 41cm	46 - 51cm
L	41 - 46cm	51 - 56cm
XL	46 - 50cm	56 - 60cm

### ORDERING INFORMATION

#### BLUE

Size	Part#
XS	FP141001
S	FP141002
M	FP141003
L	FP141004
XL	FP141005



#### BLACK

Size	Part#
XS	FP140001
S	FP140002
M	FP140003
L	FP140004
XL	FP140005

## FORMFIT® HINGED LATERAL J

Formfit Knee Hinged Lateral J is a sleeve design knee brace with a breathable Lycra-material and open popliteal area with two removeable hinges and J-shaped buttress providing support to the patella. The knee brace is designed for patients that would benefit from realignment of the patella, and/or patients that would benefit from an added lateral support of the patella.

### INDICATIONS FOR USE

Patients that would benefit from realignment of the patella, and/or patients that would benefit from an added lateral support of the patella.

- Lateral patellar subluxations and dislocations.
- Lateral patellofemoral malalignment.

### FEATURES AND BENEFITS

- Breathable material
- Removable hinges
- Easy pull-on
- Lateral J buttress
- Open popliteal area

### REQUIRED MEASUREMENTS

- Thigh Circumference: 15cm above mid-patella
- Calf Circumference: 15cm below mid-patella



## ORDERING INFORMATION

Size	Thigh circumference	Knee circumference	Calf circumference	Right leg	Left leg
XS	33 - 39.4cm	30.5 - 33cm	25.4 - 30.5cm	709152	709162
S	39.4 - 47cm	33 - 35.6cm	30.5 - 35.6cm	709154	709164
M	47 - 53.3cm	35.6 - 38.1cm	35.6 - 40.6cm	709155	709165
L	53.3 - 59.7cm	38.1 - 43.2cm	40.6 - 45.7cm	709157	709167
XL	59.7 - 67.3cm	43.2 - 48.3cm	45.7 - 50.8cm	709158	709168
XXL	67.3 - 74.9cm	48.3 - 53.3cm	50.8 - 55.9cm	709159	709169

## FORMFIT® KNEE HINGED

The Formfit Knee brace is for patients suffering from mild to moderate strains, sprains or ligament tears (ACL/PCL/MCL/LCL). It provides excellent stability, protection and compression during injury recovery. Made from strong and breathable materials it offers comfort while remaining highly durable. The open popliteal area reduces material bulging and potential irritation, while the malleable rigid hinge arms provide an intimate fit.

The Formfit Knee brace comes in either a wrap or a sleeve version.

### INDICATIONS FOR USE

- Mild to moderate ACL and PCL tears
- MCL and LCL tears
- Combined mild knee sprains and strains
- Instabilities of the knee

### FEATURES AND BENEFITS

- Breathable material
- Easy pull-on
- Malleable hinges
- Universal buttress
- Open popliteal area

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



## ORDERING INFORMATION

Size	Knee Circumference	Sleeve	Wrap
XS	33cm to 39cm	504252	
S	39cm to 47cm	504253	503253
M	47cm to 53cm	504255	503255
L	53cm to 60cm	504257	503257
XL	60cm to 67cm	504258	503258
XXL	67cm to 75cm	504259	503259
XXXL	75cm to 81cm		503260



## FORMFIT® KNEE MCL

The Formfit Knee MCL provides external stability and protection for low to medium contact/impact activities and while recovering from injury. A unique, single hinged, hybrid wrap/sleeve design sets this brace apart from the competition. The lateral, range of motion hinge keeps the medial side free of obstruction. The Dynamic Force Strap aids in the resistance of valgus forces during knee extension, while the adjustable tibial strap improves suspension by minimising slippage.

### INDICATIONS FOR USE

- Mild to Moderate Medial Collateral Ligament (MCL) sprains (Grade I and Grade II)

### FEATURES AND BENEFITS

- Single, steel ROM lateral hinge (0° or 20° extension stop, 70° flexion stop)
- Dynamic Force Strap resist valgus forces
- Hybrid wrap/sleeve for easy application
- Adjustable tibial strap for improved suspension



### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella

### ORDERING INFORMATION

Size	Thigh Circumference	Formfit Knee MCL Left	Formfit Knee MCL Right
XS	33 - 39cm	B-237600001	B-237500001
S	39 - 47cm	B-237600002	B-237500002
M	47 - 53cm	B-237600003	B-237500003
L	53 - 60cm	B-237600004	B-237500004
XL	60 - 67cm	B-237600005	B-237500005

# FORMFIT® KNEE ROM BRACE

## INDICATIONS FOR USE

- Mild to moderate ACL and PCL tears
- MCL and LCL tears
- Combined mild knee sprains and strains
- Instabilities of the knee

## FEATURES AND BENEFITS

- With 6 options to choose from, you can confidently meet the unique needs of any patient
- When you need to limit range-of-motion, our ROM hinge provides the control you need. When you don't, you can opt for a more streamlined, polycentric hinge
- ROM braces available in 12" or 16" Length
- A removable, universal buttress is included with every sleeve version of the Formfit Knee Brace for additional, customisable patella support, if necessary



## REQUIRED MEASUREMENTS

- Knee circumference at mid-patella

## ORDERING INFORMATION

Size	Knee Circumference	Sleeve Short	Wrap Short	Sleeve Long	Wrap Long
XS	33cm to 39cm	508252	507252	508352	507352
S	39cm to 47cm	508253	507253	508353	507353
M	47cm to 53cm	508255	507255	508355	507355
L	53cm to 60cm	508257	507257	508357	507357
XL	60cm to 67cm	508258	507258	508358	507358
XXL	67cm to 75cm	508259	507259	508359	507359
XXXL	75cm to 81cm		507260		507360

## NEOPRENE HINGED KNEE SUPPORT

### INDICATIONS FOR USE

For patients suffering from knee conditions that may benefit from additional knee support, such as:

- Patellar tendonitis
- Subluxation dislocation
- Post-operation rehabilitation
- Post arthroscopic surgery
- Chondromalacia
- Bursitis
- Arthritis
- Tendonitis

### FEATURES AND BENEFITS

- Dual action hinges are moulded into the brace to maintain a low profile
- Accordion molding in the popliteal area reduces bunching behind the knee
- Lycra liner offers a breathable contact area for comfortable and extensive use
- Varied areas of thickness give additional protection

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



### ORDERING INFORMATION

Size	Knee circumference	Part#
S	29 - 33cm	302153BLK
M	33 - 37cm	302155BLK
L	37 - 41cm	302157BLK
XL	41 - 44cm	302158BLK
XXL	44 - 48cm	302159BLK

## NEOPRENE KNEE SUPPORT WITH STABILISED PATELLA

### INDICATIONS FOR USE

For patients suffering from knee conditions that may benefit from additional knee and patella support, such as: patellar tendonitis, subluxation dislocation, post-operation rehabilitation, post arthroscopic surgery, chondromalacia, bursitis, arthritis and tendonitis.

### FEATURES AND BENEFITS

- Moulded material to maintain a low profile
- Accordion moulding in the popliteal area eliminates bunching

### REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



### ORDERING INFORMATION

Size	Knee circumference	Part#
S	11.5 - 13"	302563BLK
M	13 - 14.5"	302565BLK
L	14.5 - 16"	302567BLK
XL	16 - 17.5"	302568BLK
XXL	17.5 - 19"	302569BLK

# NEOPRENE WRAPAROUND HINGED KNEE SUPPORT

## INDICATIONS FOR USE:

For patients with unstable or injured knees that may benefit from additional knee support, such as: post-operation rehabilitation, tendonitis, bursitis, arthritis, sprains and strains.

## FEATURES AND BENEFITS

- Dual action hinges are removable for easy cleaning
- Open popliteal area eliminates bunching and increases comfort
- Front contact closures allow for hassle-free application
- Wraparound style accommodates all leg shape variations

## REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



## ORDERING INFORMATION

Size	Knee circumference	Part#
S	33 - 35.5cm	212003
M	35.5 - 38cm	212005
L	38 - 40.5cm	212007
XL	40.5 - 43cm	212008
XXL	43 - 45.5cm	212009
XXXL	45.5 - 58.5cm	212010
XXXXL	58.5 - 63.5cm	212011

# NEOPRENE KNEE SLEEVE

A neoprene sleeve-design knee support providing compression and warmth

## INDICATIONS FOR USE

- For patients suffering from weak and/or injured knees that may benefit from additional knee support.

## FEATURES AND BENEFITS

- Unique accordion design enhances comfort by eliminating bunching in the popliteal area
- Open or closed patella

## REQUIRED MEASUREMENTS

- Knee circumference at mid-patella



## ORDERING INFORMATION

		0.3cm Neoprene		0.6cm Neoprene	
Size	Circumference	Open Patella	Closed Patella	Open Patella	Closed Patella
S	29 - 33cm	402013BLK	412013BLK	302013BLK	312013BLK
M	33 - 37cm	402015BLK	412015BLK	302015BLK	312015BLK
L	37 - 41cm	402017BLK	412017BLK	302017BLK	312017BLK
XL	41 - 44cm	402018BLK	412018BLK	302018BLK	312018BLK
XXL	44 - 48cm	402019BLK	412019BLK	302019BLK	312019BLK

# MEASUREMENT FORM CTi® CUSTOM BRACE

## CUSTOM MEASUREMENT SYSTEM

Date: \_\_\_\_\_ Customer #: \_\_\_\_\_ Purchase order #: \_\_\_\_\_

### Required patient information

Patient ID: \_\_\_\_\_ Diagnosis: \_\_\_\_\_

Height: \_\_\_\_\_ cm. Weight: \_\_\_\_\_ kg. Date of Birth: \_\_\_\_\_ Sex(m/f): \_\_\_\_\_ ☐ Left leg ☐ Right leg

☐ Bilats - l/r

### Customer:

Contact: \_\_\_\_\_ Prescribing consultant: \_\_\_\_\_

Phone: \_\_\_\_\_ Clinic: \_\_\_\_\_

Company: \_\_\_\_\_

Please fill out all mandatory fields  
(marked in bold)

### ☐ VAPOR

(For low to medium impact levels)

### ☐ STANDARD\*

(For medium to high impact levels)

### ☐ PRO SPORT

(For highest impact levels)

### OPTIONS

#### D-ring attachment

☐ Screwed ☐ Riveted\*

☐ Internal ☐ External\*

#### Thigh Liner

☐ None\* ☐ Sensil

☐ Evazote/Foam

#### Calf Liner

☐ Sensil\* ☐ Evazote/Foam

#### Extension Stops

☐ 0° ☐ 10°\* ☐ 20° ☐ 30° ☐ 40°

### SPECIAL MODIFICATIONS

(Additional charges may apply)

#### ☐ Flexion Stop Kit

\_\_\_\_\_° (0-90° installed)

#### ☐ PCL System

(Helps reduce posterior drawer)

#### ☐ ACL Cable Kit (Additional stability in ext)

#### ☐ Hyperextension Straps (5th strap added)

#### ☐ Ski Boot Attachment (Brace-to-boot)

☐ Other: \_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

### SPECIAL NOTES:

☐ Please contact me about this order

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

\*This configuration is delivered by default, if nothing is filled in.

### SPECIAL CONDITIONS:

#### ☐ OA (Osteoarthritis unloading)

☐ Medial

☐ Lateral \_\_\_\_\_°

(Choose 1° to 7° of unloading, 4°\*)

#### ☐ Prominent VMO

#### ☐ Prominent VLO

#### ☐ Hockey Quads

#### ☐ Prominent Tibialis Anterior

#### ☐ Prominent Fibular Head

#### ☐ RTM

(Rounded Tibial Member)

#### ☐ Osgood-Schlatters Disease

#### ☐ Super Short

(For 1.60m and under)

#### ☐ Shorten Frame Component

Tibial ☐ 1.3cm ☐ 2.6cm ☐ 3.9cm

Femoral ☐ 1.3cm ☐ 2.6cm ☐ 3.9cm

### COLOURS:

Select finish:

☐ MATTE\* ☐ GLOSS

Select colour:

☐ Black\* ☐ Yellow

☐ Ocean Blue ☐ Orange

☐ White ☐ Lime Green

☐ Silver ☐ Pink

☐ Red ☐ Charcoal

☐ Champagne ☐ Navy Blue

### ACCESSORIES:

(Bolded are free with initial order only)

#### ☐ AMS WRAP

☐ Neoprene\* ☐ Evazote

#### ☐ Sofsleeve

#### ☐ Neoprene Undersleeve

#### ☐ Neoprene Oversleeve

#### ☐ Padded Sports Oversleeve

#### ☐ Sleeve with Mesh

#### ☐ MX Patella Cup

#### ☐ MX Gear Guards

#### ☐ MX Complete Kit (Patella Cup, Gear Guards, Sleeve with Mesh)

☐ Other: \_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

### CUSTOM PAINT:

(Additional fee plus extra delivery time)

Select custom finish:

☐ MATTE\* ☐ GLOSS

Select custom colour:

☐ Hibiscus ☐ Sports Equipment

☐ Flag ☐ Camouflage

☐ Flames ☐ Single Colour

☐ Metal ☐ 2-colour Fade

☐ Sponge ☐ 2-colour Half & Half

☐ Celestial ☐ Sports Fan

#### ☐ Special

(Provide design file with order)

# MEASUREMENT FORM CTi® CUSTOM BRACE

## CUSTOM MEASUREMENT SYSTEM

### CUSTOM MEASURING SYSTEM

☐ Left leg ☐ Right leg

1: \_\_\_\_\_ Hinge Width

2: \_\_\_\_\_ Hinge Depth

**LATERAL MEASUREMENTS**

3: \_\_\_\_\_ Contour

4: \_\_\_\_\_ A-P

5: \_\_\_\_\_ Contour

6: \_\_\_\_\_ A-P

7: \_\_\_\_\_ Contour

8: \_\_\_\_\_ A-P

9: \_\_\_\_\_ Tibia

10: \_\_\_\_\_ Tibia

11: \_\_\_\_\_ Contour

12: \_\_\_\_\_ A-P

13: \_\_\_\_\_ Tibia

**MEDIAL CONTOURS**

14: \_\_\_\_\_ Contour

15: \_\_\_\_\_ Contour

**JOINT SPACE**

16: \_\_\_\_\_ Contour

17: \_\_\_\_\_ Contour

**CIRCUMFERENCES (in centimeters)**

18. Level A \_\_\_\_\_

19. Level B \_\_\_\_\_

20. Level C \_\_\_\_\_

21. Level E \_\_\_\_\_

22. Distal Border of Patella \_\_\_\_\_

23.

Caliper measurement at joint space (1.6 - 2.0 cm less than #1)  
Take measurement in a weight bearing position at full extension

**SHORTEN TIBIA COMPONENT/ SKI BOOT MEASUREMENT**

\_\_\_\_\_ cm from the distal border of the patella to the top of the boot. Ski boot attachment is not available if the measurement is less than 18cm.

### CMS INSTRUCTIONS

Patient should not be wearing shoes for the measurement procedure.

There should be no space between the block and the back of the knee. The system will not be accurate if the knee is measured in a flexed position.

1. Slide the lateral pad to the knee. Tighten to patient's individual comfort level. This measurement will be the actual width of the patient's knee brace.

2. Pre-set hinge depth at approximately 3.5cm, adjust if needed.

3. – 13. Patient should remain still with foot dorsiflexed throughout the lateral measurements and recording process.

9., 10. and 13. Measure using the horizontal bar of the tool measure the position of the highest point of the tibialis anterior.

• 9. As reference take the #1 hinge width / 2 - 1 = ...

(for example 13cm / 2 - 1 = 5.5cm)

• 10. should be 0.5cm less than 9.

• 13. should be 1cm less than 9.

14. – 17. The patient may now relax throughout measuring the medial contours from the contracted position. Please note that with larger legs this measurement may be a negative value. Please read the numbers carefully and record negative values if necessary, such as -0.5 or -1.5


18. – 22. Take the measurements with a soft measuring tape ensuring that the tape is snug but not too loose/tight.




# MEASUREMENT FORM CTi® CUSTOM BRACE

CUSTOM MEASUREMENT SYSTEM


## SPECIAL CONDITIONS REFERENCES

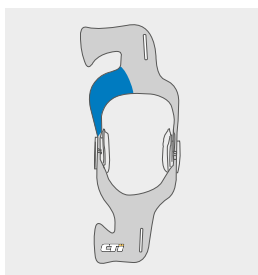


Prominent VLO

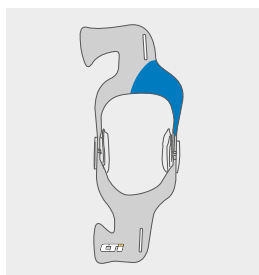


Prominent VMO

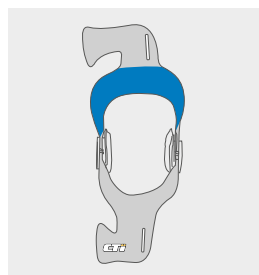


Hockey Quads

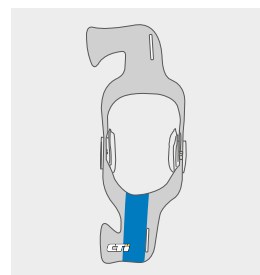


Prominent Tibialis Anterior

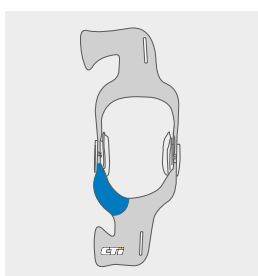


Prominent Fibular Head

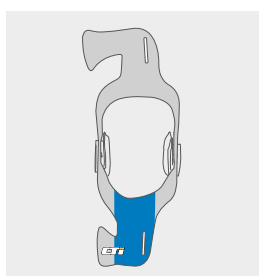

RTM  
(Rounded Tibial Member)

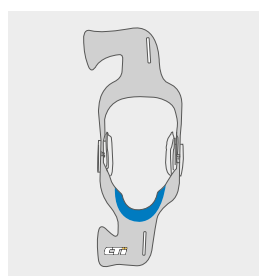


Osgood-Schlatters Disease

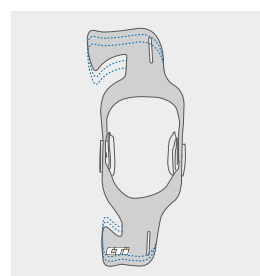

Shorten Femoral Component  
Shorten Tibial Component



Super Short

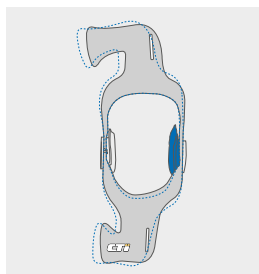

Osteoarthritis Unloading

To find out more about CTi custom, please scan the QR code



# MEASUREMENT FORM UNLOADER ONE® CUSTOM / REBOUND® ACL & PCL

FOR USE WITH DMS KIT

## PATIENT INFORMATION:


Name:			
Date of Birth:	Contact No.		
Height:		Weight :	
Diagnosis:	Prescribing consultant :		

## MEASUREMENTS:

M-L Caliper	Left	Right
Circumferences		
20cm Above mid patella:		
15cm Above mid patella:		
7.5cm Above mid patella :		
7.5cm Below mid patella:		
15cm Below mid patella:		
20cm Below mid patella:		

## BRACE DETAILS:

Left / Right / Bilateral:				
Unloader® One:	Medial	<input type="checkbox"/>	Lateral	<input type="checkbox"/>
Unloader® Plus:	Medial	<input type="checkbox"/>	Lateral	<input type="checkbox"/>
Rebound®:	ACL	<input type="checkbox"/>	PCL	<input type="checkbox"/>
Colour Options:				
Notes:				

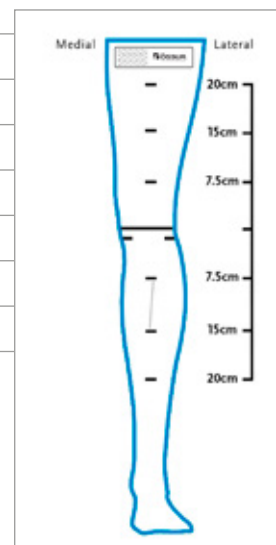

## FILE NAME:

	Anterior	Lateral
Left		
Right		

**Product Warranty** - Custom Made Knee Braces – Össur warrants the hinge components to be free of defects in material and workmanship for the lifetime of the patient. With regard to components of the brace that wear with use, which include straps, liners, covers, sleeves, return cords and hinge coatings, these components shall be warranted free from defect in material and workmanship for 6 months. Prefabricated Knee Braces – Össur warrants the frame and hinge components to be free of defects in material and workmanship for two years. With regard to components of the brace that wear with use, including straps, liners, covers, sleeves, return cords and hinge coatings, these components shall be warranted free from defect in material and workmanship for 6 months. Return Policy - Upon receipt of the Össur knee brace, the provider shall inspect it to ensure that it meets the requirements set forth in the order. If the brace does not meet the order requirements, the provider must call the Össur Customer Care Department at 03450 065 065, Monday to Friday, 8:30am to 5:00pm. Prior to returning a brace for any reason, including product warranty or patient satisfaction issues, the provider is required to obtain a Return Authorisation (RA) number from Customer Care. Final acceptance of all returned items remains the discretion of the Össur Quality Control Department. Satisfaction guarantee and product warranty time frames begin on the date the brace is delivered to the patient. Proof of brace delivery, signed and dated by the patient, is required for processing of guarantee and warranty returns. Satisfaction guarantee does not apply to the Össur post-operative braces. The stabilising effect of a knee brace will depend on the type of brace, patient age, height, weight, muscle tone, severity of the injury/OA, the type of injury and the medical treatment involved. A knee brace is designed to reduce the potential and severity of a knee injury and is not a guarantee against injury. There are no warranties expressed or implied made by Össur on their braces except the warranties against defects in materials or workmanship. Liability is limited to replacement or repair of defective parts or workmanship. Liability for incidental or consequential damage, personal injury or property damage as a result of defect is specifically denied. Össur neither assumes nor authorises any other person to assume for it any other liability for its braces.

## FOOT AND ANKLE



## TABLE OF CONTENTS

<b>FOOT AND ANKLE.....</b>	<b>125</b>
REBOUND® DIABETIC WALKER .....	127
DH OFFLOADING WALKER® .....	129
REBOUND® AIR WALKER .....	130
FORMFIT® WALKER AIR .....	132
FORMFIT® WALKER .....	133
PAEDIATRIC WALKER .....	134
AFO DYNAMIC .....	135
AFO LIGHT .....	137
PROLITE AFO .....	138
EXTRA STRONG PROLITE .....	138
REBOUND FOOT-UP® .....	139
FOOT-UP® .....	140
FORMFIT® PRO ANKLE .....	141
FORMFIT® ANKLE BRACE .....	143
FORMFIT® ANKLE WITH SPEEDLACE .....	144
FORMFIT® ANKLE STIRRUPS .....	145
EXOFORM® DORSAL NIGHT SPLINT .....	146
FORMFIT® NIGHT SPLINT .....	147
DYNAPRO® AFO .....	148
SOFTPRO® GAIT TRAINER AFO .....	149
SOFTPRO® IN-BED AFO .....	150
DH OFFLOADING POST-OP STYLE SHOE .....	151
MESH TOP POST-OP SHOE .....	152
SOFT TOP POST-OP SHOE .....	152
CANVAS ROCKER BOTTOM CAST SHOE .....	153

## REBOUND® DIABETIC WALKER

Offloading has proven to be a key factor in foot ulcer treatment and entails reducing pressure of the plantar area in general and the ulcer site in particular.

The Rebound Diabetic Walker is uniquely designed for effective treatment of plantar foot ulcers, with easy access and re-fitting during wound care, while improving the patient's mobility during the healing process.

The Rebound Diabetic Walker has major advantages over diabetic shoes and shoe inserts in its healing effectiveness and over total contact casting due to its ease of use, speed of application and low cost.

### FUNCTIONAL HEALING - WHERE MOBILITY MEANS RECOVERY



## INDICATIONS FOR USE

- Plantar foot ulcers and wound prevention

## FEATURES AND BENEFITS

- Effective tri-laminate insole with removable shock-absorbing hexagons that promote ulcer offloading and a Plastazote, heat-moldable top layer that evenly distributes the pressure and minimises friction
- Unique pressure relief valve for preventing over-inflation, while balancing compression with customised fit
- Positioning Portals allow the clinician to check the fit of the walker at the heel bottom, in order to ensure appropriate positioning of the foot in the walker
- Anatomically designed shell that minimises shear forces by immobilising the foot and ankle at 90 degrees while providing protection
- Flex Edge® technology for relief of pressure points
- Rocker Bottom outsole for a smooth roll over while allowing for individual natural toe out
- Soft and breathable, low friction, machine washable liner for hygiene promotion and prevention of infection
- Tamper evident strapping for compliance monitoring

## REQUIRED MEASUREMENTS

- Shoe size





# REBOUND® DIABETIC WALKER

## ORDERING INFORMATION

### REBOUND DIABETIC WALKER

Size	Shoe size UK	Shoe size EU	Part#
S	4 - 6.5	36 - 39	B-242500002
M	7 - 10	39.5 - 44	B-242500003
L	10.5 - 12	44.5 - 46.5	B-242500004
XL	12.5 - 14.5	47 - 50	B-242500005

## ACCESSORIES

### REPLACEMENT SOFTGOODS

Size	Liners	Insoles
S	B-418500102	B-418500202
M	B-418500103	B-418500203
L	B-418500104	B-418500204
XL	B-418500105	B-418500205

### WALKER COVERS

Size	1. Hygiene covers (2 pcs)	2. Durable weather cover	3. Disposable weather cover (20 pcs)
S	B-418500602	B-418500502	B-418500512
M	B-418500603	B-418500503	
L	B-418500604	B-418500504	B-418500514
XL	B-418500605	B-418500505	



1



2



3

### DIABETIC SOCKS (2 PCS)

Size	Part#
S, M	B-418500703
L, XL	B-418500704

### COMPLIANCE STRAPS (5 PCS)

Size	Part#
Universal	B-418500300

### EVENUP

Size	Part#
S	10350
M	10351
L	10352



## DH OFFLOADING WALKER®

Assists in the healing of plantar ulcers with a patented pressure relief insole

### INDICATIONS FOR USE

- Treatment of plantar ulcers

### FEATURES AND BENEFITS

- Shear force reduction by stabilising foot and ankle at 90°
- Wide low profile rocker bottom provides a smooth roll-through and increased stability for altered gaits
- Designed around Össur's unique Active Offloading Insole
- The softgood's patented inner bootie prevents the foot from migrating on the insole
- Provides a safe environment for the simultaneous use of wound care material
- Create an Instant TCC using a single roll of cast tape for patient compliance



### REQUIRED MEASUREMENTS

- Shoe size

## ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Part#
XS	1.5 - 3.5	33 - 35.5	DH0200BLK
S	4 - 6.5	36 - 39	DH0400BLK
M	7 - 10	39.5 - 44	DH0600BLK
L	10.5 - 12	44.5 - 46.5	DH0800BLK
XL	12.5 - 14.5	47 - 50	DH1000BLK

## ACCESSORIES

### REPLACEMENT PARTS

Size	Softgoods	Insoles
XS		11742C
S	10958	PP04C
M	10959	PP06C
L	10960	11740C
XL		11741C



## REBOUND® AIR WALKER

Rebound Air Walker is a circumferential walker with fully integrated air pump and release valve that provides easy, tool-free inflation and deflation for fine-tuned, consistent compression. It features a host of details designed to provide added protection, simplified application and usage.



### INDICATIONS FOR USE

Foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- Tendon or ligamentous sprain
- During postoperative or rehab phase. If used with the Achilles Wedge Kit, Achilles ruptures can be treated as well.
- Rebound Air Walker Low Top
- Foot and ankle conditions that may benefit from motion restriction, such as:
- Stable foot fractures
- Tendon or ligamentous sprain
- During postoperative or rehab phase.

### FEATURES AND BENEFITS

- Linked medial and lateral air bladders with a single inflation point provide balanced compression and are shaped to cradle the Achilles, holding the heel in place
- Rocker bottom promotes a natural, stable gait, while a trimmable, semi-soft toe bumper provides added protection
- Moisture-wicking softgood helps keep skin dry and comfortable
- Straps fasten in front for easy application, adjustment and removal
- EVA material provides greater shock absorption in the heel
- Fully-integrated air pump and release valve provides easy, tool-free inflation and deflation for fine-tuned, consistent compression
- Flex Edge™ overmolding conforms to each patient's unique anatomy to help limit the formation of pressure points
- Ventilated panels wick moisture to prevent microbial buildup and improve patient comfort

### REQUIRED MEASUREMENTS

- Shoe size



# REBOUND® AIR WALKER

## ORDERING INFORMATION

### REBOUND AIR WALKER

Size	Shoe Size UK	Shoe Size EU	Standard Liner / Standard	Standard Liner / Low Top	Slim Liner / Standard	Slim Liner / Low Top
XS	1.5 - 3.5	33 - 35.5	B-242900001E	B-242900061E	B-242900001SE	B-242900061SE
S	4 - 6.5	36 - 39	B-242900002E	B-242900062E	B-242900002SE	B-242900062SE
M	7 - 10	39.5 - 44	B-242900003E	B-242900063E	B-242900003SE	B-242900063SE
L	10.5 - 12	44.5 - 46.5	B-242900004E	B-242900064E	B-242900004SE	B-242900064SE
XL	12.5 - 14.5	47 - 50	B-242900005E	B-242900065E	B-242900005SE	B-242900065SE

## ACCESSORIES

### REPLACEMENT SOFTGOODS

Size	Standard Liner / Standard	Standard Liner / Low Top	Slim Liner / Standard	Slim Liner / Low Top
XS	B-418000101	B-418000161	B-418030101	B-418030161
S	B-418000102	B-418000162	B-418030102	B-418030162
M	B-418000103	B-418000163	B-418030103	B-418030163
L	B-418000104	B-418000164	B-418030104	B-418030164
XL	B-418000105	B-418000165	B-418030105	B-418030165

### ACHILLES HEEL WEDGE KIT

Size	Description	Part#
S	Wedge kit for walkers XS-S	AWK0001
M	Wedge Kit for walkers M	AWK0005
L	Wedge kit for walkers L-XL	AWK0009
N/A	5 layer Wedge kit	AWK00015

### EVENUP

Size	Part#
S	10350
M	10351
L	10352

### WALKER COVERS

Size	Hygiene covers (2 pcs)	Durable weather cover	Disposable weather cover (20 pcs)
S	B-418500602	B-418500502	B-418500512
M	B-418500603	B-418500503	
L	B-418500604	B-418500504	B-418500514
XL	B-418500605	B-418500505	

### OTHER ACCESSORIES

Description	Part#
Adhesive Insole for Wedge	WHW-W0107
Strap Extension	B-502100101
Rebound Hightop Sock SM/MD SM/MD	B-418500703
Rebound Hightop Sock LG/XL LG/XL	B-418500704
Rebound Lowtop Sock SM/MD SM/MD	B-418500903
Rebound Lowtop Sock LG/XL LG/XL	B-418500904

## FORMFIT® WALKER AIR

The Formfit Walker Air provides immobilisation for patients suffering from soft tissue injury; grades 2 & 3 strains and sprains, and stable fractures. The patented pneumatic system is designed to increase stability while decreasing pain and swelling. The intuitive pump and clearly marked decompression valve make air/pressure adjustments easy.

### INDICATIONS FOR USE

For foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- Soft tissue injuries
- During postoperative or rehab phase

### FEATURES AND BENEFITS

- Available in low top or standard height
- Integrated pneumatic system
- Low, wide rocker bottom promotes a stable, natural gait
- Shock absorbing sole for reduced impact at heel strike
- Breathable, seamless liner

### REQUIRED MEASUREMENTS

- Shoe size



## ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Standard	Low Top
XS	1.5 - 3.5	33 - 35.5	A-W0200BLK	A-W0300BLK
S	4 - 6.5	36 - 39	A-W0400BLK	A-W0500BLK
M	7 - 10	39.5 - 44	A-W0600BLK	A-W0700BLK
L	10.5 - 12	44.5 - 46.5	A-W0800BLK	A-W0900BLK
XL	12.5 - 14.5	47 - 50	A-W1000BLK	A-W1100BLK

## ACCESSORIES

### REPLACEMENT SOFTGOODS

Size	Standard	Low Top
XS	12349	12356
S	10675	12357
M	10678	12358
L	10681	12359
XL	12347	12360

### OTHER ACCESSORIES

Description	Part#
10cm softgood width extension	WKSG-1BLK
Tibia and calf plates for use when additional splinting is required	WKSP100
Hot/Cold therapy insert for safe and reusable cryotherapy	WKGEL100
5 layer Wedge kit	AWK0005
Adhesive insole for heel wedge	WHW-W0107

## FORMFIT® WALKER

The Formfit Walker provides immobilisation for patients suffering from soft tissue injury; grades 2 & 3 strains and sprains; and stable fractures.

### INDICATIONS FOR USE

For foot and ankle conditions that may benefit from motion restriction, such as:

- Grade 2 and 3 ankle sprains
- Stable fractures
- Soft tissue injuries
- During postoperative or rehab phase

### FEATURES AND BENEFITS

- Available in low top or standard height
- Low, wide rocker bottom promotes a stable, natural gait
- Shock absorbing sole for reduced impact at heel strike
- Breathable, seamless liner

### REQUIRED MEASUREMENTS

- Shoe size



## ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Standard	Low Top
XS	1.5 - 3.5	33 - 35.5	W0200BLK	W0300BLK
S	4 - 6.5	36 - 39	W0400BLK	W0500BLK
M	7 - 10	39.5 - 44	W0600BLK	W0700BLK
L	10.5 - 12	44.5 - 46.5	W0800BLK	W0900BLK
XL	12.5 - 14.5	47 - 50	W1000BLK	W1100BLK

## ACCESSORIES

### REPLACEMENT SOFTGOODS

Size	Standard	Low Top
XS	WKSG200BLK	WKSG0300BLK
S	WKSG400BLK	WKSG500BLK
M	WKSG600BLK	WKSG700BLK
L	WKSG800BLK	WKSG900BLK
XL	WKSG1000BLK	WKSG1100BLK

### OTHER ACCESSORIES

Description	Part#
10cm softgood width extension	WKSG-1BLK
Tibia and calf plates for use when additional splinting is required	WKSP100
Hot/Cold therapy insert for safe and reusable cryotherapy	WKGEL100
5 layer Wedge kit	AWK0005
Adhesive insole for heel wedge	WHW-W0107

## PAEDIATRIC WALKER

Paediatric Walkers are designed to stabilise and protect the lower leg, ankle and foot. They are easy to operate and comfortable to wear, making them ideal for an active child who requires stability following a sprain, stress fracture, or surgery.

### INDICATIONS FOR USE

- Soft tissue injuries
- Grade 2 and 3 sprains
- Stable fractures
- Post-operative ankle stabilisation
- Trauma and rehab use

### FEATURES AND BENEFITS

- High quality rigid outer skeleton for protection
- Dense foam lining within the sole to provide comfort and support
- Shock absorbing sole to improve comfort and protects the user from slips, providing additional security in each step
- Uprights give additional stability by restricting motion of the leg
- Lightweight design perfectly designed for kids to keep active

### REQUIRED MEASUREMENTS

- Shoe size



## ORDERING INFORMATION

### PAEDIATRIC WALKER

Size	Shoe Size	Part#
S	Child 6.5 - 9.5	PEW0400
M	Child 10 - Youth 1	PEW0600
L	Adult 1.5 - 2.5	PEW0800

### REPLACEMENT LINERS

Size	Shoe Size	Part#
S	Child 6.5 - 9.5	PEW0001
M	Child 10 - Youth	PEW0002
L	Adult 1.5 - 2.5	PEW0003

## AFO DYNAMIC

The AFO Dynamic provides a strong, lightweight solution for patients with varying degrees of drop foot secondary to a variety of neurological conditions. Össur has taken the carbon fibre technology from its Flex-Foot® product and incorporated it into the new AFO Dynamic to provide increased support over traditional ankle foot orthoses. The energy storing properties of carbon fibre make it an ideal material for this dynamic ankle foot orthosis because it provides real energy return. For those people leading busy lives, the AFO Dynamic will ensure a more efficient walk for the user.



### INDICATIONS FOR USE

Drop foot secondary to:

- Cerebro Vascular Accident (CVA)
- Other neurological conditions
- Multiple Sclerosis
- Mild knee instability
- Charcot Marie Tooth disease (HMSN)

Contraindications:


- Severe ankle - foot deformities
- Fluctuating oedema
- Severe spasticity
- Neuropathy

### FEATURES AND BENEFITS

- Carbon fibre design provides excellent strength-to-weight ratio and dynamic motion
- The tapered carbon heel stores energy and manages forces at heel strike
- Full-length toe lever provides full support while foot is loaded, as well as reduced shock to the contralateral limb
- Creates medio-lateral stability at mid-stance
- Reduces energy consumption by reducing hip hike
- Medial strut makes it less visible in shoe
- Open anterior tibia shell provides superior anatomical fit, and reduces heat and perspiration
- Weight limit: 120 kg

### REQUIRED MEASUREMENTS

- Shoe size
- Calf height at widest part of the calf

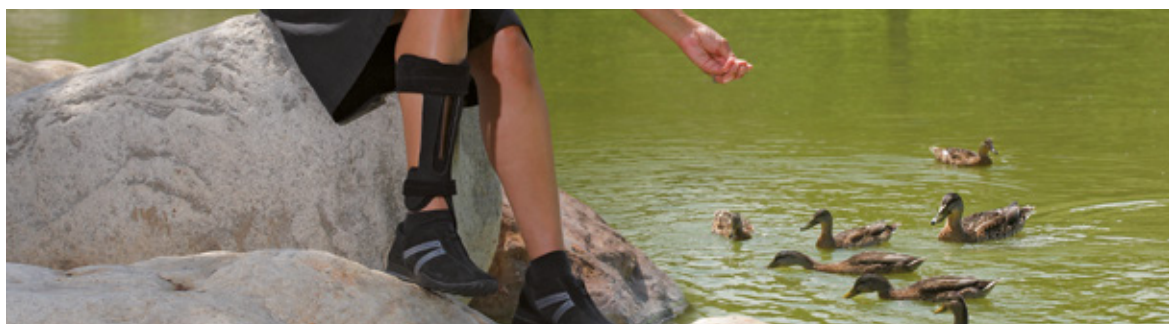

# AFO DYNAMIC

## ORDERING INFORMATION

### AFO DYNAMIC

		Shoe Size UK		Shoe Size EU			
Size	Calf Height	Male	Female	Male	Female	Left	Right
XS	30 - 34cm	2.5 - 4.5	< 4	34 - 37	< 37	O-102021	O-102011
S	32 - 36cm	5 - 7	4.5 - 6.5	37.5 - 40	37.5 - 40	O-102022	O-102012
M	34 - 38cm	7.5 - 10	7 - 9.5	40.5 - 44.5	40.5 - 44	O-102024	O-102014
L	36 - 40cm	10.5 - 13.5		45 - 48.5		O-102026	O-102016

## ACCESSORIES

### ACCESSORY KITS

Size	Part#
XS	O-102101
S	O-102102
M	O-102104
L	O-102106

Note: Accessory kits come with foam padding, upper and lower Hook & Loop straps, and Hook & Loop dots.

## AFO LIGHT

AFO Light provides support only where it is needed, eliminating unnecessary bulk and providing a cosmetically-appealing look. Using the best of Flex-Foot® technology and the energy storing properties of carbon fibre, AFO Light offers a strong and lightweight solution for people with mild to moderate drop foot.



### INDICATIONS FOR USE

- |                                   |                                      |                                   |
|-----------------------------------|--------------------------------------|-----------------------------------|
| Drop foot secondary to:           | • Charcot Marie Tooth disease (HMSN) | Contraindications:                |
| • Cerebro Vascular Accident (CVA) | • Lower extremity overuse injuries   | • Severe ankle - foot deformities |
| • Other neurological conditions   |                                      | • Fluctuating edema               |
| • Multiple Sclerosis              |                                      | • Severe spasticity               |
| • Mild knee instability           |                                      | • Neuropathy                      |

### FEATURES AND BENEFITS

- Energy return and propulsion assistance of the limb at toe off
- Supports the foot for ground clearance during swing phase
- Carbon fibre design provides excellent strength-to-weight ratio and dynamic motion
- Gradual plantar flexion at initial contact
- Stiffness gradient at heel and the toe correspond to ground reaction forces as they are applied to the foot
- Easy to don and doff
- Weight limit: 100kg

### REQUIRED MEASUREMENTS

- Shoe size
- Calf height at widest part of the calf

### ORDERING INFORMATION

#### AFO LIGHT

		Shoe Size UK		Shoe Size EU			
Size	Calf height	Male	Female	Male	Female	Left	Right
S	32 - 36cm	< 7	4.5 - 6.5	< 40	37.5 - 40	O-121022	O-121012
M	34 - 38cm	7.5 - 10	7 - 9.5	40.5 - 44.5	40.5 - 44	O-121024	O-121014
L	36 - 40cm	10.5 - 13.5		45 - 48.5		O-121026	O-121016
XL	38 - 42cm	11.5 - 14.5		46 - 50		O-121028	O-121018

### ACCESSORIES

Description	Part#
Refurb Kit	O-122101



## PROLITE AFO

Comfortable, low profile prefabricated leaf spring design to correct flaccid foot drop, especially on post-stroke patients. Injection moulded polypropylene allows for different thicknesses throughout the orthosis — thicker on the vertical aspect for rigidity, thinner footplate for ease of trimming, comfort and flexibility.

### INDICATIONS FOR USE

- Drop foot
- Mild drop foot secondary to other neurological pathologies

### FEATURES AND BENEFITS

- Injection-moulded for a high quality finish
- Easily trimmed footplate
- May be heat modified for minor fitting adjustments

### REQUIRED MEASUREMENTS

- Foot length



## ORDERING INFORMATION

Size	Foot length	Brace height	Left	Right
S	24cm	26.5cm	3500OP	3505OP
M	28cm	28.5cm	3501OP	3506OP
L	28cm	33cm	3502OP	3507OP
XL	30cm	39cm	3503OP	3508OP

## EXTRA STRONG PROLITE

### INDICATIONS FOR USE

- Drop foot
- Mild drop foot secondary to other neurological pathologies

### FEATURES AND BENEFITS

- Made from a specific acetal resin, polyoxymethylene, ensuring no moisture absorption, outstanding fatigue strength, impact resistance
- Increased rigidity in the sagittal plane
- May be heat modified for minor fitting adjustments

### REQUIRED MEASUREMENTS

- Foot length



## ORDERING INFORMATION

Size	Foot length	Brace height	Left	Right
S	24cm	26.5cm	3520BLU	3525BLU
M	28cm	28.5cm	3521BLU	3526BLU
L	28cm	33cm	3522BLU	3527BLU
XL	30cm	39cm	3523BLU	3528BLU

## REBOUND FOOT-UP®

Rebound Foot-Up is a lightweight ankle/foot orthosis (AFO) that offers dynamic support for drop-foot or related disorders that require dorsiflexion support. The Rebound Foot-Up supports the ankle dorsiflexion during swing phase. Dynamic support is provided by an elastic strap connected to the shoe by either a transparent plastic inlay placed between shoelaces, or by a nut and bolt fastened through the eyelets. The positioning of the fastening allows for individualising the support by providing the option of positioning the dorsiflexion support on the medial or lateral side. When using the plastic inlay, the dorsiflexion support becomes neutral.

The Rebound Foot-Up Ankle Cuff has a smart clamp that holds the cuff in place and facilitates one-handed donning and doffing.

Optional: Rebound Foot-Up Foot Wrap can be used when barefoot or in open shoes or sandals. The adjustable elastic strap is attached to the Foot Wrap using a nut and bolt fastened through the eyelets.



### INDICATIONS FOR USE

- Drop-foot
- Disorders that require dorsiflexion support

### FEATURES AND BENEFITS

- Dynamic and discreet support for drop-foot or related disorders that require dorsiflexion support
- Improved function, ease of use, comfort, and quality
- Allows for voluntary plantar flexion
- The elastic strap is adjustable and easily replaceable
- The Rebound Foot-Up Ankle Cuff and Rebound Foot-Up Foot Wrap are made from breathable material that can be worn comfortably for long periods of time
- The Rebound Foot-Up is hand-washable
- Positioned medial or lateral support
- Contoured to the shape of leg
- Adjustable strap length
- Smart clamp facilitates one-handed donning / doffing
- Optional Rebound Foot-Up Foot Wrap can be used without shoes or in open shoes or sandals

### REQUIRED MEASUREMENTS

- Circumference of ankle, measured around the malleoli



### ORDERING INFORMATION

#### ANKLE CUFF

Description	Size	Ankle Circumference	Part #
Rebound Foot-Up Ankle Cuff	S/M	18 - 25cm	RFU50001
Rebound Foot-Up Ankle Cuff	L/LX	26 - 33cm	RFU50002

#### FOOT WRAP

Description	Size	Foot Circumference	Part #
Rebound Foot-Up Foot Wrap	One Size	20 - 28cm	RFU50101

### ACCESSORIES

Description	Part #
Rebound Foot-Up Attachment Kit (includes plastic inlay, elastic strap, and 1 x nut and screw)	RFU50201
Rebound Foot-Up Replacement Nuts And Bolts (5 nuts included: 2 short and 3 long screws)	RFU50202

Description	Size	Foot Circumference	Part #
Rebound Foot-Up Bundle	S/M	18 - 25cm	RFU500011
Rebound Foot-Up Bundle	L/LX	26 - 33cm	RFU500021

## FOOT-UP®

Foot-Up is a lightweight AFO that offers dynamic support for drop foot or similar complaints. The orthosis provides visible improvement in gait by supporting the foot the moment it is raised.



### INDICATIONS FOR USE

- Drop foot
- Support of dorsiflexion

### FEATURES AND BENEFITS

- Streamlined, lightweight and invisible when worn
- Breathable material, padded and comfortable to wear
- Easy to fit
- Available in black and beige
- Attached to the shoe, or available as shoeless model

### REQUIRED MEASUREMENTS

- Basic version: Circumference of ankle, measured around the malleoli
- Shoeless version: Circumference of foot immediately behind the ball of the foot

### ORDERING INFORMATION

#### FOOT-UP

Size	Ankle Circumference	Beige	Black
M	18 - 21cm	07811-1	07810-1
L	22 - 26cm	07811-2	07810-2
XL	27 - 33cm	07811-3	07810-3

#### SHOELESS VERSION\*

Size	Foot Circumference	Beige	Black
S	< 20cm	27811-1	27810-1
M	20 - 24cm	27811-2	27810-2
L	24 - 28cm	27811-3	27810-3
XL	> 28cm	27811-4	27810-4

\*Shoeless version item comprises of foot wrap only and must be ordered with the Foot-Up unless the patient already has one.

### ACCESSORIES

Description	Beige	Black
Foot-Up plastic insert	17811	17810

Description	Beige	Black
Foot-Up with Shoeless add-on Bundle	07811xy	07810xy

x = size of main foot-up part

y = size of shoeless add-on

e.g. 0781012 = size 1 Foot-up with a size 2 Shoeless add-on in Black

## FORMFIT® PRO ANKLE

Össur Formfit® Pro Ankle is a compressive ankle sleeve with 2 integrated malleoli pads, designed to be worn after mild ankle injuries. The dynamic 3-D MotionTech™ knit is supportive and meets certified medical grade compression requirements. Proven superior to leading brands for moisture wicking and breathability, your ankle stays cool during wear. CoolVent™ technology in the area in front of the ankle further enhances breathability and comfort.



MotionTech™



CoolVent™



SmartPad™



FoldLock™



### INDICATIONS FOR USE

- For ankle conditions that may benefit from compression, such as:
- Ankle injury rehabilitation
- Mild sprains and strains
- Mild instability


### CONTRAINDICATIONS

- Must not be used by individuals for whom compression is contraindicated.

### FEATURES AND BENEFITS

- MotionTech™ for compression and dynamic fit with superior moisture wicking and breathability, keeping your ankle cool during wear
- Certified medical grade compression (20-30 mmHg)
- CoolVent™ technology in the area in front of the ankle for added breathability and comfort
- 2 integrated malleoli SmartPads™ for targeted support and cushioning
- Patented FoldLock™ system for easier and more secure application

### REQUIRED MEASUREMENTS

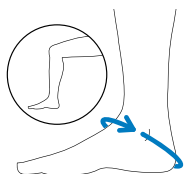

Measure ankle circumference diagonally over the base of the heel and across the ankle bone. Ankle in 90 degrees.

Size	Ankle circumference
XS	18 - 20cm
S	20 - 23cm
M	23 - 26cm
L	26 - 29cm
XL	29 - 32cm

# FORMFIT® PRO ANKLE

## ORDERING INFORMATION

### BLUE

Size	Left	Right	Size	Left	Right
XS	FP201201	FP201101	XS	FP200201	FP200101
S	FP201202	FP201102	S	FP200202	FP200102
M	FP201203	FP201103	M	FP200203	FP200103
L	FP201204	FP201104	L	FP200204	FP200104
XL	FP201205	FP201105	XL	FP200205	FP200105

### BLACK



## FORMFIT® ANKLE BRACE

The Formfit Ankle brace provides stability and protection for the ankle – whether used prophylactically, for chronic instability or following an injury. A hybrid between a rigid ankle stirrup and a soft ankle support, the Formfit Ankle ensures a close fit which limits inversion, eversion, flexion and extension movements. The low profile design fits easily into shoes and the quick lace up feature makes it easy to don and doff. Breathable material ensures it can be comfortably worn all day. The Formfit Ankle with Figure 8 strap version, offers an even greater level of support, by further limiting the range of motion of the ankle.

### INDICATIONS FOR USE


- Ankle injury rehabilitation/immobilisation (Grade I/II sprains)
- Support chronic instability
- Prophylactic use

### FEATURES AND BENEFITS

- Breathable material
- Quick lace Up
- Inversion/Eversion Control
- Figure 8 strap version

### REQUIRED MEASUREMENTS

- Measure ankle circumference diagonally over the base of the heel and across the ankle bone.

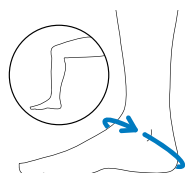

## ORDERING INFORMATION

Size	Ankle Circ.	Form Fit Ankle Brace	Form Fit Ankle with Figure 8
XS	28 - 30.5cm	B-212010001	B-212000001
S	30.5 - 33cm	B-212010002	B-212000002
M	33 - 35.5cm	B-212010003	B-212000003
L	35.5 - 38cm	B-212010004	B-212000004
XL	38 - 40.5cm	B-212010005	B-212000005

## FORMFIT® ANKLE WITH SPEEDLACE

### INDICATIONS FOR USE

- Ankle injury rehabilitation/immobilisation (Grade I/II sprains)
- Support chronic instability
- Prophylactic use


### FEATURES AND BENEFITS

- Durable materials
- Quick-Pull closure straps
- Figure 8 straps
- Removable stays
- Circumferential cuff



### REQUIRED MEASUREMENTS

- Measure ankle circumference diagonally over the base of the heel and across the ankle bone. Ankle in 90 degrees.

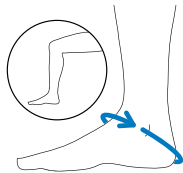

### ORDERING INFORMATION

Size	Ankle Circ.	Part#
XXS	23 - 25.5cm	W-10621
XS	25.5 - 28cm	W-10622
S	28 - 30.5	W-10623
M	30.5 - 33cm	W-10625
L	33 - 35.5cm	W-10627
XL	35.5 - 38cm	W-10628
XXL	38 - 40.5cm	W-10629

## FORMFIT® ANKLE STIRRUPS

Formfit Ankle Stirrups are lightweight and sturdy. Whether worn to accommodate an acute injury, chronic instability, or post-cast healing, Formfit Ankle Stirrups stabilise injured ankles by restricting the inversion/eversion motion. The universal design has been carefully engineered to accommodate the malleoli on both sides and ensures a great, comfortable fit. The slip resistant heel strap can be adjusted to accommodate most ankles. Our signature Flex-Edge™ overmould surrounds the stirrup's outer shell protecting patient's skin from any hard edges, further adding to a comfortable fit. The universal stirrup design is available with three different liner options: inflatable air, gel, or foam, offering each patient a solution that best fits their needs.

### INDICATIONS FOR USE

Situations requiring inversion/eversion restriction. These may include:

- Mild to severe ankle sprains
- Chronic ankle instability
- Post-cast and post-walker support
- Ankle injury rehabilitation and/or post-surgical use

### FEATURES AND BENEFITS

- Universal shell, carefully crafted to fit both right and left ankles
- Soft Flex-Edge overmould for increased comfort
- Adjustable heel strap to optimise patient fit and comfort
- Slip-resistant silicone dots to reduce slipping and stabilise the foot inside of shoe
- 2 bidirectional hook and loop straps ensure a secure and stable fit
- Choice of inflatable, foam, and gel liners to suit your needs
- Available in Adult and Youth sizes

### REQUIRED MEASUREMENTS

- Patient's height



## ORDERING INFORMATION



Size	Patient's Height	Formfit Air Ankle Stirrup	Formfit Gel Ankle Stirrup	Formfit Foam Ankle Stirrup
Adult	Above 160 cm	70350	70351	70352
Youth	Below 160 cm	71350	71351	71352



## EXOFORM® DORSAL NIGHT SPLINT

### INDICATIONS FOR USE

- Plantar fasciitis
- Post-static pain
- General heel pain
- Drop foot
- Achilles tendonitis
- Lower extremity overuse injuries

### FEATURES AND BENEFITS

- Adjustable aluminum stay inside dorsal shell allows modification in the angle of dorsiflexion
- Soft, overmolded edges enhance comfort
- Hinged shell at the calf area prevents brace rotation
- Semi-rigid plantar pad reduces discomfort along the sides of the foot while providing an improved angle of dorsiflexion
- Non-slip surface on plantar pad prevents slipping during normal walking
- Circumferential heel and posterior strapping prevents migration while increasing overall comfort and effectiveness
- Strapping and shell perforations improve comfort and breathability



### REQUIRED MEASUREMENTS

- Shoe size

### ORDERING INFORMATION

Size	Shoe Size UK	Shoe Size EU	Part#
M	4.5 - 8	37.5 - 42.5	W-50085
L	8.5 - 13.5	43 - 48.5	W-50087

## FORMFIT® NIGHT SPLINT

The Formfit Night Splint, designed to help relieve the pain associated with plantar fasciitis by providing a gentle overnight stretch. Straps can be adjusted from 0° up to 10° degrees as needed. The soft breathable liner and padded straps provide comfort and help keep patient cool. Straps adjust from 0°-10° dorsiflexion

### INDICATIONS FOR USE

Conditions that may benefit from resting with the ankle in a neutral position, such as:

- Plantar fasciitis
- Achilles tendinopathy

### FEATURES AND BENEFITS

- Straps adjust from 0°-10° of dorsiflexion
- Contoured for a comfortable fit
- Soft foam interior is washable and breathable quick release buckles for easy application
- Slip resistant tread

### REQUIRED MEASUREMENTS

- Shoe size



## ORDERING INFORMATION

	Shoe Size UK		Shoe Size EU		Standard
Size	Male	Female	Male	Female	with tread
S	Up to 6.5	Up to 5	Up to 39	Up to 38	50023
M	7 - 9.5	5.5 - 8	39.5 - 44	38.5 - 43	50025
L	10 and up	8.5 and up	44.5 and up	43.5 and up	50027

## ACCESSORIES

Description	Part#
Foam Wedge	10700
Replacement Liner S	11760
Replacement Liner M	11761
Replacement Liner L	11762

## DYNAPRO® AFO

The DynaPro AFO is indicated for the treatment of plantarflexion, inversion, eversion contractures whilst protecting the heel from unwanted pressure while in bed. The semi-rigid insert can be positioned to accommodate the angle of the ankle/ foot combines with a removable pull strap to initiate a therapeutic dorsiflexion stretch to help combat mild plantarflexion contracture with/ or without inversion/ eversion. The cover has a non-skid surface sewn into the bottom to improve grip during transfers. Intended for recumbent use the DynaPro AFO has a toe post to provide protection from shear forces during in bed usage.

### INDICATIONS FOR USE

- For the treatment of plantarflexion contractures of the ankle and/or to protect the heel from unwanted pressure while in bed

### FEATURES AND BENEFITS

- Semi-rigid frame covered with ultra-soft, washable, fleece-material
- Positionable semi-rigid insert to accomodate angle of the ankle/foot
- Toe post protection
- Rotation control bar
- Removable, pull straps to initiate theraputic dorsi-stretch to help reverse a mild plantar-flexion ankle/foot contracture
- Non-skid surface sewn into bottom for safer transfers

### REQUIRED MEASUREMENTS

- Calf circumference
- Foot length



## ORDERING INFORMATION

Part#	Description	Lenght of Foot	Max Calf Circumference
2DPAFO21S	DynaPro AFO Small	15 - 20cm	38cm
DPAFO22M	DynaPro AFO Medium	20 - 25cm	42cm
DPAFO23L	DynaPro AFO Large	25 - 30cm	48cm
DPAFO24C	DynaPro AFO Child	9 - 13cm	30cm
DPAFO25Y	DynaPro AFO Youth	13 - 15cm	36cm

## SOFTPRO® GAIT TRAINER AFO

The SoftPro® Gait Trainer AFO is an off the shelf boot, which effectively treats non fixed plantar flexion contractures, whilst protecting the heel from unwanted pressure while in bed. The semi-rigid insert can be positioned to accommodate the angle of the ankle/ foot combines with a removable pull strap to initiate a therapeutic dorsiflexion stretch to help combat mild plantarflexion contracture. The SoftPro Gait Trainer AGO has a removeable pad attached to the bottom of for limited and assisted weight bearing, transfer and gait training in the device.

### INDICATIONS FOR USE

- For the treatment of plantarflexion contractures of the ankle and/or to protect the heel from unwanted pressure while in bed

### FEATURES AND BENEFITS

- Semi-rigid frame covered with ultra-soft, washable, fleece-material
- Positionable semi-rigid insert to accommodate angle of the ankle/foot
- Toe post protection
- Rotation control bar
- Removable transfer pad that facilitates initial weight bearing and assisted gait training
- Adjustable dorsi-strap that provides (optional) enhanced dorsiflexion

### REQUIRED MEASUREMENTS

- Calf circumference
- Foot length



## ORDERING INFORMATION

Part#	Description	Length of Foot	Max Calf Circumference
SP30-00	SoftPro Gait Trainer SM Fleece	15 - 20cm	38cm
SP30-01	SoftPro Gait Trainer MD Fleece	20 - 25cm	43cm
SP30-02	SoftPro Gait Trainer LG Fleece	25 - 30cm	48cm
SP30-03	SoftPro Gait Trainer XL Fleece	28 - 30cm	58cm

## SOFTPRO® IN-BED AFO

The SoftPro In-Bed AFO is indicated for the treatment of plantarflexion contractures whilst protecting the heel from unwanted pressure while in bed. The semi-rigid insert can be positioned to accommodate the angle of the ankle/ foot. The cover has a non-skid surface sewn into the bottom to improve grip during transfers. Intended for recumbent use the SoftPro In-Bed AFO has a toe post to provide protection from shear forces during in-bed usage.

### INDICATIONS FOR USE

- For the treatment of plantarflexion contractures of the ankle and/or to protect the heel from unwanted pressure while in bed

### FEATURES AND BENEFITS

- Semi-rigid frame covered with ultra-soft, washable, fleece-material
- Positionable semi-rigid insert to accommodate angle of the ankle/foot
- Toe post protection
- Rotation control bar
- Non-skid bottom for assisted transfers

### REQUIRED MEASUREMENTS

- Calf circumference
- Foot length



## ORDERING INFORMATION

Part#	Description	Length of Foot	Max Calf Circumference
SP10-00	SoftPro In-Bed SM Fleece	15 - 20cm	38cm
SP10-01	SoftPro In-Bed MD Fleece	20 - 25cm	43cm
SP10-02	SoftPro In-Bed LG Fleece	25 - 30cm	48cm
SP10-03	SoftPro In-Bed XL Fleece	28 - 30cm	58cm
SP10-04	SoftPro In-Bed Child Fleece	9 - 13cm	30cm
SP10-05	SoftPro In-Bed Youth Fleece	13 - 15cm	36cm
SP10-07	SoftPro In-Bed Bariatric Fleece	30 - 33cm	74cm

## DH OFFLOADING POST-OP STYLE SHOE

Designed for use in the treatment of plantar wounds, this shoe can be an excellent alternative to post-op style shoes with static pads (wool, felt, foam, etc.). It accommodates the ambulatory and compliant patient for short-term treatment (1 to 4 months).

### INDICATIONS FOR USE

- Plantar wounds
- Post-operative use
- Plantar foot ulcers

### FEATURES AND BENEFITS

- Designed to accommodate either left or right foot
- Designed around Össur's unique Active Offloading Insole
- Designed to accommodate bulky dressings
- Super soft nylon upper with reinforced heel counter provides extra comfort and stability
- Patented removable toe pieces aides in wound accessibility for daily wound site inspection



### REQUIRED MEASUREMENTS

- Shoe size

## ORDERING INFORMATION

	Shoe Size UK		Shoe Size EU		
Size	Male	Female	Male	Female	Part#
S	3.5 - 5.5	4 - 6	35.5 - 38	37 - 39	10340
M	6 - 8	6.5 - 8.5	38.5 - 42	39.5 - 43.5	10341
L	8.5 - 10.5	9 - 10	42.5 - 45	44 - 45	10342
XL	11 - 13.5	10.5+	45.5 - 48.5	45.5-48.5	10343

## ACCESSORIES

### REPLACEMENT PARTS

Size	Insoles
S	12365
M	12366
L	12367
XL	12368

## MESH TOP POST-OP SHOE

### INDICATIONS FOR USE

- For recovery after soft tissue procedures and post trauma

### FEATURES AND BENEFITS

- Breathable mesh upper with reinforced heel counter and padded heel collar provides increased stability overall to the rear of the foot while reducing slippage
- Semi-rigid sole provides superior protection and control after osteotomies
- Extended strap and dual side closure system accommodates extra bulky dressings

### REQUIRED MEASUREMENTS

- Shoe size



## ORDERING INFORMATION

	Shoe Size UK		Shoe Size EU			
Size	Male	Female	Male	Female	Male	Female
S	3.5 - 5.5	2.5 - 3.5	35.5 - 38	35 - 36	17013	17003
M	6 - 8	4 - 5.5	38.5 - 42	36.5 - 38.5	17015	17005
L	8.5 - 10.5	6 +	42.5 - 45	39 +	17017	17007
XL	11 - 13.5		45.5 - 48.5		17018	

## SOFT TOP POST-OP SHOE

### INDICATIONS FOR USE

- Post-operative use or minor trauma wounds
- Geriatric or insensate foot

### FEATURES AND BENEFITS

- The cotton-poly blend upper allows the patient's foot to breathe and provides added comfort
- Designed specifically for recovery after soft tissue procedures post trauma
- The contact closure system provides even torsion and is designed for ease of use

### REQUIRED MEASUREMENTS

- Shoe size



## ORDERING INFORMATION

	Shoe Size UK		Shoe Size EU			
Size	Male	Female	Male	Female	Male	Female
S	3.5 - 5.5	2.5 - 3.5	35.5 - 38	35 - 36	18013	18003
M	6 - 8	4 - 5.5	38.5 - 42	36.5 - 38.5	18015	18005
L	8.5 - 10.5	6 +	42.5 - 45	39 +	18017	18007
XL	11 - 13.5		45.5 - 48.5		18018	

# CANVAS ROCKER BOTTOM CAST SHOE

## INDICATIONS FOR USE

- For use with a short leg cast

## FEATURES AND BENEFITS

- Cushioned rocker bottom cast sandal
- Contact closure, open toe

## REQUIRED MEASUREMENTS

- Insole length



## ORDERING INFORMATION

Size	Insole length	Blue	Black
XS	18cm	308XS	
S	19 - 23cm	308SM	B308SM
S/M	23 - 25cm	308SM/MED	B308S/M
M	25.5 - 26.5cm	308MED	B308MED
M/L	26.5 - 28cm	308MLG	B308MLG
L	28 - 30.5cm	308LG	B308LG
XL	30.5 - 35cm	308XLG	B308XLG







## TABLE OF CONTENTS

---

CASTING & TRACTION .....	155
TECHFORM® PREMIUM .....	157

## TECHFORM® PREMIUM

Techform Casting Tape was designed by and for orthopaedic casting professionals who look for a superior handling experience and a smooth, strong finished cast. It incorporates a patented rub-responsive formula that works with our unique fabric to produce the finished cast you'll ever see.

### INDICATIONS FOR USE

- Immobilisation

### FEATURES AND BENEFITS

- Glides off the roll for easier application
- Soft, conformable rolls allow for easy handling
- "Rub-responsive" resin formula adapts to the technician's technique - the more it is rubbed, the smoother the cast gets
- Excellent lamination ensures a strong and durable cast
- Smoother finished cast is non-abrasive and will not snag on clothing
- Precision welded ends do not fray and lay down easily
- Finger holes in the pouch allow for easy opening



### ORDERING INFORMATION

WIDTH	WH	BK	OR	RD	PK	HP	LB	DB	PR	GN	RB
1.5"	154WH										
2"	204WH	204BK	204OR	204RD	204PK	204HP	204LB	204DB	204PR	204GN	204RB
3"	304WH	304BK	304OR	304RD	304PK	304HP	304LB	304DB	304PR	304GN	304RB
4"	404WH	404BK	404OR	404RD	404PK	404HP	404LB	404DB	404PR	404GN	404RB
5"	504WH										

Note: Packed in boxes of 10

## NOTES

---





FOLLOW ÖSSUR ON



TEL 03450 065 065  
ukinfo@ossur.com

WWW.OSSUR.COM

