Instructions for Use

4-HOLE ADAPTERS

EN		Instructions for Use	3
DE		Gebrauchsanweisung	8
FR		Notice d'utilisation	14
ES		Instrucciones para el uso	20
IT		Istruzioni per l'uso	25
NO		Bruksanvisning	31
DA		Brugsanvisning	36
SV		Bruksanvisning	41
EL		Οδηγίες Χρήσης	46
FI		Käyttöohjeet	51
NL		Gebruiksaanwijzing	56
PT		Instruções de Utilização	62
PL		Instrukcja użytkowania	67
TR		Kullanım Talimatları	73
RU		Инструкция по использованию	78
JA		取扱説明書	84
ZH		中文说明书	89
KO		사용 설명서	94

Medical Device

INTENDED USE

The device is intended as a connection component of a prosthetic system that replaces a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

For more detailed device information, see **Table 1**.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

For weight limit and impact level, see **Table 2**.

GENERAL SAFETY INSTRUCTIONS

Warning: Use of a lower limb prosthetic device carries an inherent risk of falling which may lead to injury.

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

Warning: Only use Össur replacement parts for the device for ensured durability.

The device is for single patient use.

ASSEMBLY INSTRUCTIONS

Warning: When extreme alignment is necessary, use extended set screws to achieve secure fastening.

Warning: For secure fastening use medium strength threadlocker and apply rated torque twice, see **Table 1**.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

See which environmental conditions apply to the device in **Table 1**.

A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e.g., rain), no submersion is allowed. No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

A Waterproof device can be used in a wet or humid environment and submerged in up to 3-meter-deep water for a maximum of 1 hour.

It can tolerate contact with: Salt water, chlorinated water, perspiration, urine, and mild soaps.

It can also tolerate occasional exposure to sand, dust, and dirt.

Continuous exposure is not allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water after exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

SPECIFICATIONS

Table 1 – Device Specifications

Part #	Description	Material	Torque Values		Environmental Conditions
			Mounting Screws	Set Screws	
AJ-233100	4-Hole Male Pyramid Adapter Junior	Aluminum, Stainless Steel	6 Nm	None	Weatherproof
AJ-235100		Titanium	6 Nm	None	Waterproof
AJ-242100	4-Hole Female Pyramid Adapter Junior	Aluminum	6 Nm	8 Nm	Weatherproof
AJ-245100		Titanium	6 Nm	8 Nm	Waterproof
A-233100	4-Hole Male Pyramid Adapter AL	Aluminum, Stainless Steel	10 Nm	None	Weatherproof
A-242100	4-Hole Female Pyramid Adapter AL	Aluminum	10 Nm	15 Nm	
A-242500	4-Hole Female Pyramid Adapter 20/20 Offset AL		10 Nm	15 Nm	
A-242510			10 Nm	15 Nm	
A-242520			10 Nm	15 Nm	
A-235100	4-Hole Male Pyramid Adapter TI	Titanium	10 Nm	None	Waterproof
A-235400	4-Hole Male Pyramid Adapter 8mm Offset TI		10 Nm	None	
A-245100	4-Hole Female Pyramid Adapter TI		10 Nm	15 Nm	

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur prosthetic devices are designed and verified to be safe and compatible in combination with each other and custom-made prosthetic sockets with Össur adapters, and when used in accordance with their intended use.

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

Table 2 – Usage Specifications

Part #	Impact Level	Weight Limit	ISO 10328 – label text
AJ-233100	The device is intended for children and adolescent patients.	45 kg / 100 lbs	ISO10328 - P3 - 45 kg
AJ-242100		60 kg / 132 lbs	ISO10328 - P4 - 60 kg
AJ-235100			
AJ-245100			
A-233100	The device is for low to moderate impact use, e.g., walking.	100 kg / 220 lbs	ISO10328 - P5 - 100 kg
A-242100		125 kg / 275 lbs	ISO10328 - P6 - 125 kg
A-242500			
A-242510			
A-242520			
A-235100	The device is for low to high impact use, e.g., walking and occasional running.	166 kg / 366 lbs	ISO 10328 - P7 - 166 kg
A-235400			
A-245100			

ISO 10328 - “P” - “m”kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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