- Tighten the strap until snug. Secure the contact closure to a comfortable fit. Adjustments may be necessary as forearm swelling patterns change.

WASHING INSTRUCTIONS

- This product can be cleaned using a mild detergent and cold water. Do not use bleach or strong detergent. Do not wash in washing machine.
- Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not place in dryer or near heater.

REPORT A SERIOUS INCIDENT

Important notice to users and/or patients established in Europe:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY TEXT

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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