ENGLISH

Medical Device

INTENDED USE

The device is intended to hold the arm in an elevated fixed position.

The device must be fitted and adjusted by a healthcare professional.

Indications for use

For arm conditions that may benefit from supporting the arm, such as:

- Post-trauma and pre or post-operative
- To support a cast or a splint

No known contraindications.

Warnings and Cautions:

Inspect the skin under the device's contact area before and after each use.

GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

FITTING INSTRUCTIONS

Device Application

1. Place the forearm into the device at a 90° angle, seating the elbow snugly in the corner pocket of the sling. Ensure that the open end of the device supports the hand and fingers.
2. Place shoulder strap behind the neck and over the opposite shoulder. Adjust the strap length and strap position until a comfortable fit and proper support are achieved.

Note: Adjustments may be necessary as swelling patterns change.

Device Removal

Remove the shoulder strap from behind the neck and take the forearm out of the device.

USAGE

Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry.
- The device should not be exposed to excessive heat or sunlight.

Note: Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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