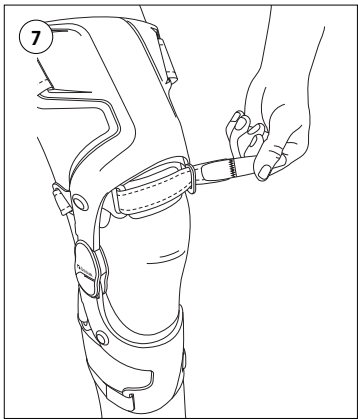
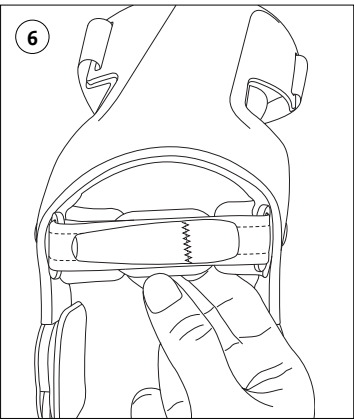
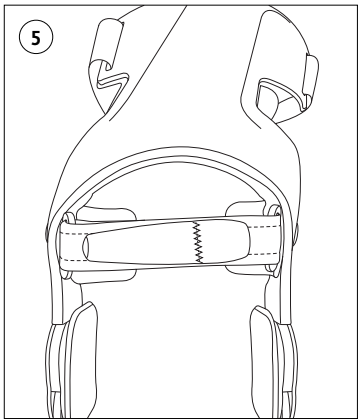
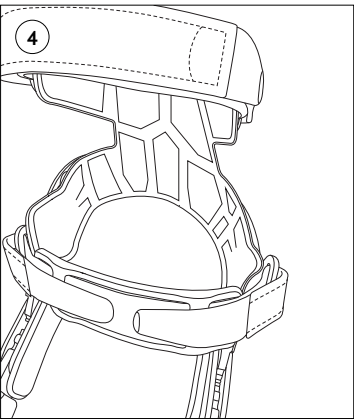
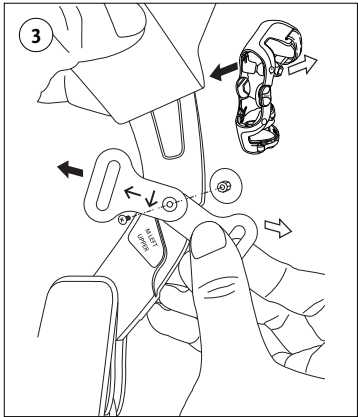
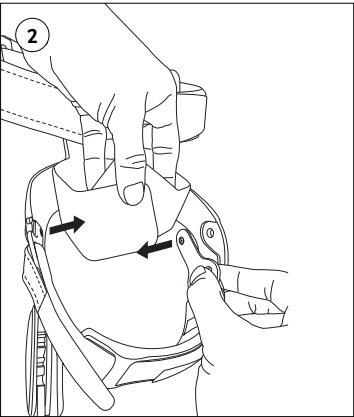
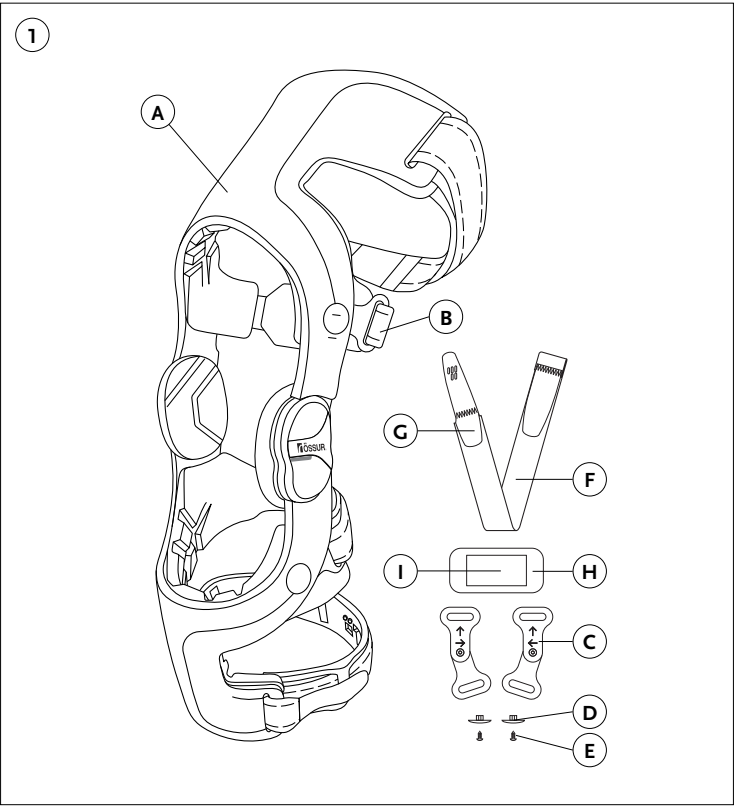
ENGLISH

MD Medical Device

DESCRIPTION

These Instructions for Use (IFU) provide additional information that enables the usage of an accessory with a particular medical device. Refer to the applicable IFU for the usage of the medical device.

INTENDED USE

The device is intended to enhance support/stability around the knee.

FITTING INSTRUCTIONS

While carrying out the following instructions, please refer to the overview figure for locating components mentioned in the text (**Fig. 1**).

- Lift the Upper Frame (A) liner and remove both D-Rings and Upper Cruciate Strap and Strap Pad (B) located just above the knee (**Fig. 2**).
- Install the PCL D-Rings (C) using the provided Nuts (D) and Screws (E), ensuring that the directional arrows face “down” and “forward” (**Fig. 3**).
Note: Make sure not to over-tighten the screws.
- Re-install the Upper Cruciate Strap and Strap Pad that was removed in Step 1 (**Fig. 4**).
- Install the PCL Strap (F) into the anterior slots on the PCL D-Rings and attach it using the Alligator Tab (G) (**Fig. 5**).
Note: Trim the PCL Strap to length as needed.
- Attach the PCL Strap Pad (H) to the PCL Strap using the Back-to-Back Hook Fastener (I) (**Fig. 6**).
- With the leg in full extension. Tighten the PCL Strap so that the PCL Strap and Pad are comfortably snug (**Fig. 7**). Reposition the Upper Cruciate Strap Pad, if needed.

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