# ENGLISH

## SYMBOLS

**MD** Medical Device

## INTENDED USE

The device is intended for weight bearing while wearing a cast

### *Indications for use*

For use with a short leg cast.

No known contraindications.

### **Warnings and Cautions:**

Patients should inspect their cast, foot and the device before and after each use.

## GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

## USAGE

### *Device Application*

1. Unfasten all contact closures.
2. Insert casted foot into the device making sure cast is positioned all the way back to the heel.
3. Fasten all contact closure straps snugly, starting with the heel strap.
4. Straps may be adjusted and/or cut as desired.

### *Cleaning and care*

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry, do not expose to direct heat.

**Note:** Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.

**Note:** Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

## DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

## LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.

- Device used outside of recommended use condition, application, or environment.

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©Copyright Össur 2021