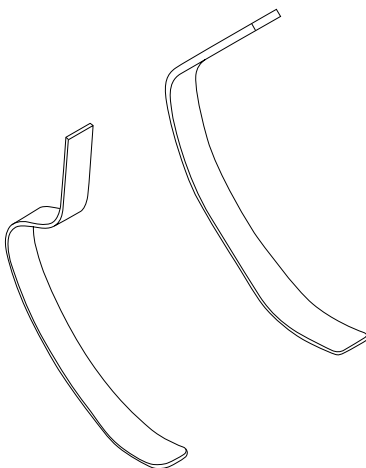
Instructions for Use

Flex-Foot Cheetah®

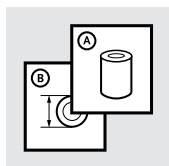
Cheetah® Xcel

Cheetah® Xtend






Cheetah® Xtreme

Cheetah® Xpanse

Cheetah® Junior

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	14
ES	Instrucciones para el uso	19
IT	Istruzioni per l'uso	24
NO	Bruksanvisning	29
DA	Brugsanvisning	34
SV	Bruksanvisning	39
EL	Οδηγίες Χρήσης	44
FI	Käyttöohjeet	49
NL	Gebruiksaanwijzing	54
PT	Instruções de Utilização	59
PL	Instrukcja użytkowania	64
TR	Kullanım Talimatları	69
RU	Инструкция по использованию	74
JA	取扱説明書	79
ZH	中文说明书	84
KO	사용 설명서	89
AR	تعليمات الاستخدام	94

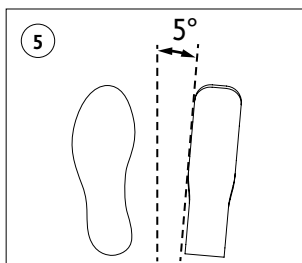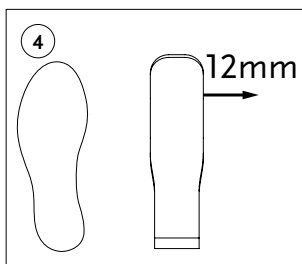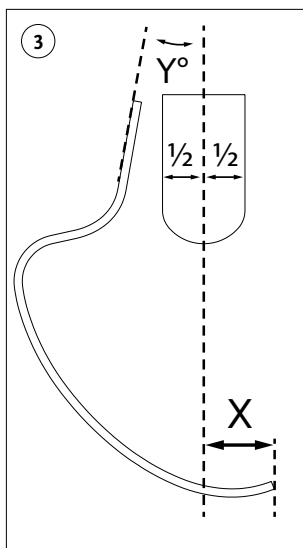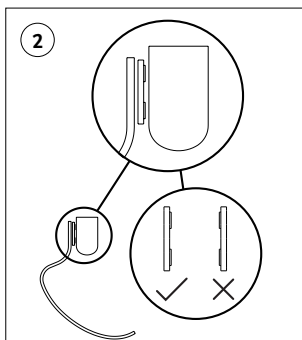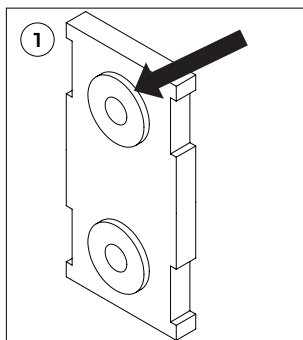DESCRIPTION

The device is a prosthetic foot for sports activities.

INTENDED USE

The device is intended as a part of a prosthetic system that replaces the foot and ankle function of a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

Cheetah Junior

- The device is intended for children and adolescent patients.

All other devices:

- The device is for high to extreme impact use, e.g., fast walking, running, and sports.
- Cheetah Xpanse is also for long jump.

GENERAL SAFETY INSTRUCTIONS

Warning: Use of a lower limb prosthetic device carries an inherent risk of falling which may lead to injury.

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

The device is for single patient use.

REQUIRED COMPONENTS

The foot can be attached to the socket with direct lamination or by using one of the following options:

Lamination Connector Kit - see assembly instructions below.

Alignment Adapters - refer to Instructions for relevant alignment adapter.

DEVICE SELECTION

Verify that selected variant of the device is suitable for the impact level and weight limit according to the following table.

Warning: Do not exceed weight limit. Risk of device failure.

Incorrect category selection may also result in poor device function.

Cheetah Junior

Weight kg	15-24	25-34	35-45	46-55
Weight lbs	33-53	54-75	76-99	100-121
Category	1	2	3	4

All other devices:

Weight kg	45-52	53-59	60-68	69-77	78-88	89-100	101-116	117-130	131-147
Weight lbs	99-115	116-130	131-150	151-170	171-194	195-220	221-256	257-287	288-325
Category	1	2	3	4	5	6	7	8	9

ASSEMBLY INSTRUCTIONS

Caution: Epoxy can cause allergic reactions when in direct contact with body.

Caution: Sharp edges or splinters can cause a cutting hazard.

Nike Soles

The device can be fitted with a Nike spike pad or Nike sole. Refer to instructions for relevant sole/pad for assembly instructions.

Lamination Connector

The Lamination Connector must be laminated directly into the posterior wall of the socket.

Warning: Before starting make sure to orient the Lamination Connector the correct way. The shoulders of the inserts (marked with arrow) should be oriented towards the socket (**Fig. 1&2**).

1. Put the Lamination Connector in the middle third of the socket to make height adjustment easier.
2. Make sure the socket surface is flat. A flat surface below the Lamination Connector distributes the load to the connected foot.
3. Use a friction pad between the Lamination Connector and the foot.
4. Apply thread locker and torque bolts as shown in **Table 3**.

Kit	Device	Weight Limit	Bolt Size	Torque Values
Lamination connector kit (FSX50009)	Flex-Foot Cheetah Cheetah Xtend Cheetah Xtreme Cheetah Xcel Cheetah Xpanse	147 kg	3/8"	45 Nm
Lamination connector kit, pediatric (CHX50001)	Cheetah Junior	55 kg	5/16"	28 Nm

ALIGNMENT INSTRUCTIONS

Bench Alignment (Fig. 3-5)

Sagittal plane

1. Use an alignment jig to determine the vertical socket position.
2. Make sure the load line bisects the socket.
3. Make sure:
 - that the load line falls through the ground contact point
 - or that the load line is posterior to the toe end per dimension X below (**Fig. 3**).
4. Clamp the foot into the correct flexion position Y (**Fig. 3**).

Coronal Plane

1. Set the applicable socket abduction/adduction.
2. Offset the foot module 12 mm away from centerline (**Fig. 4**). Cheetah Junior: No offset.
3. Set up for 5° of external rotation (**Fig. 5**).

4. Set the correct height - see compression in table below to compensate for vertical compression.

Device	Dimension X [mm]	Flexion Angle Y [°]	Compression [mm]
Flex-Foot Cheetah	50	7	25
Cheetah Xtend	63	7	38
Cheetah Xtreme	63	7	38
Cheetah Xcel Large	110	9	38
Cheetah Xcel Small	100	7	38
Cheetah Xpanse Large	110	48	38
Cheetah Xpanse Small	110	54	38
Cheetah Junior	35	7	15

Dynamic Alignment

Physical ability and the length of residual limb will have an effect on alignment. Optimal alignment reduces energy expenditure and reduces muscle and ligament strain.

Adjust alignment to improve gait symmetry.

The A-P alignment and vertical deflection from socket to foot can have an effect on stride length.

Cutting the foot

Make sure the user has sufficient clearance during knee flexion.

Use a hack saw or band saw to trim the foot.

Note: The foot must be flush with or extend above the lamination connector.

Troubleshooting Guide

Make sure socket fit and suspension are optimal. Examine running symmetry at patient's preferred running speed.

If the duration of loading the prosthesis is longer compared to the contralateral side:

- The foot module is too soft
- The prosthesis is too short
- The foot is aligned too far anteriorly

If the duration of loading the prosthesis is shorter compared to the contralateral side:

- The foot module is too stiff
- The prosthesis is too long
- The foot is aligned too far posteriorly

Note: If you move the A-P position of the foot, there will be an effect on hip flexion, stride length and trunk motion.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

The device is Waterproof.

A Waterproof device can be used in a wet or humid environment and submerged in up to 3-meter-deep water for a maximum of 1 hour.

It can tolerate contact with: Salt water, chlorinated water, perspiration, urine, and mild soaps.
It can also tolerate occasional exposure to sand, dust, and dirt.
Continuous exposure is not allowed.
Dry with a cloth after contact with fresh water or humidity.
Clean with fresh water after exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

MAINTENANCE

The device must be examined for splinters and/or delamination regularly by a healthcare professional. Interval should be determined based on patient activity.
Note: Cheetah Xpanse must be examined for splinters and/or delamination every three months by a healthcare professional if used for long jump.

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur prosthetic devices are designed and verified to be safe and compatible in combination with each other and custom-made prosthetic sockets with Össur adapters, and when used in accordance with their intended use.

- Össur does not assume liability for the following:
- Device not maintained as instructed by the instructions for use.
 - Device assembled with components from other manufacturers.
 - Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to two million load cycles.
Depending on patient activity this may correspond to 1 year of use.

Cheetah Junior

ISO 10328 - Label		
Category	Weight (Kg)	Label Text
1	24	ISO 10328-P3*-24 kg
2	34	ISO 10328-P3*-34 kg
3	45	ISO 10328-P3-45 kg
4	55	ISO 10328-P3-55 kg
5	67	ISO 10328-P5-67 kg
6	84	ISO 10328-P6-84 kg
*Actual test loads reflect maximum body mass		

All other devices:

ISO 10328 - Label		
Category	Weight (Kg) Extreme Impact	Label Text
1	52	ISO 10328-P3-52 kg
2	59	ISO 10328-P3-59 kg
3	68	ISO 10328-P4-68 kg
4	77	ISO 10328-P5-77 kg
5	88	ISO 10328-P6-88 kg
6	100	ISO 10328-P7-100 kg
7	116	ISO 10328-P8-116 kg
8	130	ISO 10328-P8-130 kg
9	147	ISO 10328-P8-147 kg
*Actual test loads reflect maximum body mass		

ISO 10328 - “P” - “m”kg *) 

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use
see manufacturer’s written instructions on
intended use!

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