

Instructions for Use

FLEX-RUN™ JUNIOR
FLEX-RUN™
CHEETAH® XCEED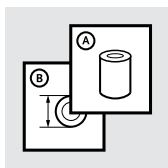

		3
EN	Instructions for Use	4
CS	Návod k použití	10
SK	Indikácie použitia	15
RO	Indicații de utilizare	21
HR	Upute za uporabu	27
HU	Használati javallat	33
BG	Указания за употреба	39
SL	Indikácie pre použitie	45

1

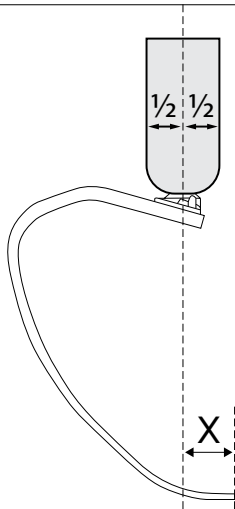ENGLISH

Medical Device

DESCRIPTION

The device is a prosthetic foot for sports activities.

INTENDED USE

The device is intended as a part of a prosthetic system that replaces the foot and ankle function of a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

Flex-Run Junior

The device is intended for children and adolescent patients.

Cheetah® Xceed/Flex-Run

The device is for high to extreme impact use, e.g., fast walking, running, and sports.

GENERAL SAFETY INSTRUCTIONS

Warning: Use of a lower limb prosthetic device carries an inherent risk of falling which may lead to injury.

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

The device is for single patient use.

DEVICE SELECTION

Verify that selected variant of the device is suitable for the weight limit according to the following table.

Warning: Do not exceed weight limit. Risk of device failure.

Incorrect category selection may also result in poor device function.

Flex-Run Junior

Weight kg	15-24	25-34	35-45	46-55	56-67	68-84
Weight lbs	33-53	54-75	76-99	100-121	122-147	148-185
Category	1	2	3	4	5	6

Flex-Run

Weight kg	37-44	45-52	53-59	60-68	69-77	78-88	89-100	101-116	117-130
Weight lbs	81-96	99-115	116-130	131-150	151-170	171-194	195-220	221-256	257-287
Category	1	2	3	4	5	6	7	8	9

Cheetah Xceed

Weight kg	45-52	53-59	60-68	69-77	78-88	89-100	101-116	117-130	131-147
Weight lbs	99-115	116-130	131-150	151-170	171-194	195-220	221-256	257-287	288-325
Category	1	2	3	4	5	6	7	8	9

ASSEMBLY INSTRUCTIONS

Caution: Epoxy can cause allergic reactions when in direct contact with body.

Caution: Sharp edges or splinters can cause a cutting hazard.

Attaching Pyramid

Applies to Flex-Run Junior and Flex-Run

If the foot is supplied without an adapter a suitable Össur adapter set must be used – see table below for compatibility.

Warning: Do not exceed adapter weight limit - see table below.

Attachment plate must be added when using male or female adapter.

Roughen top surface of the foot and attachment plate. Mix epoxy and apply to contact surfaces on foot and attachment plate. Bond both parts together and clamp securely using a center bolt and the adapter.

After resin has set, remove bolt and adapter, then trim off excess resin.

Clean parts, assemble, using threadlocker and torque as per following specifications.

Part Number	Description	Weight Limit	Center Bolt Torque	Bolt Size	Compatibility
FWX00030	Male Pyramid Adapter Set for cat 1 - 6	88 kg	41 Nm	M10	Both
CM150161	Female Pyramid Adapter Set for cat 1 - 6	88 kg	41 Nm	M10	Both
CM150162	Female Pyramid Adapter Set for cat 7 - 9	130 kg	43 Nm	M12	Flex-Run
FWX00031	Junior Male Pyramid Adapter Set for cat 1-3	45 kg	28 Nm	M8	Flex-Run Junior
FWX40001	Attachment Plate, for cat 1-6	N/A	N/A	M8/M10	Both
FWX40002	Attachment Plate, for cat 7-9	N/A	N/A	M12	Flex-Run

Female Pyramid Adapter Set Screw torque is 15Nm.

Flex-Run Large is compatible with Össur 4-hole adapters.

Attaching adapters (Cheetah Xceed)

Cheetah Xceed is compatible with Össur 4-hole adapters. The device should only be used with adapters with weight limit of 166 kg.

Nike Soles

The device can be fitted with a Nike spike pad or Nike sole. Refer to instructions for relevant sole/pad for assembly instructions.

ALIGNMENT INSTRUCTIONS

Bench Alignment

Sagittal plane

1. Put the ground contact point of the foot module in front of the alignment reference line according to dimension X (**Fig. 1**) in the table below.
2. Find the applicable socket flexion.

Coronal Plane

1. Set the applicable socket abduction/adduction.
2. Set for up to 5° of external rotation.
3. Set the correct height - see Compression in table below to compensate for vertical compression.

Device	Dimension X	Compression
Flex-Run Junior	25-40mm	10-20mm
Flex-Run	25-40mm	30-40mm
Cheetah Xceed	50-60mm	30-40mm

Dynamic Alignment

Physical ability and the length of residual limb will have an effect on alignment. Optimal alignment reduces energy expenditure and reduces muscle and ligament strain.

Adjust alignment to improve gait symmetry.

The A-P alignment and vertical deflection from socket to foot can have an effect on stride length.

Troubleshooting Guide

Make sure socket fit and suspension are optimal. Examine running symmetry at patient's preferred running speed.

If the duration of loading the prosthesis is longer compared to the contralateral side:

- The foot module is too soft
- The prosthesis is too short
- The foot is aligned too far anteriorly

If the duration of loading the prosthesis is shorter compared to the contralateral side:

- The foot module is too stiff
- The prosthesis is too long
- The foot is aligned too far posteriorly

Note: If you move the A-P position of the foot, there will be an effect on hip flexion, stride length and trunk motion.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

The device is Waterproof.

A Waterproof device can be used in a wet or humid environment and submerged in up to 3-meter-deep water for a maximum of 1 hour.

It can tolerate contact with: Salt water, chlorinated water, perspiration, urine, and mild soaps.

It can also tolerate occasional exposure to sand, dust, and dirt.

Continuous exposure is not allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water after exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

MAINTENANCE

The device must be examined for splinters and/or delamination regularly by a healthcare professional. Interval should be determined based on patient activity.

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur prosthetic devices are designed and verified to be safe and compatible in combination with each other and custom-made prosthetic sockets with Össur adapters, and when used in accordance with their intended use.

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to two million load cycles.

Depending on patient activity this may correspond to 1 year of use.

Flex-Run Junior

ISO 10328 - Label		
Category	Weight (Kg)	Label Text
1	24	ISO 10328-P3*-24 kg
2	34	ISO 10328-P3*-34 kg
3	45	ISO 10328-P3-45 kg
4	55	ISO 10328-P3-55 kg
5	67	ISO 10328-P5-67 kg
6	84	ISO 10328-P6-84 kg
*Actual test loads reflect maximum body mass		

Flex-Run

ISO 10328 - Label		
Category	Weight (Kg)	Label Text
1	45	ISO 10328-P4*-45 kg
2	52	ISO 10328-P4*-52 kg
3	59	ISO 10328-P4-59 kg
4	68	ISO 10328-P5-68 kg
5	77	ISO 10328-P6-77 kg
6	88	ISO 10328-P7-88 kg
7	100	ISO 10328-P8-100 kg
8	116	ISO 10328-P8-116 kg
9	130	ISO 10328-P8-130 kg
*Actual test loads reflect maximum body mass		

Cheetah Xceed

ISO 10328 - Label		
Category	Weight (Kg)	Label Text
1	52	ISO 10328-P3-52 kg
2	59	ISO 10328-P3-59 kg
3	68	ISO 10328-P4-68 kg
4	77	ISO 10328-P5-77 kg
5	88	ISO 10328-P6-88 kg
6	100	ISO 10328-P7-100 kg
7	116	ISO 10328-P8-116 kg
8	130	ISO 10328-P8-130 kg
9	147	ISO 10328-P8-147 kg

*Actual test loads reflect maximum body mass

ISO 10328 - "P" - "m"kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use
see manufacturer's written instructions on
intended use!

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