











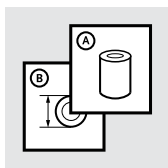
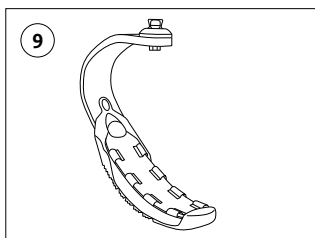
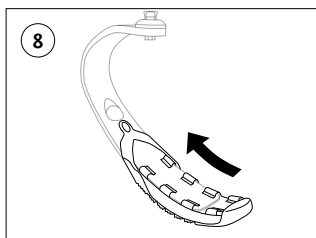
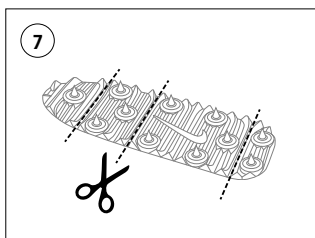
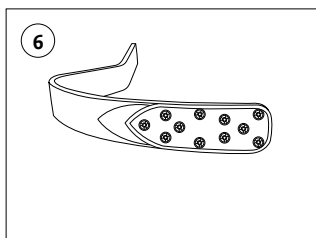
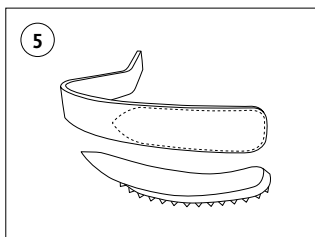
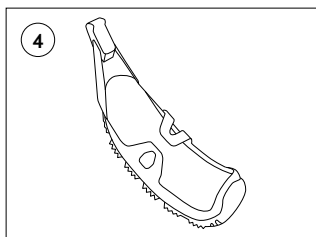
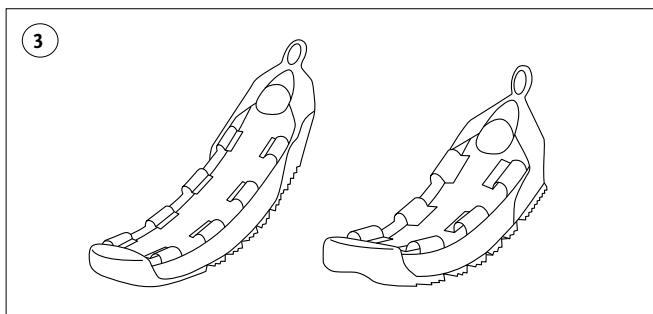
Instructions for Use




SPORT SOLES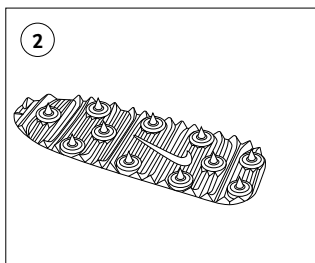

		3
EN	Instructions for Use	6
DE	Gebrauchsanweisung	11
FR	Notice d'utilisation	16
ES	Instrucciones para el uso	21
IT	Istruzioni per l'uso	26
NO	Bruksanvisning	31
DA	Brugsanvisning	36
SV	Bruksanvisning	41
EL	Οδηγίες Χρήσης	46
FI	Käyttöohjeet	51
NL	Gebruiksaanwijzing	56
PT	Instruções de Utilização	61
PL	Instrukcja użytkowania	66
CS	Návod k použití	71
TR	Kullanım Talimatları	76
RU	Инструкция по использованию	81
JA	取扱説明書	86
ZH	中文说明书	91
KO	사용 설명서	96
AR	تعليمات الاستخدام	102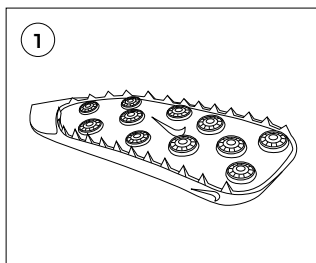

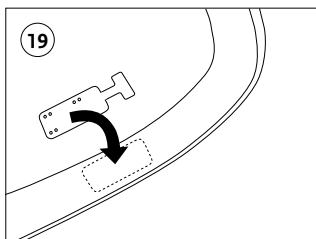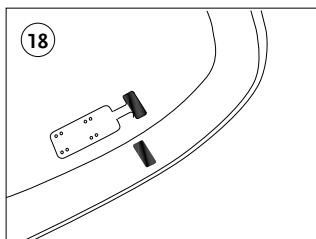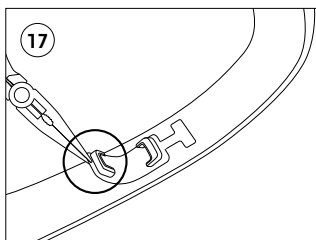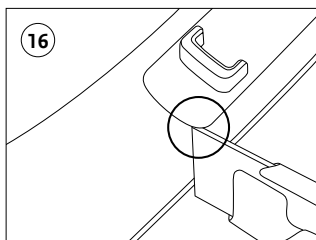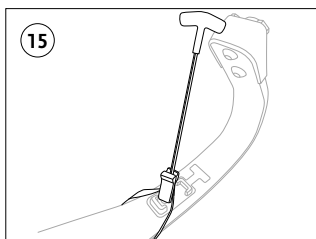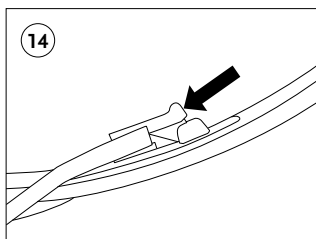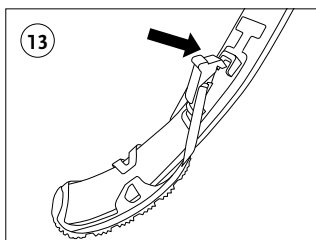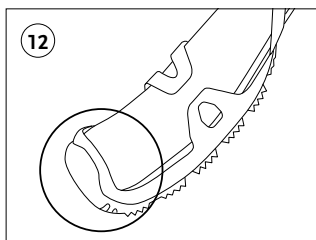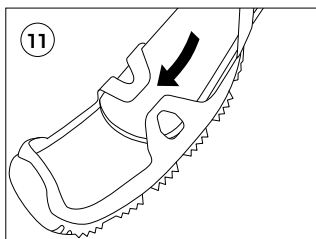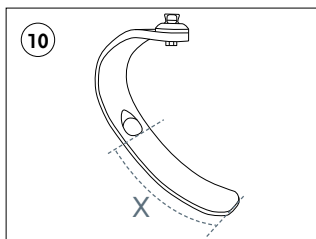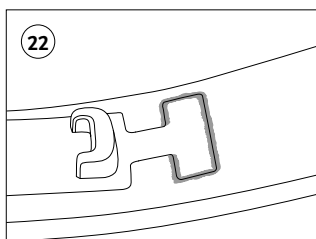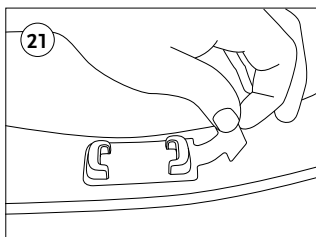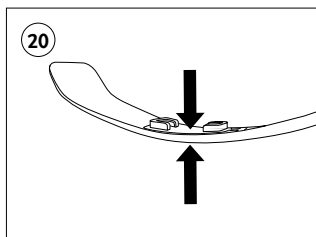ENGLISH

Medical Device

The sport soles are referred to as the “*device*” in the following document. This document is intended for healthcare professionals and patients.

PRODUCT DESCRIPTION

The device is a sport sole that is fitted on a prosthetic foot for sports activities.

This document covers the following devices:

- Nike Spike Pad (**Figure 1**)
- Nike Spike Pad 2.0 (**Figure 2**)
- Nike Sole (**Figure 3**)
- Nike Sole Junior (**Figure 3**)
- Nike Traction SoleX (**Figure 4**)
- Nike Spike SoleX (**Figure 4**)

INDICATIONS FOR USE

- Lower limb amputation
- Lower limb deficiency

CONTRAINDICATIONS

None known.

INTENDED USE

This device is a modular component for a lower limb prosthesis. Suitability of the device for the prosthesis must be evaluated by a healthcare professional.

The Nike Spike Pad and Nike Spike Pad 2.0 are spike soles to be used with sport feet during track and field activities.

The Nike Sole and Nike Sole Junior are traction devices made for running and sports activities.

The Nike Traction SoleX and Nike Spike SoleX are interchangeable sole systems that can be used on the same foot and are intended for sport activities.

The device is for single patient use.

The device is for high to extreme impact use, e.g. fast walking, running and sports.

GENERAL SAFETY INFORMATION

Inform your patient about everything in this document that is required for safe use of this device.

Warning: In cases of functional change or functional loss, the patient should stop using the device and contact a healthcare professional.

Warning: If the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

REQUIRED COMPONENTS

Sport foot that fits the intended sole is needed. Overview of fit between sole and sport foot is provided in table below.

Soles are compatible to all categories of their intended feet.

	Nike Spike Pad (Figure 1)	Nike Spike pad 2.0 (Figure 2)	Nike Sole (Figure 3)	Nike Sole Junior (Figure 3)	Nike Traction SoleX (Figure 4)	Nike Spike SoleX (Figure 4)
Flex-Foot Cheetah®	x	x				
Cheetah® Xtreme	x	x				
Cheetah® Xtend	x	x				
Cheetah® Xcel	x	x			x	x
Cheetah® Xpanse	x	x				
Cheetah® Xceed	x	x			x	x
Flex-Run™			x			
Flex-Run™ Junior				x		
Cheetah® Junior				x		

Nike Spike Pad and Nike Spike Pad 2.0:

For the Nike Spike Pads, a glue must be used to bond the sole to the foot.

Nike Sole and Nike Sole Junior:

For the Nike Soles a medallion must be glued onto the foot. The medallion is then used to secure the sole to the foot.

Nike Traction SoleX and Nike Spike SoleX:

For SoleX system the foot must contain an attachment that is glued onto the foot.

ASSEMBLY INSTRUCTIONS

Nike Spike Pad and Nike Spike Pad 2.0

Bonding the Nike Spike Pad to the foot (**Figure 5 and 6**):

- Trace the outer lines of the sole onto the foot module
- Sand the bottom surface of the foot module
- Sand the top surface of the sole
- Clean both surfaces with acetone
- Bond the sole using contact cement adhesive or industrial strength glue
- Follow glue manufacturer bonding instruction

Note: For Cheetah Xpanse and Cheetah Xcel. All 11 spikes should be inserted into the Nike Spike Pad before use.

Note: Nike Spike Pad 2.0 has built in trim lines for trimming length of sole if needed (**Figure 7**).

Nike Sole and Nike Sole Junior

Donning and Doffing the Nike Soles on feet:

- Slide the sole on to the foot and make sure all the clips go over the foot (**Figure 8**)
- Pull the rubber strap of the sole over the medallion and slide it under (**Figure 9**)
- When doffing the sole, start by pulling the strap away from the medallion, then slide the sole off the foot

- For tighter fit of the sole or to reduce noise: Bond plastic insert inside of sole using Loctite 495.
Keep under pressure for 10 minutes to cure

Replacement of the medallion **(Figure 10)**:

- For optimal placement of the medallion, measure along the foot blade from tip of the toe to lower edge of medallion, and mark:
 - For Nike Sole: 21.5 cm
 - For Nike Sole Junior: 12.5 cm
- Sand bottom of the medallion
- Clean bonding surfaces with acetone
- Bond medallion to foot blade using Loctite 495. Keep pressure on medallion for 10 minutes to cure

Nike Spike SoleX and Nike Traction SoleX

Donning and Doffing the SoleX on feet:

- Slide the sole on to the foot and make sure both clips go over the foot **(Figure 11)**
- The toe of the foot should engage within the front pocket of the sole **(Figure 12)**
- Lock latch into SoleX attachment on foot **(Figure 13)**
- When doffing the sole, start by releasing the latch by pressing the button down to open lock **(Figure 14)**. Then slide the sole off the foot
- Always check if lock is engaged and sole is securely fastened to foot before using device
- If the system creates noise or is perceived as loose: The system should be tensioned on the blade by tightening the screw clockwise into the system **(Figure 15)**
- To reduce tensions in system, the screw can be turned counterclockwise

Replacement / Retrofitting of SoleX attachment

Removing SoleX attachment from foot:

- Remove carbon plate by first loosening the front side of plate with knife blade or standard screwdriver **(Figure 16)**
- Once plate is slightly loose, grip with pliers and peel away from the foot **(Figure 17)**
- Repeat same steps for back side of plate
- Once plate has been removed from the bonding surface, all remaining glue should be completely removed by sanding

Bonding SoleX attachment to foot:

- For optimal placement of new plate on foot:
 - Measure along the foot blade from tip of the toe to lower edge of plate approximately 19 cm and mark **(Figure 10)**
 - Alternatively, the system can be clipped onto the latch. Sole should then be donned on foot and system tensioned. Mark location of plate.
- Sand the gluing area of the plate and mating surface on foot **(Figure 18)**
- Clean gluing area with acetone
- Remove protective film on bottom of plate and clean mating area on foot with acetone **(Figure 19)**
- Locate plate on foot and align
- Apply pressure and hold for at least 10 minutes until plate is fastened to blade.

Note: clamp can be used to apply pressure (**Figure 20**)

- Pull back section of plate away from blade (**Figure 21**)
- Apply recommended glue (see catalogue) onto highlighted area on foot (**Figure 18**)
- Press back section of plate into glue and hold
- Apply glue over the highlighted outer edges of plate (**Figure 22**)
- Apply pressure on area and hold for 10 minutes (**Figure 20**)
- Once complete, don sole on foot and check for fit. Turn screw inside latch clockwise if more tension is needed in system

PRODUCT CARE

Maintenance (Nike Sole, Nike Sole Junior and SoleX system)

Noise may occur if sand or debris is present in device. In that case, the device can be doffed, cleaned, and re-fitted.

Clean device with water if it is exposed to sand or similar substances to avoid increased wear or noise.

The device should be examined regularly by a healthcare professional. Interval should be determined based on patient activity.

Cleaning (all soles)

The device can be rinsed with fresh water and/or cleaned using a wet cloth with a mild soap. The device should be dried with a cloth after cleaning.

ENVIRONMENTAL CONDITIONS

The device can be used in temperatures between -15°C to 50°C. Exception for Nike Spike Pad: -10°C to 40°C.

The Nike Sole, Nike Sole Junior, Nike Spike Pad 2.0, Nike Traction SoleX and Nike Spike SoleX are waterproof.

A waterproof device can be used in a wet or humid environment and temporarily submerged in shallow fresh water.

Dry thoroughly after contact with fresh water. No salt water or chlorinated water allowed. Clean with fresh water in case of accidental exposure to salt water or chlorinated water.

REPORT A SERIOUS INCIDENT

Important notice to users and/or patients established in Europe:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.

- Device used outside of recommended use condition, application, or environment.

COMPLIANCE

This device has been tested according to ISO 10328 standard to two million load cycles.

The load level specification is shown in the table below.

The device service life cannot be determined since load varies depending on the sport and patient activity level.

Category	Weight (Kg) Extreme impact	Label Text
1	52	ISO 10328-P3-52 kg
2	59	ISO 10328-P3-59 kg
3	68	ISO 10328-P4-68 kg
4	77	ISO 10328-P5-77 kg
5	88	ISO 10328-P6-88 kg
6	100	ISO 10328-P7-100 kg
7	116	ISO 10328-P8-116 kg
8	130	ISO 10328-P8-130 kg
9	147	ISO 10328-P8-147 kg

ISO 10328 - "P" - "m"kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use
see manufacturer's written instructions on
intended use!

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	Nike Spike Pad (Şekil 1)	Nike Spike pad 2.0 (Şekil 2)	Nike Sole (Şekil 3)	Nike Sole Junior (Şekil 3)	Nike Traction SoleX (Şekil 4)	Nike Spike SoleX (Şekil 4)
Flex-Foot Cheetah®	x	x				
Cheetah® Xtreme	x	x				
Cheetah® Xtend	x	x				
Cheetah® Xcel	x	x			x	x
Cheetah® Xpanse	x	x				
Cheetah® Xceed	x	x			x	x
Flex-Run™			x			
Flex-Run™ Junior				x		
Cheetah® Junior				x		

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Nike Traction SoleX ve Nike Spike SoleX:

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Nike Spike SoleX ve Nike Traction SoleX

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