Instructions for Use

---

## ALIGNMENT ADAPTERS


Dynamic L - Adapter 13°

Dynamic - Adapter 10°









Bolted L - Adapter 13°






Bolted - Adapter 10°

Alignment adapter Xpanse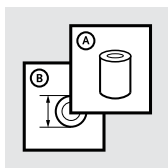


		3
EN	Instructions for Use	6
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	12
ES	Instrucciones para el uso	15
IT	Istruzioni per l'uso	18
NO	Bruksanvisning	21
DA	Brugsanvisning	24
SV	Bruksanvisning	27
EL	Οδηγίες Χρήσης	30
FI	Käyttöohjeet	33
NL	Gebruiksaanwijzing	36
PT	Instruções de Utilização	39
PL	Instrukcja użytkownika	42
CS	Návod k použití	45
TR	Kullanım Talimatları	48
RU	Инструкция по использованию	51
JA	取扱説明書	54
ZH	中文说明书	57
KO	사용 설명서	60
AR	تعليمات الاستخدام	63

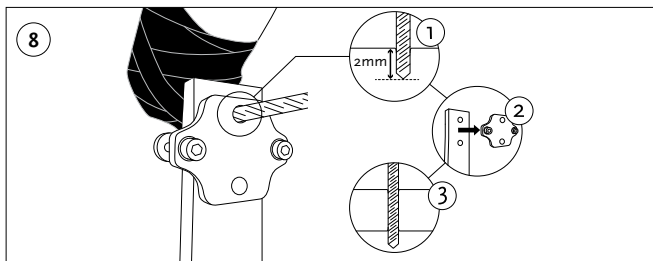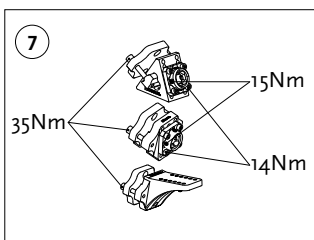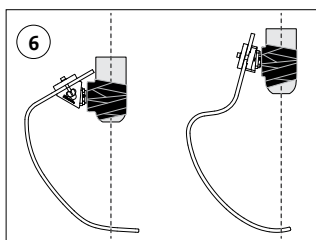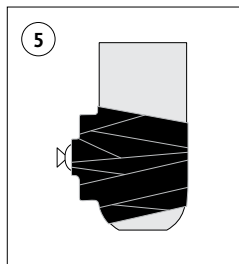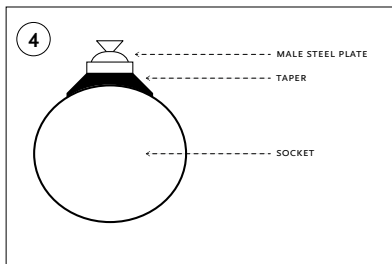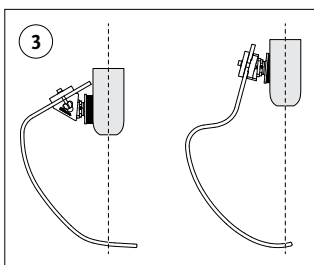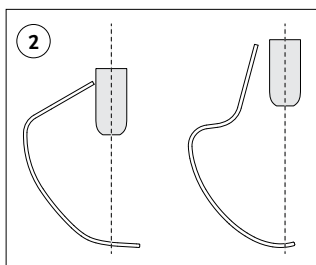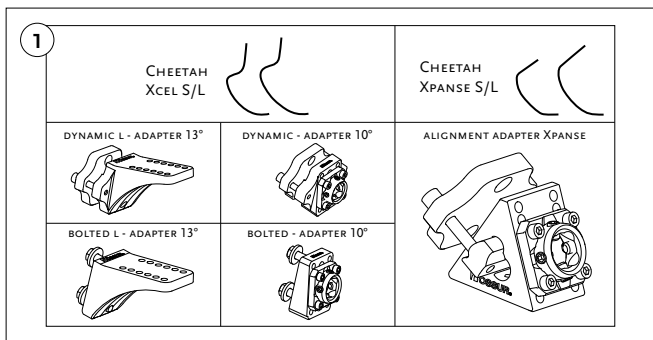
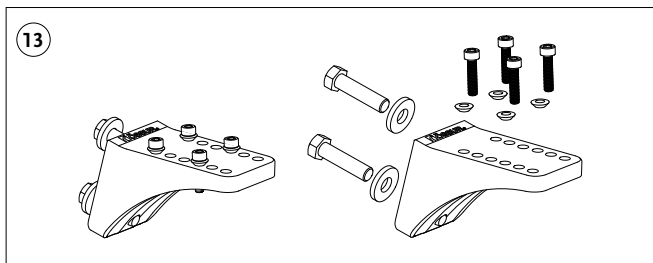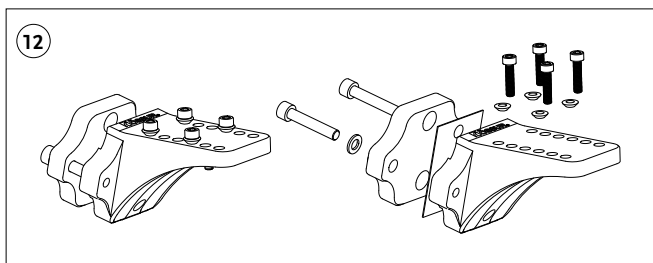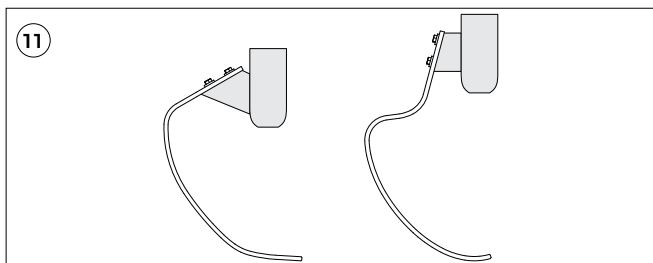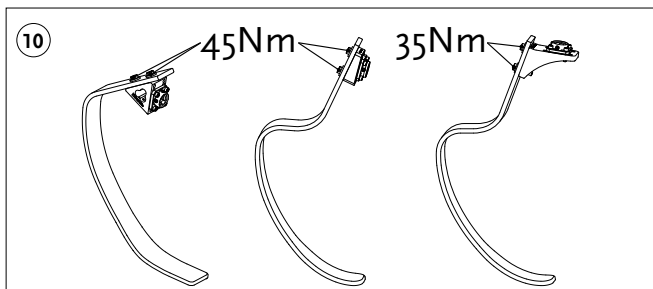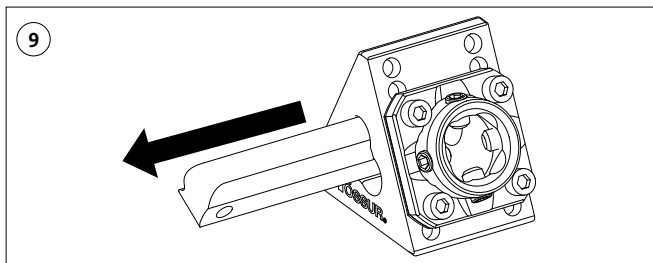
14

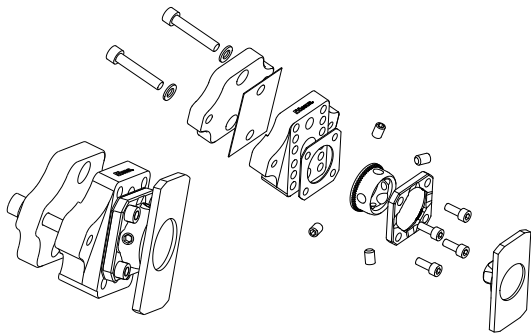15

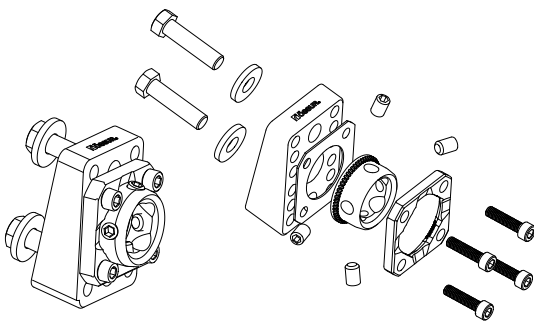16
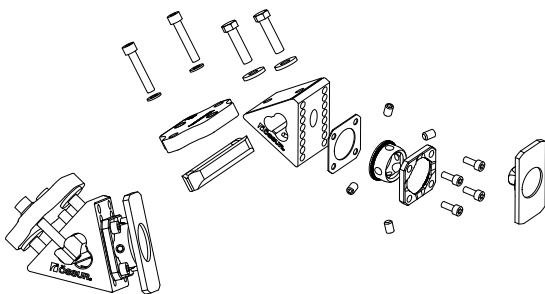
# ENGLISH

---

Medical Device

## PRODUCT DESCRIPTION

Thank you for choosing an Össur Product

The device is an alignment adapter for Össur sports feet. This document covers the following products.

Product Name	Intended use	Product Weight	Patient Activity
Dynamic L- Adapter 13°	Alignment only	700 g	Walk, jog
Bolted L- Adapter 13°	Long term	500 g	High activity
Dynamic - Adapter 10°	Alignment only	720 g	Walk, jog
Bolted - Adapter 10°	Short term	350 g	High activity (NOT jumping)
Alignment Adapter Xpanse	Alignment/ short term	980 g	High activity (NOT jumping)

## INDICATIONS FOR USE

Lower limb amputation.

## CONTRAINDICATIONS

None known.

## INTENDED USE

Maximum patient weight is 100 kg (220 lbs)

The adapters are intended to aid alignment of Össur sports feet. They can be used temporarily to adjust alignment to individual patients before attaching the sport foot more permanently to the socket. The Bolted L-Adapter 13° can be used long term. The device service life cannot be determined since load varies depending on the sport and patient activity level.

## GENERAL SAFETY INSTRUCTIONS

**WARNING:** The Alignment Adapter Xpanse and Bolted Adapter 10° are NOT to be used for jumping. Structural failure may occur.

**WARNING:** In cases of functional change or functional loss, the patient should stop using the device and contact a healthcare professional.

**WARNING:** If the device shows signs of damage or wear, the patient should stop using the device and contact a healthcare professional.

## PRODUCT SELECTION / VERIFICATION

The devices can be used with Össur feet that use a lamination connection.

## ASSEMBLY INSTRUCTIONS

### *Dynamic adapters*

1. Perform bench alignment in jig according to foot instructions.  
**NOTE:** If a sole is to be used then remember to include thickness of sole in bench alignment.
2. Clamp dynamic adapter on the foot. Align so that at least 1/3 of socket is free from bottom. Adjust steel plate if needed.
3. Connect steel plate to socket with Epoxy or Composite 1, (**Figure 3**).  
**NOTE:** Sand the joining area of the steel plate and the socket for better bonding before bonding steel plate to socket.
4. Once epoxy is cured, remove socket from jig. Sand the epoxy to form a slight taper, (**Figure 4**).
5. Wrap casting tape around steel plate and socket. Trim casting tape around male pyramid if needed, (**Figure 5**).
6. Fasten socket to dynamic adapter (**Figure 6**). Use friction pad between blade and adapter.
7. Torque according to **Figure 7**.
8. Let the patient walk and/or jog on prosthesis.
9. Adjust height and alignment as needed.
10. Ensure the patient has enough clearance during knee flexion.
11. Trim pylon of foot module min 1" from adapter to allow for later height adjustments. Use a hack saw or a band saw to trim the foot.

### *Bolted adapters*

1. Drill holes in the foot with 3/8" or Ø9,50 mm drill using the drill guide holes in dynamic adapter (**Figure 8**).  
**NOTE:** For Cheetah Xpanse foot: Drill only 2 mm into foot. Then remove bracket and complete drilling through the foot.
2. Remove adapter from foot. Remove clamping plate.  
**NOTE:** For Cheetah Xpanse remove also mid-section (**Figure 9**).
3. Bolt adapter onto foot and torque according to Figure 10. Check bolt for full thread engagement. Use thread locker on all screws.
4. The patient can now run on blade.

**WARNING:** The Alignment Adapter Xpanse and Bolted Adapter 10° are NOT to be used for jumping. Structural failure may occur. For long term usage it is recommended to remove adapter and connect the foot directly to the socket. For this a lamination connector can be used.

## ENVIRONMENTAL CONDITIONS

The device is usable in all normal weather conditions and daily activities. Weatherproof - Can tolerate light splashing (e. g. rain).

## REPORT A SERIOUS INCIDENT

### **Important notice to users and/or patients established in Europe:**

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## DISPOSAL

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations.

## **LIABILITY**

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

## **COMPLIANCE**

This device has been tested according to ISO 10328 standard to two million load cycles. The device service life cannot be determined since load varies depending on the sport and patient activity level.

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