


Instructions for Use

EXOFORM[®] DORSAL NIGHT
SPLINT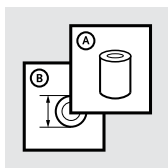

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	6
FR	Notice d'utilisation	8
ES	Instrucciones para el uso	10
IT	Istruzioni per l'uso	12
NO	Bruksanvisning	14
DA	Brugsanvisning	16
SV	Bruksanvisning	18
EL	Οδηγίες Χρήσης	20
FI	Käyttöohjeet	22
NL	Gebruiksaanwijzing	24
PT	Instruções de Utilização	26
PL	Instrukcja użytkowania	28
JA	取扱説明書	30
ZH	中文说明书	32
KO	사용 설명서	34
HU	Használati javallat	36

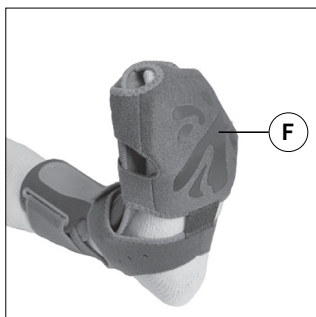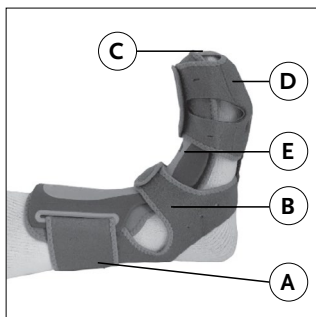ENGLISH

Medical Device

INTENDED USE

The device is intended to keep the foot/ankle in a neutral position

Indications for use

Conditions that may benefit from resting with the ankle in a neutral position, such as:

- Plantar Fasciitis
- General Heel Pain
- Achilles Tendonitis
- Post-Static Dyskinesia
- Drop Foot

Contraindications

Plantar flexion contracture greater than 20°, diabetes, neuropathy and/or spasticity of lower extremities.

Warnings and Cautions:

- This product is not intended for prolonged walking or weight bearing. Any ambulation in the product should be kept to a minimum.
- Care should be taken not to overtighten the device.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

Never wear the device directly over an open wound.

The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

USAGE

Device Application

The device should be applied while seated.

An athletic sock may be worn on foot for additional comfort.

1. Loosen the calf strapping (A) without pulling through D-rings and unfasten the heel strapping (B) to provide enough room to pass the foot into and through the soft good (**fig. 1.**).
2. Ensure that the device is “*right side up*” – the anti-slip sole (F) should be positioned on the bottom, under the forefoot. Slide the foot into the device until the toes reach the end of the soft good as indicated by the toe strap (C). The toes should not extend past the toe strap (**fig. 2.**).
3. Secure the device to the foot by first adjusting the foot strapping (D) as necessary being careful to maintain a gentle and comfortable stretch of the foot (**fig. 3.**). Do not over-tighten the foot strapping.

4. Position the shell (E) against the shin and apply the calf strapping to secure the device in a comfortable manner (**fig. 4.**).
5. Fasten the heel strapping being sure to center the heel of the foot into the heel strapping opening as illustrated (**fig. 5.**). Do not overtighten the heel strapping.

Device removal

Loosen the calf strapping without pulling through D-rings and unfasten the heel strapping to provide enough room to remove the device from the foot.

Device Adjustments

The device has been pre-shaped to provide the optimal degree of stretch to the foot. In situations where a more customized level of stretch of the foot is required, the device shell can be modified by hand to either increase or decrease the angle and degree of flexion of the foot.

When modifying the angle of the shell to increase or decrease the stretch of the foot, the device should be held and bent in such a way as to create a bending point at the mid section of the shell in the area of the heel strapping. The shell is not intended to be repeatedly bent and modified as this will adversely affect product integrity.

Note: The shell angle should be adjusted and modified only in special circumstances and only by a healthcare professional. Repeated modification of the shell angle will compromise the shell integrity.

Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry

Note: Do not tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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