ENGLISH

Medical Device

INTENDED USE

The device is intended to pull the shoulders back.

Indications for use

For conditions that may benefit from pulling the shoulders back, such as:

- Clavicle fractures and dislocations
- Postural problems
- Selected soft tissue injuries

No known contraindications.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities. Never wear the device directly over an open wound.

The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

USAGE

Device Application

1. Unfold device and unfasten strap ends from the buckles.
2. Position the felt pad on back between the shoulder blades with the arm straps lying over the shoulders and to the front of the body. Ensure the straps are not twisted and the arm pads are between the straps and the shoulders.
3. Wrap the straps under the arm and feed the straps through the buckles. Pull tight as desired. The straps should be tight enough to pull the shoulders back. The arm pad should be adjusted so that it fits under the arm and in front of the shoulder.

Device removal

Unfasten the strap ends from the buckles to remove the device.

Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry

Note: Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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