







Instructions for Use

UNITY[®] FOR ASSURE &
BALANCE[™] FOOT J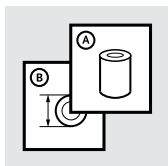

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	13
ES	Instrucciones para el uso	17
IT	Istruzioni per l'uso	21
DA	Brugsanvisning	25
SV	Bruksanvisning	29
EL	Οδηγίες χρήσης	33
FI	Käyttöohjeet	37
NL	Gebruiksaanwijzing	41
PT	Instruções de Utilização	45
PL	Instrukcja użytkownika	49
JA	取扱説明書	53
ZH	中文说明书	57
KO	사용 설명서	61

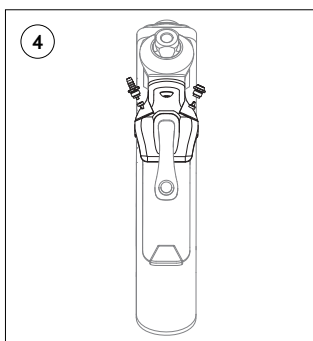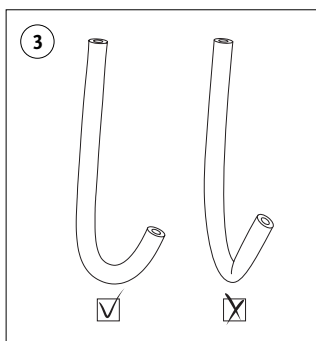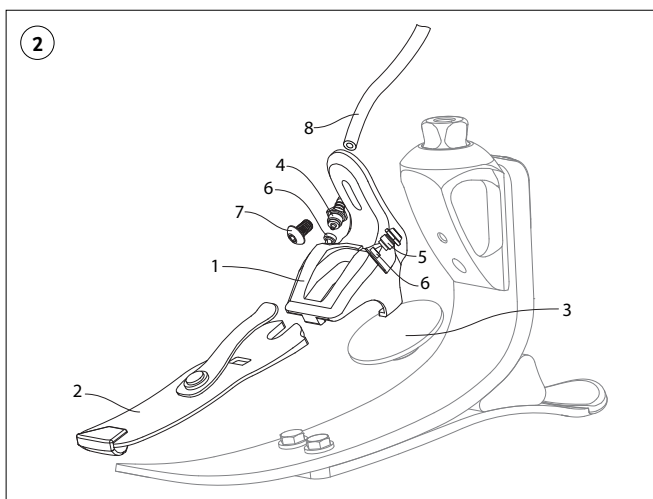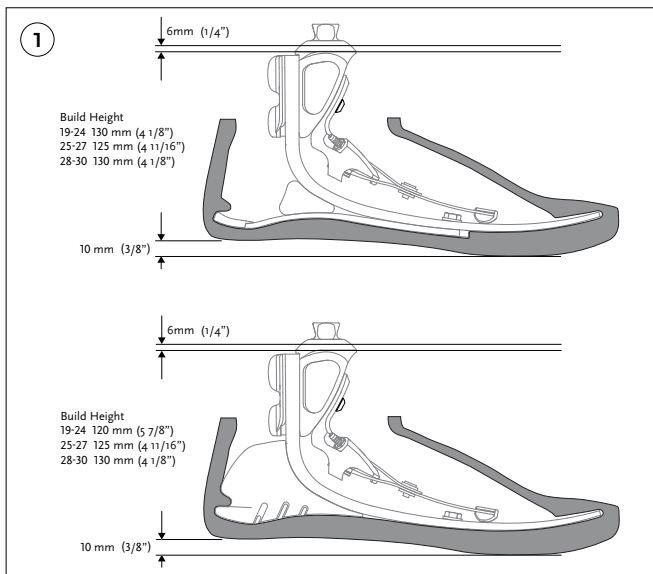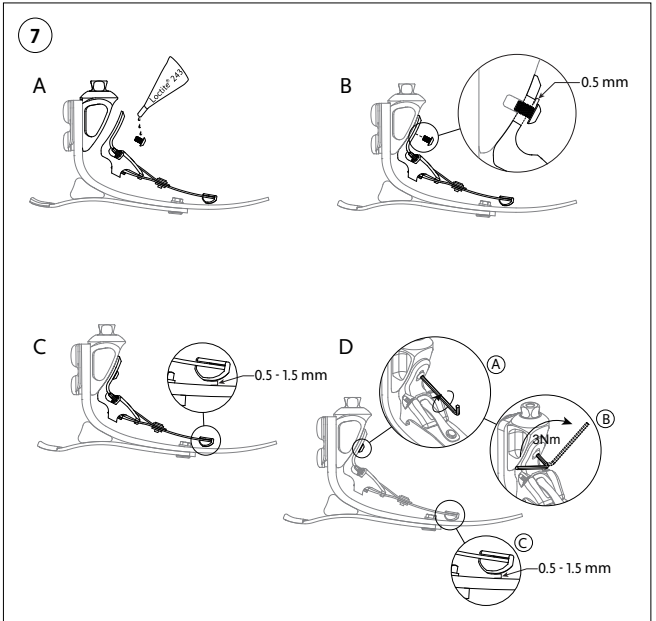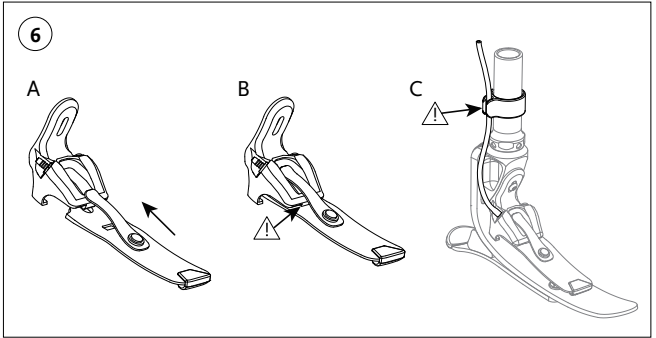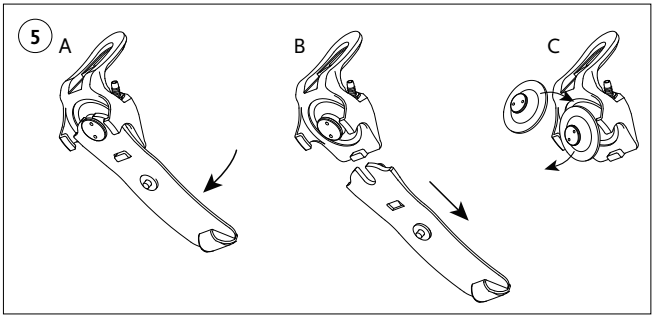UNITY® SYSTEM COMPONENTS (Figure 2)

1. Membrane Housing
2. Pump Blade
3. Membrane
4. Inlet Port
5. Outlet Port
6. Duckbill
7. Mounting Screw
8. Tube

Flex-Foot Assure / Balance Foot J

See IFU for Flex-Foot Assure / Balance Foot J

SAFETY PRECAUTIONS

Please follow the safety precautions as failure to do so can lead to malfunction of the Unity System and risk injury to the user.

- This product is to be used exclusively for unilateral or bilateral major lower-limb amputations.
- Use a Unity Socket Valve Kit to connect the system to the socket.
- Make sure that the socket is a close fit to prevent blistering and/or wounds.
- Make sure the tube is not folded, bent, or closed off in any way (**Figure 3**).
- Make sure the tube is secured firmly away from any sharp objects or components.
- Clinical specialists should make sure their clients are aware of and understand the following:
 - The tube must be well secured at all times.
 - Never operate this product if a change in its operation is observed. In such cases the user should consult the clinical specialist.

CAUTION: U.S. federal law requires this device to be sold by a prosthetist and by the order of a physician. The clinical specialist must provide each user with training for appropriate use and proper application of the Unity System.

UNITY SYSTEM OVERVIEW

The Unity System is designed to provide elevated vacuum for socket suspension. Expected attainable vacuum achieved by the Unity System during normal operation ranges from 15-18 inHg or higher.

With the Unity System, assisted suspension is obtained through the act of a pump which draws air out from the socket in each step creating an elevated vacuum inside the socket interface.

The Unity System is designed to function in any temperature and humidity that the user encounters during normal use.

Unity for Flex-Foot Assure / Balance Foot J is single patient device.

The users of Unity for Flex-Foot Assure / Balance Foot J are clinical specialists and end user. This device should only be used with a prescription from a physician that includes the necessary treatment information.

NOTE: The Unity System is designed to be used with Iceross Seal-In® liners, other applications are sole responsibility of the provider.

INDICATIONS

Prosthetic interface with suspension properties for a major lower limb amputee; Low impact use.

CONTRAINDICATIONS

Amputee equipped with a loose fitting socket.

Amputee is unable to follow instructions.

Amputee is unable to ambulate with a lower limb prosthesis.

WARNING!

Make sure that the socket is well fitting before any use; a loose fitting or leaking socket may never be used with the Unity System.

STRUCTURAL SPECIFICATION

Net weight: approx. 104g including tube.

Frame construction: Carbon fiber and aluminum frame.

Clearance and dimensions: See IFU for Flex-Foot Assure / Balance Foot J.

FITTING THE UNITY SYSTEM

NOTE: To connect the system to a socket a Unity Socket Valve Kit is recommended, the use of other applications is the sole responsibility of the provider.

1. Fit the user with the foot containing the system
2. Cut the tube to correct length and connect to the Unity System and the socket (using the Unity Socket Valve kit)
3. Use the supplied tube fixture to secure the tube against the pylon. The tube should be located medially rather than laterally to avoid damage during ambulation. Once the Unity System has been connected make sure the pump is working properly; creating vacuum during gait and not leaking air.

DISASSEMBLING / ASSEMBLING THE UNITY SYSTEM

The Unity System can be disassembled in case it needs to be cleaned or for troubleshooting purposes.

CAUTION: The Unity System should only be disassembled / assembled by a clinical specialist.

CAUTION: The inlet and outlet (**Figure 4**) ports on the membrane housing should not be disassembled unless absolutely necessary. This part of the system contains miniature duckbill valves that are easily lost and can cause the Unity System to malfunction if not correctly reassembled.

HOW TO CHANGE MEMBRANE IN THE UNITY SYSTEM

1. Use the Flex-foot shoehorn (or equivalent) to remove the foot from the foot cover.
2. Disconnect the tube from the Unity Membrane housing.
3. Untighten the screw that fastens the membrane housing to the Flex-Foot Assure / Balance Foot J pyramid.
4. Follow Figure 5.
5. Lubricate the membrane before sliding it into the housing with Iceross Clean&Simple Spray or equivalent.

HOW TO ASSEMBLE THE UNITY SYSTEM

See Figure 6.

AFTER DISASSEMBLY / ASSEMBLY

Check the following:

- Are the duckbill valves oriented in the correct direction (**Figure 4**)?
- Is the membrane fully inserted into the housing?
- Are the inlet and outlet ports tightened?
- Is the tube fully inserted on the inlet port? (Check both at the Unity Socket valve and at the Unity Membrane housing)

FINISHING / LINER OPTIONS

Össur recommends that the Unity System is used with an Iceross Seal-In liner.

CAUTION: Make sure the Flex-Foot sock does not interfere with the Unity System as it could compromise the function of the pump.

THE TUBE

The tube is the most sensitive part of the Unity System and special attention should be paid to it for potential functional issue that might arise.

CAUTION: Ensure that Unity tubing is not compressed or folded by the application of cosmetic finishing.

Make sure the tube is NOT:

- Folded (**Figure 3**).
- Pressed up against sharp objects.
- Punctured.
- Dangling during gait.
- Located laterally, as there is more likelihood of damage.

IN CASE THE UNITY SYSTEM IS NOT FUNCTIONING PROPERLY

Check the following:

- Is there a leakage in the socket?
- Is the tube punctured?
- Is the membrane punctured or torn?
- Has the membrane been pulled out of the membrane housing?
- Is there dust or grit inside the system?
- Is the tube fully inserted on the inlet port? (Check both at the Unity Socket valve and at the Unity Membrane housing)
- Are the inlet and outlet ports tightened?
- Is the Unity Socket valve leaking?
- Is the sock inside the foot cover affecting the systems mechanical movements?

MAINTENANCE

The following components of the Unity System might need cleaning or replacing during the lifetime of the product. This is to be considered normal wear and is not covered by the Limited Product Warranty.

- Membrane
- Duckbill valves
- Tube

It is recommended to have the product inspected every 6 months by a clinical specialist.

Clean the components with mild soap and rinse with water, isopropyl alcohol or Iceross Clean&Simple spray.

DO NOT use solvents stronger than isopropyl alcohol.

Please refer to the current Össur printed catalog or online pages via www.ossur.com for replacement parts available and appropriate item numbers.

WARRANTY

The Unity System has a 24 month limited warranty, excluding the membrane.

The membrane has a 12 month limited warranty.

The following activities will void the warranty:

- Exposure to seawater or extreme corrosive environment.
- Usage with Copolymer Gel liners.
- Usage without liners or other means which can lead to sweat or other fluids than Iceross Lubricant entering the system.
- Exposure to environment with extreme airborne particle contamination.
- Abuse and damage caused by abnormal impact.
- Dents and scratches on sealing surfaces.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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