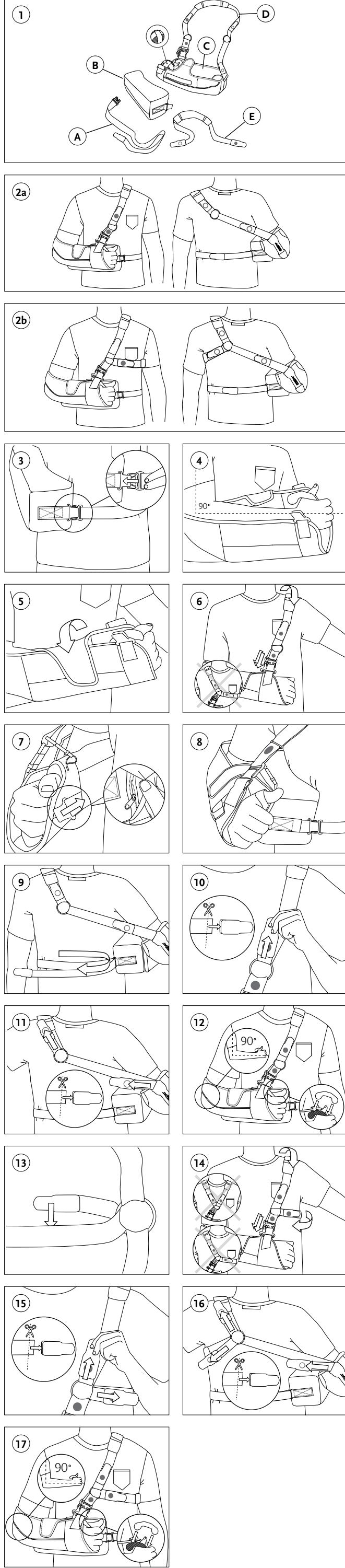
ENGLISH

Medical Device

INTENDED USE

The device is intended to immobilize the shoulder and/or elbow

Indications for use

- Indication for rotator cuff repairs
- Bankart repairs
- Capsular shifts
- Glenohumeral dislocations/subluxations
- Soft tissue repairs/strains
- Shoulder instabilities/injuries
- No known contraindications.

Warnings and Cautions:

- This device is designed to be worn, in part, directly on the skin. Inspect the skin under the device's contact area before and after each use.
- If you have any questions as to how this device applies to your specific condition, consult a healthcare professional.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference. Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities. The patient should stop using the device and inform their healthcare provider if:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reactions occurs with the use of the device.

The device is for single patient – multiple use. This device is a supportive brace only and is not intended or guaranteed to prevent injury.

USAGE

While carrying out the following instructions, please refer to the overview figure for the device components mentioned in the text (Fig. 1).

Device Application

Abduction Pillow and Sling (Fig. 2a)

1. Detach the quick-release buckle of the Immobilizer Strap (A) to the Abduction Pillow (B) (Fig. 3).
2. Place the forearm into the Sling (C) at a 90° angle, seating the elbow snugly in the corner pocket of the Sling and the thumb through the thumb loop (Fig. 4). The thumb loop can be repositioned, and the installed elbow pillow can be removed, if necessary, to improve fit and comfort.
3. Close the top flap to secure the front panel (Fig. 5).
4. Note: Right arm application is shown in the figure. For left arm application, the flap will fold towards the body.
5. Place the adjusted Shoulder Strap (D) around the neck over the opposite shoulder and fasten it to the Sling via the quick-release buckle (Fig. 6). Position the shoulder strap pad where it is most comfortable.

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5. Locate the pocket on the side of the Sling facing towards the body and ensure the device is placed into the pocket (Fig. 7). Position the Abduction Pillow above the hip with the curved surface against the body and the quick-release buckle towards the front (Fig. 8). Using the double-sided adhesive strip, connect the Sling to the Abduction Pillow.
6. Secure the Abduction Pillow in place using the Immobilizer Strap and the D-Ring at the back of the pillow (Fig. 9). Fold the strap at the appropriate length into the alligator clip and fasten the strap.
7. Re-adjust the Shoulder Strap from the front (Fig. 10) or back (Fig. 11), if necessary.
8. Note: When final fit has been confirmed (Fig. 12), trim the straps to the appropriate length.

Offloading strap (Fig. 2b)

8. To use the optional Offloading Strap (E), feed the Offloading Strap through the O-Ring behind the Abduction Pillow in the opposite direction, and fasten it snugly to itself with the alligator clip (Fig. 13).
9. Pass the Offloading Strap underneath the opposite arm and secure using the O-Ring above the affected wrist (Fig. 14). Position the strap so that where it is most comfortable.
10. If necessary, adjust the Shoulder and Offloading Strap from the front (Fig. 15) or back (Fig. 16). Note: If the strap is too long, remove the alligator clip, trim the strap to length, and re-attach the alligator clip. A correctly applied device with Offloading Strap (Fig. 17).

Device Removal

For easy drying, unfasten the quick-release buckles of the Abduction Pillow and Sling. Remove the Sling and open the top flap of the front panel to remove the arm from the device.

Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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