ENGLISH

MD Medical Device

INTENDED USE

The device is intended to relief pain due to epicondylitis

Indications for use
No known contraindications.

Warnings and Cautions:

- The device should not be used for injuries requiring orthopedic treatment with a plaster cast, resin cast or surgery
- The strap must not be in direct contact with a skin lesion.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

USAGE

Device Application

- Pull the strap onto the forearm.
- Tighten using the strap to obtain firm but comfortable support.

Note: The cushions should be placed over the painful tendons (just below the elbow) (**Fig. 1,2**).

Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry.

Note: Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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