LIABILITY

Össur does not assume liability for the following:
• Device not maintained as instructed by the instructions for use.
• Device assembled with components from other manufacturers.
• Device used outside of recommended use condition, application, or environment.

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Indikationen für die Verwendung und Ziel-Patientengruppe
The device is intended to connect and release a prosthetic system that replaces a missing lower limb.
Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.
The device must be fitted and adjusted by a healthcare professional.

INTENDED USE

The device is intended to connect and release a prosthetic system that replaces a missing lower limb.
Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

Components (Fig. 1)

- 1. Attachment Pin Smooth, 53 mm.
- 2. Screw Funnel (Included in L-651000).
- 3. Smooth Lock.
- 4. Release Unit.
- 5. Protective Sleeve.
- 6. Push Button.
- 7. Dust Cap.
- 8. Cross Driver (Included in L-680000).

NOTES

The device is intended to connect and release a prosthetic system that replaces a missing lower limb.
Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.
The device is for single patient use.

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USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

The device is waterproof.
A Waterproof device can be used in a wet or humid environment and submerged up to 3-meter-deep water for a maximum of 1 hour.

It can tolerate contact with: Salt water, chlorinated water, perspiration, urine, and mild soaps.

It can also tolerate occasional exposure to sand, dust, and dirt.

Continuous exposure is not allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water after exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

MAINTENANCE

Unscrew the Release Unit, blow with compressed air alternately between the Pin hole and the hole for the Release Unit.

Caution: Do not use lubricant or any cleaning material. Only use air.

Regular cleaning is necessary to prevent compromise on function of locking mechanism.

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REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

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