Instructions for Use

LivingSkin® fingers

LivingSkin®

EN		Instructions for Use	3
DE		Gebrauchsanweisung	4
FR		Notice d'utilisation	6
ES		Instrucciones para el uso	8
IT		Istruzioni per l'uso	10
NO		Bruksanvisning	12
DA		Brugsanvisning	14
SV		Bruksanvisning	16
EL		Οδηγίες Χρήσης	18
FI		Käyttöohjeet	20
NL		Gebruiksaanwijzing	22
PT		Instruções de Utilização	24
PL		Instrukcja użytkowania	26
TR		Kullanım Talimatları	27
RU		Инструкция по использованию	29
JA		取扱説明書	31
ZH		中文说明书	33
KO		사용 설명서	35
CS		Návod k použití	37
SL		Indikácie pre použitie	39
RO		Indicații de utilizare	40
HR		Upute za uporabu	42
HU		Használati javallat	44
BG		Указания за употреба	46
SL		Indikácie pre použitie	48

INTENDED USE

The device is intended to impart some of the aesthetic and supporting functions of a missing limb. Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional. The device must be fitted and adjusted by a healthcare professional. It must be altered or repaired by the manufacturer, Össur.

INDICATIONS FOR USE

- Upper limb loss, amputation, or deficiency
- Mid tarsal or distal level lower limb loss, amputation, or deficiency

No known contraindications.

GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

The device is for single patient use.

The device should not be used for extended periods of time and removed during long periods of rest, e.g., sleep.

Only use water-based lubricants, oil-based lubricants will damage the device.

Do not expose to high temperatures or to flames.

Never wear the device directly over an open wound.

USAGE

Device Application

Apply a thin layer of water-based lubricant on the appropriate area.

Gently push or pull the device into place, making sure all air has escaped.

- For a finger device, pinch the skin at the edge of the device to create a channel for air to enter or escape.
- For a hand device, secure it between your knees to make the application easier. Do not tug or pull the wrist section.

Note: Pulling the device from one point can tear the silicone or cause stretching.

Device Removal

Pinch the open end, allowing air to enter and break the suction. Slowly ease the device off. If suction is not broken, pinch the open end again until air enters.

Water can also be used to help remove it by holding it under a faucet, pinching the open area and letting warm water enter.

To remove from a hard socket, insert a finger or wiggle while gently easing it off.

Note: Pulling the device from one point can tear the silicone or cause stretching.

- For a hand device, never remove it by pulling the fingers.

Using the Armature

The device can be fitted with an armature, metal wires that allow positioning.

- If armatures are included in the finger(s) of your device, only bend at the knuckle or finger joint closest to the palm. Never bend the armature at the fingernail, as this can make the armature puncture the surface. If an armature punctures the exterior of a device, contact your healthcare professional to schedule a repair.

Note: Do not constantly use the armatures for positioning. Frequently bending the wires will cause them to break. Broken wires cannot be repaired.

Nails

Acrylic fingers or toenails may be polished with standard nail polish. Only use non-acetone nail polish remover.

Note: Do not polish silicone nails or use gel manicures.

Prosthetic Adhesive

- If required, prosthetic adhesive can be used to aid suspension.

- Check for an allergic reaction to the adhesive on the skin before use. Sensitive formulas are available should skin react allergically.
- Apply the adhesive both to the proximal edge of the device and the skin. Let dry. Once contact is made, a strong bond will be created.
- To remove it, wet a cotton ball with the remover. Roll the cotton ball around the edge of the device and work it off the skin.
- If remover is not available, wet a cotton ball with a mild soap.

Cleaning and care

Clean the device daily after use and before first use.

- Clean the device inside and out using warm water and degreasing dish soap with no additives, e.g. no aloe vera or vitamin E.
- Clean the device regularly with 70 - 90% isopropyl alcohol to help with disinfection.
- A small soft nail brush may be used with extreme care.

If your skin requires oil or lotion to be applied, wait at least 30 minutes for it to be completely absorbed before using the device.

Note: Body perspiration contains oils that can build up and damage the prosthesis if not removed by proper regular cleaning.

Substances that can stain the device should not come in contact with it.

For example

- Unwashed blue jeans and other clothing.
- Chronic exposure to cigar and cigarette smoke.
- Rings and other jewelry that is not made of silver and gold.

If your device is stained, quickly wash it with mild soap and water, then rub it with isopropyl alcohol.

If the stain cannot be removed or the device is damaged in any way, contact your healthcare professional.

STORAGE

Do not store your device in an air-tight container, moisture can breed bacteria and mold.

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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