- Position the device with the selected size. Press the lock into the collar until secured (Fig. 3). Verify by attempting to readjust.
- Slide the Back Panel behind the patient’s neck (Fig. 4).
**Note:** Long hair should be placed outside of the device.
- Position chin support and center it. Tighten the device and attach the hook-and-loop strap (Fig. 5).

#### USAGE

##### Cleaning and care

- Wipe with a damp cloth and mild soap.
- Air dry.

**Note:** Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.
**Note:** Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

#### DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

#### LIABILITY

Osúr does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

#### DEUTSCH

#### SYMBOLE

Conditions requiring gross immobilization of the cervical spine. This may include:

- C spine precaution for trauma patients
- Immobilization for pre and post c-spine surgery
- Other conditions requiring gross immobilization of the mid-cervical spine

#### ENGLISH

#### SYMBOLS

#### INTENDED USE

The device is intended to provide gross immobilization to the cervical spine
The device must be fitted and adjusted by a healthcare professional.
X-ray and CT lucent.

#### INDICATIONS FOR USE

Conditions requiring gross immobilization of the cervical spine. This may include:

- C spine precaution for trauma patients
- Immobilization for pre and post c-spine surgery
- Other conditions requiring gross immobilization of the mid-cervical spine

#### Contraindications

- Patients with a compromised airway or known spinal deformities such as ankylosing spondylitis.
- Patients with penetrating trauma injuries.

#### Warnings and Cautions:

**Warning:** If an unstable fracture is suspected or unknown, with or without a sustained trauma, ensure additional spinal precautions are implemented to immobilize the spine.

**Warning:** Use of a cervical collar may increase intracranial pressure (ICP) through jugular venous compression.

**Warning:** Use of a cervical collar may increase complexity of airway management.

**Warning:** Cervical spine immobilization, including use of a cervical collar, has been associated with:

- Impaired respiratory effort and forced expiratory volume
- Pneumonia
- Aspiration
- Worsening of existing cervical spine injury
- Severe neurological deterioration in patients with ankylosing spondylitis
- Triggering of non-compliance or agitation
- Delay to definitive treatment
- Impaired physical examination/secondary survey

**Caution:** Cervical spine immobilization, including use of a cervical collar, has been associated with:

- Increased pain and discomfort, which may lead to increased spinal movement
- Impaired swallowing

**Caution:** Perfumes and heavy cleaning agents should not be used on or under the collar, as they may compromise the integrity of materials.

**Caution:** Use of a cervical collar for cervical spine immobilization is not recommended in situations where patients are awake, alert, not intubated, without neck tenderness or pain and without abnormal sensory or motor findings on examination.

**Caution:** Regular cleaning of the collar, pads and the skin beneath along with inspection of the skin for any signs of irritation is required to reduce the risk of skin ulceration. Bedridden patients are at an increased risk for skin ulceration.

#### Precautions:

- At least two persons are needed for initial collar placement: one to maintain the patient’s head and neck in proper alignment, the second to fit the collar.
- Collar removal or any adjustments to the collar should be done with physician permission only and according to the physician’s instructions.
- Unless otherwise specified by the physician, the patient should not remove the collar except to wash under it and change the pads.
- The patient will need a second person to help when cleaning the collar.

These instructions do not supersede hospital protocol and/or direct orders of the patient’s healthcare professional.

#### GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

The patient should immediately contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

#### FITTING INSTRUCTIONS

##### Device Application

Ensure that correct spinal protocol is followed.

- Maintain neutral alignment. Measure from the bottom of the chin to the top of the shoulder at the neck with your fingers (Fig. 1).
- Position your lowest finger on the sizing line. Select the size closest to the height of your fingers (Fig. 2).

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