## ENGLISH

**MD** Medical Device

### INTENDED USE

The device is intended as a connection component of a prosthetic system that replaces a missing lower limb.  
Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.  
The device must be fitted and adjusted by a healthcare professional.

Indications For Use and Target Patient Population  
• Lower limb amputation and/or congenital deficiency  
• No known contraindications

The weight limit for the device is 45 kg.

The device is for high impact use, e.g., walking and occasional running.

### GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

**Warning:** If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

**Warning:** Only use OSSUR replacement parts for the device for ensured durability. The device is for single patient use.

### ASSEMBLY INSTRUCTIONS

#### Components:

- Aluminum 4-Hole 774 (3)
- Screws M5x16
- Silicone Seal (5)

Note: Use with L-780000 Fabrication Tooling Kit (reusable)

Note: When fabrication is complete, insert the following locking component (include Fabrication Ring needed in the fabrication process)  
– L-721000 Icelock ratchet 721

#### Lamination Instructions:

1. Flatten distal end of Plaster Cast (8) to center Steel Guide (6).
2. Attach with double-sided tape or two nails maintaining original pin alignment.
3. Apply inner PVA bag and tie it around "neck" of Steel Guide (6).
4. Trim bag around string and place Silicone Seal (5) over "neck" of Steel Guide (6), onto PVA bag.
5. Screw 4-Hole Adapter (3) onto Lock Dummy (4).
6. Choose appropriate length of Release Dummy (7) and screw it firmly into Lock Dummy (4).
7. Screw Pin Dummy Bolt (2) into shorter Pin Dummy (1) making sure that hexagon hole on Pin Dummy Bolt stays visible.
8. Place the components onto Steel Guide (6) and secure them with Pin Dummy and Bolt (1;2) (make sure that O-ring on top of Lock Dummy is in place).
- Note:** Be sure to tighten (1-2 Nm) so that resin cannot leak in between components.
9. Apply release agent to components (A) to enable easy removal after lamination (I-400300 Silicone Adhesive is recommended).
10. Apply clay into hexagon hole on Release Dummy (7) and through hole on Pin Dummy (C).
11. Tie-up material into groove on 4-Hole Adapter (3) and over components.
- Warning:** Tie at least two layers of fibers in groove and over the distal part of adapter.
12. Apply outer PVA bag and pour resin.
13. When curing has finished, grind down to Release Dummy (7) and remove.
14. Remove socket from plaster cast.
15. Remove Bolt (2) in Pin Dummy (1) from inside of socket.
16. Remove Steel Guide (6) and Silicone Seal (5).
17. Use Hex Driver (9) and socket wrench to release Lock Dummy (4).
18. Remove Pin Dummy (1).
19. Grind down to spacers on 4-Hole Adapter (3) and remove all four silicone sealings.

**Warning:** Do not grind any further since that reduces strength of connection between socket and adapter. Insert appropriate locking component according to instructions enclosed.


**Warning:** For secure fastening use medium strength threadlocker and torque twice to 6 Nm.

### USAGE

Cleaning and care  
Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

### Environmental Conditions

The device is Weatherproof.  
A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e.g., rain), no submersion is allowed.  
No contact with salt water or chlorinated water is allowed.

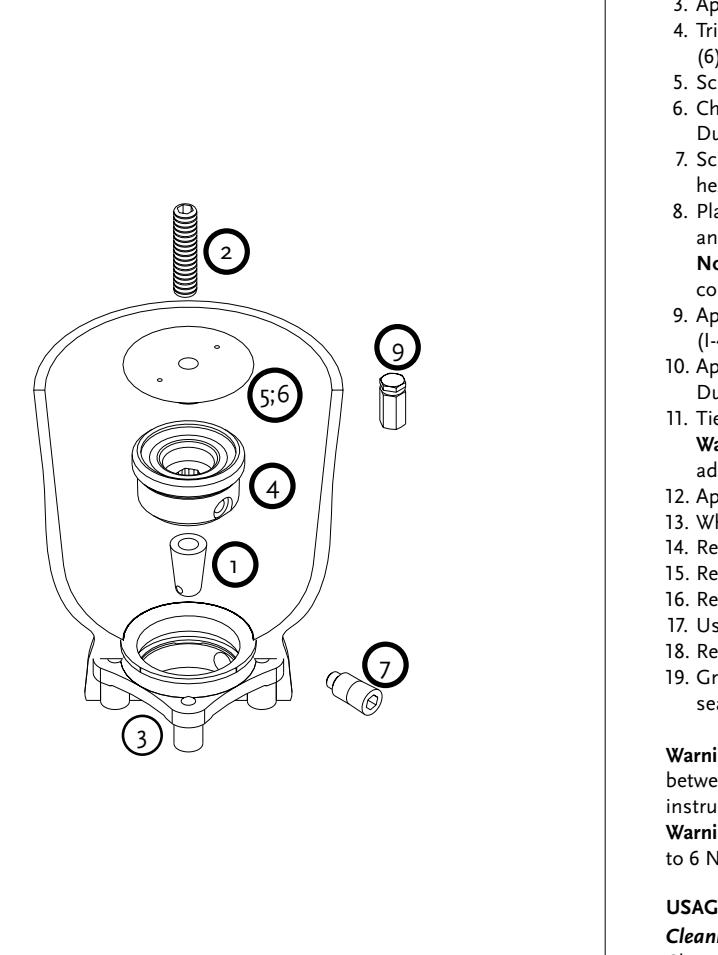Dry with a cloth after contact with fresh water or humidity.  
Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

### MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

### REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

### DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

### LIABILITY

OSSUR does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

### Compliance

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

**ISO 10328 - P3 - 45 kg \*)**

\* Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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1. Flaten distal end of Plaster Cast (8) to center Steel Guide (6).
2. Attach with double-sided tape or two nails maintaining original pin alignment.
3. Apply inner PVA bag and tie it around "neck" of Steel Guide (6).
4. Trim bag around string and place Silicone Seal (5) over "neck" of Steel Guide (6), onto PVA bag.
5. Screw 4-Hole Adapter (3) onto Lock Dummy (4).
6. Choose appropriate length of Release Dummy (7) and screw it firmly into Lock Dummy (4).
7. Screw Pin Dummy Bolt (2) into shorter Pin Dummy (1) making sure that hexagon hole on Pin Dummy Bolt stays visible.
8. Place the components onto Steel Guide (6) and secure them with Pin Dummy and Bolt (1;2) (make sure that O-ring on top of Lock Dummy is in place).

**Note:** Be sure to tighten (1-2 Nm) so that resin cannot leak in between components.

9. Apply release agent to components (A) to enable easy removal after lamination (I-400300 Silicone Adhesive is recommended).

10. Apply clay into hexagon hole on Release Dummy (7) and through hole on Pin Dummy (C).

11. Tie-up material into groove on 4-Hole Adapter (3) and over components.

**Warning:** Tie at least two layers of fibers in groove and over the distal part of adapter.

12. Apply outer PVA bag and pour resin.

13. When curing has finished, grind down to Release Dummy (7) and remove.

14. Remove socket from plaster cast.

15. Remove Bolt (2) in Pin Dummy (1) from inside of socket.

16. Remove Steel Guide (6) and Silicone Seal (5).

17. Use Hex Driver (9) and socket wrench to release Lock Dummy (4).

18. Remove Pin Dummy (1).

19. Grind down to spacers on 4-Hole Adapter (3) and remove all four silicone sealings.

**Warning:** Do not grind any further since that reduces strength of connection between socket and adapter. Insert appropriate locking component according to instructions enclosed.

**Warning:** For secure fastening use medium strength threadlocker and torque twice to 6 Nm.

### USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

### Environmental Conditions

The device is Weatherproof.  
A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e.g., rain), no submersion is allowed.  
No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.  
Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

### MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

### REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

### DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

### LIABILITY

OSSUR does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

### Compliance

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

**ISO 10328 - P3 - 45 kg \*)**

\* Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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