ENGLISH

MD Medical Device

INTENDED USE

The device is intended to immobilize the shoulder and/or elbow

Indications for use

For shoulder conditions that may benefit from shoulder immobilization and/or abduction, such as:

- Pre or post-operative
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

No known contraindications.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

Never wear the device directly over an open wound. Inspect the skin under the device's contact area before and after each use.

The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

USAGE

Device Application

- Unfasten the shoulder strap and open the front panel of the device.
- Place the forearm into the device at a 90° angle, seating the elbow snugly in the corner pocket of the sling (**Fig. 1**).
- Place the thumb through the thumb loop (**Fig. 2**).
- Close the buttons to secure the front panel (**Fig. 3**).
- Place the shoulder strap around the neck over the opposite shoulder and fasten it to the device via the D-ring (**Fig. 4**). Adjust the strap length for proper fit. Strap can be trimmed to length if needed.
- Place the abduction pillow between the body and the device at the waistline of the affected side. Slide the pillow to the desired position (**Fig. 5**).
- Attach the sling to the outside of the pillow using the hook and loop strips (**Fig. 6**).
- Pull the abduction pillow strap around the back and fasten it to the abduction pillow via the D-ring located on the front of the pillow. Strap can be trimmed to length if needed (**Fig. 7**).
- Re-adjust the shoulder strap from the front or back, if necessary.
- Position the shoulder pad, where it is most comfortable.

Device removal

Unfasten the abduction pillow strap first. Remove the shoulder strap and open the front panel to remove the arm from the device.

Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry

Note: Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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©Copyright Össur 2021