ENGLISH

Medical Device

INTENDED USE

The device is intended to immobilize the shoulder and/or elbow

Indications for use

For shoulder conditions that may benefit from immobilizing the shoulder, such as:

- Pre or post-operative
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

No known contraindications.

Warnings and Cautions:

- Care should be taken not to overtighten the device.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

Never wear the device directly over an open wound. Inspect the skin under the device's contact area before and after each use.

The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

USAGE

Device Application

1. Unfasten the shoulder strap and open the front panel of the device.
2. Place the forearm into the device at a 90° angle, seating the elbow snugly in the corner pocket.
3. Place the thumb through the thumb loop.
4. Place the shoulder strap around the neck over the opposite shoulder and fasten it to the device via the D-ring. Adjust the strap length for proper fit.
5. Apply the waist strap to the device, pull the strap around the back and feed it through the D-ring. This will prevent the device from rotating.

Device removal

Unfasten the waist strap, the shoulder strap and open the front panel to remove the arm out of the device.

Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry

Note: Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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©Copyright Össur 2021 IFU 0590 1488_001