ENGLISH

Medical Device

INTENDED USE

The device is intended to immobilize the shoulder and/or elbow

The device must be fitted and adjusted by a healthcare professional.

Indications for use

For shoulder conditions that may benefit from immobilizing the shoulder, such as:

- Pre or post-operative
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

No known contraindications.

GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

The patient should immediately contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

FITTING INSTRUCTIONS

Device Application

1. The wrist cuff will be supplied attached to the abdominal belt. Place injured side hand into the wrist cuff and secure the waistband around the body using the hook-and-loop fastening. The humeral cuff should be positioned at the side of the injured arm. Position the injured arm internally rotated in a comfortable position.
2. Place the shoulder strap over the healthy shoulder, attach to the wrist cuff and adjust the strap to the appropriate length.
3. Adjust the waist belt and tighten.
4. Adjust the position and angle of the arm of the injured shoulder.
5. Fasten the humeral cuff just above the flexed elbow with the attached hook-and-loop.
6. Attach the separate hook-and-loop strip to secure the wrist cuff to the abdominal belt in order to maintain the appropriate position.

USAGE

Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry, do not expose to direct heat.

Note: Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry. Do not squeeze or wring out the device.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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