
Instructions for Use


---

RELAX NIGHT CARE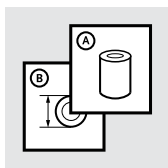

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	6
FR	Notice d'utilisation	8
ES	Instrucciones para el uso	10
IT	Istruzioni per l'uso	12
NO	Bruksanvisning	14
DA	Brugsanvisning	16
SV	Bruksanvisning	18
EL	Οδηγίες Χρήσης	20
FI	Käyttöohjeet	22
NL	Gebruiksaanwijzing	24
PT	Instruções de Utilização	26
PL	Instrukcja użytkowania	28
CS	Návod k použití	30
TR	Kullanım Talimatları	32
RU	Инструкция по использованию	34
JA	取扱説明書	36
ZH	中文说明书	38
KO	사용 설명서	40

1

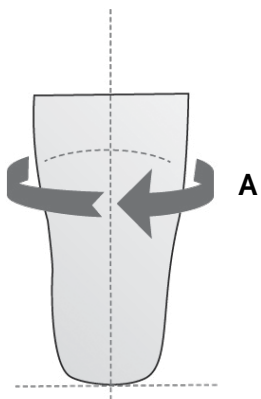# ENGLISH

---

Medical Device

## DESCRIPTION

The device is a sock made of knitted fabrics with integrated conductive filaments which show electromagnetic shielding properties and avoid an electrostatic charge.

## INTENDED USE

The device is intended to aid in the reduction of phantom pain. Suitability of the device for the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

### *Indications for use*

- Lower limb amputation and/or congenital deficiency
- No known contraindications

## GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

**Warning:** If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

The device is for single patient use.

### **Do not use the device in case of:**

- Skin necrosis
- Recent skin grafts (less than 4 weeks since grafting and before epithelialisation starts)

**Caution:** Do not wear the device with a prosthesis.

## DEVICE SELECTION

**Caution:** The wrong size can lead to discomfort, skin irritations, or a less reliable function.

Length 35 cm		
Size	Item number	Circumference A
S	188000S	20 - 26 cm
M	188000M	26 - 33 cm
L	188000L	33 - 42 cm
XL	18800XL	42 - 52 cm
XXL	1880XXL	52 - 62 cm

## USAGE

**Caution:** Open wounds or damaged skin should be covered with a bandage or other appropriate covering to prevent direct contact between the wound and device.

Make sure that the device does not have wrinkles.

### ***Cleaning and care***

The device must be washed carefully before first use.

Daily cleaning of the residual limb and the device is essential. Use of a mild, pH balanced, 100% fragrance- and dye-free liquid soap is recommended. After washing rinse the device several times.

**Note:** Only wash by hand in warm water. Do not wash in a washing machine. Do not use bleach.

Hang device up to dry.

Do not leave to dry on heaters or in direct sunlight.

Do not iron. Do not dry clean.

**Caution:** Skin health should be monitored daily. In case of abnormal symptoms, discontinue use of the device and contact the healthcare professional immediately.

**Caution:** Be aware that common household or bath products may cause or contribute to skin irritation, e.g., soaps, deodorants, perfumes, abrasive cleaners, aerosol- or alcohol sprays.

## REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

## DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

## LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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