Instructions for Use

SACH FOOT ADAPTER JUNIOR
SACH FOOT ADAPTER AL
SACH FOOT ADAPTER TI

EN		Instructions for Use	3
DE		Gebrauchsanweisung	7
FR		Notice d'utilisation	11
ES		Instrucciones para el uso	15
IT		Istruzioni per l'uso	19
NO		Bruksanvisning	23
DA		Brugsanvisning	27
SV		Bruksanvisning	31
EL		Οδηγίες Χρήσης	35
FI		Käyttöohjeet	39
NL		Gebruiksaanwijzing	43
PT		Instruções de Utilização	47
PL		Instrukcja użytkowania	51
TR		Kullanım Talimatları	55
RU		Инструкция по использованию	59
JA		取扱説明書	63
ZH		中文说明书	67
KO		사용 설명서	71

ENGLISH

MD

Medical Device

INTENDED USE

The device is intended as a connection component of a prosthetic system that replaces a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

For more detailed device information, see **Table 1**.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

For weight limit and impact level, see **Table 2**.

GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

Warning: Only use Össur replacement parts for the device for ensured durability.

The device is for single patient use.

ASSEMBLY INSTRUCTIONS

Warning: For secure fastening use medium strength threadlocker and apply rated torque twice, see **Table 1**.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

See which environmental conditions apply to the device in **Table 2**.

A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e. g., rain), no submersion is allowed.

No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

SPECIFICATIONS

Table 1 - Device Specifications

Part #	Description	Material	Torque Values	Environmental Conditions
			Mount Screw	
AJ-833100	8 mm Sach Foot Adapter	Aluminum & Stainless Steel	20 Nm	Weatherproof
A-833100	Sach Foot Adapter		30 Nm	
A-835100	Sach Foot Adapter	Titanium	30 Nm	

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

Table 2 - Usage Specifications

Part #	Impact Level	Weight Limit	ISO 10328 - Label Text
AJ-833100	The device is for low to high impact use, e.g., walking and occasional running.	45 kg	ISO 10328 - P3 - 45 kg
A-833100	The device is for low to moderate impact use, e.g., walking.	100 kg	ISO 10328 - P5 - 100 kg
A-835100	The device is for low to high impact use, e.g., walking and occasional running.	166 kg	ISO 10328 - P7 - 166 kg

ISO 10328 - "P" - "m"kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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