Instructions for Use

---

SACH FOOT ADAPTER JUNIOR  
SACH FOOT ADAPTER AL  
SACH FOOT ADAPTER TI

EN		Instructions for Use	3
DE		Gebrauchsanweisung	7
FR		Notice d'utilisation	12
ES		Instrucciones para el uso	16
IT		Istruzioni per l'uso	20
NO		Bruksanvisning	25
DA		Bruksanvisning	29
SV		Bruksanvisning	33
EL		Οδηγίες Χρήσης	37
FI		Käyttöohjeet	41
NL		Gebruiksaanwijzing	45
PT		Instruções de Utilização	50
PL		Instrukcja użytkowania	54
CS		Návod k použití	59
TR		Kullanım Talimatları	63
RU		Инструкция по использованию	67
JA		取扱説明書	72
ZH		中文说明书	76
KO		사용 설명서	80

# ENGLISH

---

Medical Device

## INTENDED USE

The device is intended as a connection component of a prosthetic system that replaces a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

For more detailed device information, see **Table 1**.

## *Indications For Use and Target Patient Population*

- Lower limb amputation and/or congenital deficiency
- No known contraindications

For weight limit and impact level, see **Table 2**.

## GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

**Warning:** If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

**Warning:** Only use Össur replacement parts for the device for ensured durability

The device is for single patient use.

## **ASSEMBLY INSTRUCTIONS**

**Warning:** For secure fastening use medium strength threadlocker and apply rated torque twice, see **Table 1**.

## **USAGE**

### **Cleaning and care**

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

### **Environmental Conditions**

See which environmental conditions apply to the device in **Table 2**.

A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e. g., rain), no submersion is allowed.

No contact with salt water or chlorinated water is allowed.

A Waterproof device can be used in a wet or humid environment and submerged in up to 1-meter-deep fresh water for a maximum of 30 minutes. No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

## **MAINTENANCE**

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

## SPECIFICATIONS

Table 1 - Device Specifications

Part #	Description	Material	Torque Values	Environmental Conditions
			Mount Screw	
AJ-833100	8 mm Sach Foot Adapter	Aluminum & Stainless Steel	20 Nm	Weatherproof
A-833100	Sach Foot Adapter		30 Nm	
A-835100	Sach Foot Adapter	Titanium	30 Nm	Waterproof

### REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

### DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

### LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

### ***Compliance***

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

**Table 2 - Usage Specifications**

Part #	Impact Level	Weight Limit	ISO 10328 - Label Text
AJ-833100	The device is for low to high impact use, e.g., walking and occasional running.	45 kg	ISO 10328 - P3 - 45 kg
A-833100	The device is for low to moderate impact use, e.g., walking.	100 kg	ISO 10328 - P5 - 100 kg
A-835100	The device is for low to high impact use, e.g., walking and occasional running.	166 kg	ISO 10328 - P7 - 166 kg

**ISO 10328 - “P” - “m”kg \*)**

\*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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**Össur UK Ltd**