## ENGLISH

### DESCRIPTION OF SYMBOLS

**MD** Medical Device

### PRODUCT DESCRIPTION

The Formfit® Shoulder Brace with Airplane Kit is a textile arm sling with a metal frame and waist attachment component.

### INDICATIONS FOR USE

Rotator cuff repairs, Bankart repairs, capsular shifts, glenohumeral dislocations/subluxation, soft tissue, repairs/strains, shoulder instabilities/injuries, and post-surgical rehabilitation.

### CONTRAINDICATIONS

None known.

### INTENDED USE

The Formfit® Shoulder Brace is a device intended to provide the option of positioning the shoulder in immobilization while in abduction.

### GENERAL SAFETY INSTRUCTIONS / WARNINGS AND PRECAUTIONS

**Caution:** This immobilizer is supportive only and is not guaranteed to prevent injury. If any problems occur related to use of product please contact your certified medical professional.

**Warning:** In cases of functional change or functional loss, the patient should stop using the device and contact a healthcare professional.

**Warning:** If the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

### REQUIRED COMPONENTS

(Figure 1)

- A. Hook
- B. Arm Rest
- C. Plastic film
- D. Waistband
- E. Waist strap
- F. Arm Bar

G. Screws, 3/16" bolts (4 each)

Not included but required for assembly: 3/8" (10mm) wrench

### ASSEMBLY / FITTING INSTRUCTIONS

#### Setting up 90° Arm Bar

- Remove plastic film temporarily to remove waist strap, set aside (Figure 2a). Replace plastic film (Figure 2b).
- Lift the rectangular hook to open protective foam and access screw mounts. Set hook aside. Using 3/8" (10mm) wrench (not included), loosen and remove bolts (Figure 3).
- Remove armrest from waistband and set aside.

- Position waistband as shown in Figure 4a/b, short side towards you. Affix slotted side of arm bar to the waist band as shown for desired Left (Figure 4a) or Right (Figure 4b) configuration.
- Loosely tighten bolts so that the arm bar is attached, then slide arm bar to desired height. Tighten bolts to firmly secure positioning.
- Use hook material to secure the foam covering in place (Figure 5).
- Carefully affix arm rest to arm bar with additional screws and bolts, so that the longer end of the armrest is facing toward the shorter end of the waistband (anterior) (Figure 6).

#### Apply waistband and shoulder sling

- Mold waistband to desired curvature (Figure 7).
- Re-attach waist strap on the waistband (double sided hook will secure the posterior of waist strap, allowing the buckle to orient anterior). Trim strap and secure waist strap (Figure 8).
- Place arm in sling with elbow at 90° (Figure 9a) and close arm sling (Figure 9b).
- Rest arm and sling on top of the arm rest (Figure 10).
- Optional: Adjust for shoulder internal or external rotation by lifting or depressing the arm rest.
- Remove plastic film to secure sling to arm rest hook.
- Adjust for shoulder horizontal abduction / adduction by positioning the entire unit more posterior or anterior relative to the patient.

### CLEANING AND CARE

The brace can be hand washed using mild detergent and cold water. Do not wash in washing machine. Do not use bleach.

### REPORT A SERIOUS INCIDENT

**Important notice to users and/or patients established in Europe:**

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

### LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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