ENGLISH

DESCRIPTION OF SYMBOLS

 Medical Device

INTENDED USE

The device is intended to relieve pain due to epicondylitis.

INDICATIONS FOR USE

Epicondylitis including Tennis Elbow, Golfers elbow, and other overuse injuries.

CONTRAINDICATIONS

None known.

WARNING AND CAUTIONS

Caution: Use caution when applying hot therapy as gel may become very hot.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

Warning: Patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions;
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

FITTING INSTRUCTIONS

1. Loop strap through the plastic ring and slide up forearm.
2. Position gel over top of the forearm. Use a barrier between your skin and gel pad.
3. Tighten strap by wrapping snugly but do not over tighten.
4. For cold therapy, cool the gel pads in a freezer (20-60 min).
5. For hot therapy, heat the gel pad in warm water or in a microwave oven (5 second intervals at lowest setting) until it is warm but not too hot to hold.

Accessories and Replacement Parts

Please refer to the Össur Catalog for a list of available replacement parts or accessories.

CLEANING AND CARE

Remove gel pad when cleaning the device.

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry, do not place near direct heat.

Note: Do not machine wash, tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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