ENGLISH

SYMBOLS

Medical Device

INTENDED USE

The device is intended to restrict motion of the wrist and/or thumb

The device is intended to restrict motion of the wrist and/or thumb

The device is intended to restrict motion of the wrist and/or thumb

Indications for use
For conditions that may benefit from restricting motion of the wrist and thumb, such as:

- DeQuervain’s Tendonitis
- Gamekeeper’s Thumb
- Torn Collateral Ligaments
- Post cast healing
- Other selected soft tissue injuries

No known contraindications.

The device is intended to restrict motion of the wrist and/or thumb

Warnings and Cautions:

- Care should be taken not to overtighten the device.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference. Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

USAGE

Device Application

- Undo all straps.
- Position the device straight down the thumb side of the forearm.
- Insert the thumb into the thumb guide and fasten. The thumb end joint should remain freely movable.
- Fasten remaining straps, starting with the one closest to the thumb and readjust for comfort.

The device is intended to restrict motion of the wrist and/or thumb

Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry, do not expose to direct heat.

Note: Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.

- Device used outside of recommended use condition, application, or environment.

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©Copyright Össur 2025-04-03