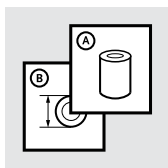
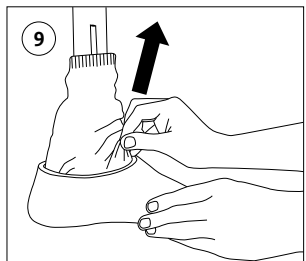
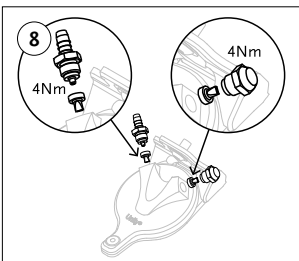
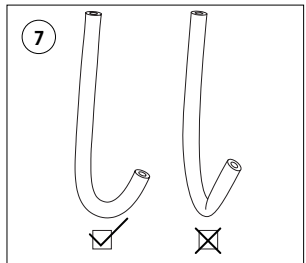
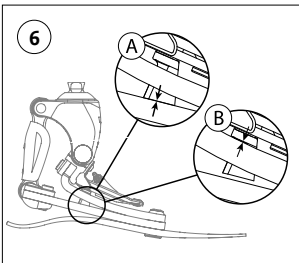
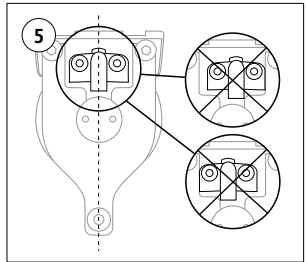
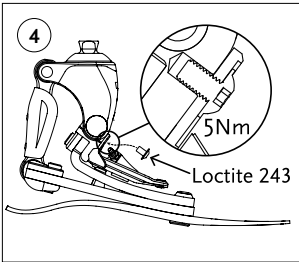
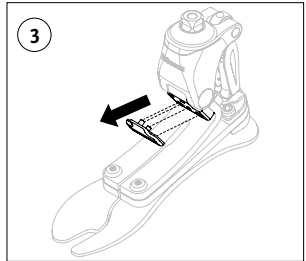
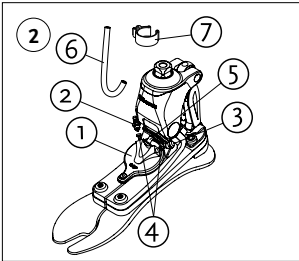
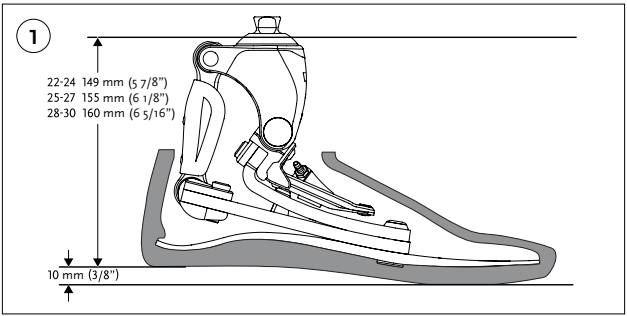
Instructions for Use

UNITY[®] FOR PRO-FLEX PIVOT

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Medical Device

PRODUCT DESCRIPTION

Unity for Pro-Flex Pivot is a mechanical vacuum pump intended for vacuum assisted prosthetic suspension. Suspension is obtained through the act of a pump which draws air out from the socket in each step creating an elevated vacuum inside the socket interface.

UNITY SYSTEM COMPONENTS (Fig. 2)

1. Membrane housing
2. Inlet port
3. Outlet port
4. Duckbill
5. Bolt
6. Tube
7. Tube clamp

INDICATIONS

- Lower limb amputation.
- Lower limb deficiency.

CONTRAINDICATIONS

None known.

INTENDED USE

The device is a single patient device designed for daily use. This device should only be fitted by a healthcare professional.

It is intended to be used with the Pro-Flex Pivot Foot, FST0 foot cover and a Unity Socket Valve Kit for connection to a suspension socket with an Icross[®] Seal-In Liner.

The function of Unity[®] is activated during gait, the weight placed on the prosthetic foot expands the vacuum pump reservoir that efficiently draws air out from the socket in each step and expels it into the atmosphere during swing as the reservoir compresses. This pumping mechanism creates a negative pressure at least 15 inHg inside the socket, resulting in a secure and reliable liner suspension.

GENERAL SAFETY INFORMATION

Inform the patient about everything in this document that is required for safe use of this device.

WARNING: In case of functional change or functional loss, the patient should stop using the device and contact a healthcare professional.

WARNING: If the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

ASSEMBLY

1. Visual check to make sure the bumper is properly aligned and centered (**Fig. 5**) to avoid noise from the device.

2. Remove the cover (**Fig. 3**) and attach the device to the prosthetic foot. Device placement depends on the foot size and category. To achieve optimum device placement, the bumper must rest on the middle Pro-Flex Pivot blade (**Fig. 6A**) or on the upper Pro-Flex Pivot blade (**Fig. 6B**).
3. Apply Loctite 243 and tighten the bolt to 5 Nm (**Fig. 4**).
4. Tube Connection
 - i. Connect the tube between the inlet port and the Unity valve on the close fitted socket.
 - ii. Cut off any excess tube, if necessary.
 - iii. Secure the tube to the prosthesis medially using the tube clamp to avoid damage during ambulation.

CAUTION: Ensure that Unity tubing is not compressed (**Fig. 7**) or folded by the application of cosmetic finishing (**Fig. 9**).

WARNING: A loose or dangling tube can create a tripping hazard. Secure the tube to the prosthesis in a permanent manner.

MAINTENANCE

The following components of the Unity System might need cleaning or replacing during the lifetime of the product.

- Duckbill valves
- Outlet Port
- Tube

NOTE: It is recommended to have the product inspected every 6 months by a healthcare professional.

NOTE: The duckbill valves must be oriented correctly as instructed in (**Fig. 8**) for correct function. Apply Loctite 243 and torque the outlet port and inlet port to 4 Nm.

Disassembling

Remove the tube, outlet port and inlet port to access the duckbill valves.

Reassembly

Assemble the parts in reverse order.

TROUBLESHOOTING GUIDE

In case the Unity system is not functioning properly check the following:

Problem	Possible Cause	What to do
Noise from the Unity product (clicking, tapping, squeaking, scratching, whistling).	The Unity has become loose.	Align the Unity accordingly and reattach. Apply Loctite 243 and torque the screw to 5 Nm (Fig. 5 & Fig. 6).
	Sand is between bumper and blade.	Clean according to cleaning instructions.
	Sand and/or debris is between the blades.	Clean according to cleaning instructions.
	Duckbill is worn, torn, or dirty.	Change both of the duckbills (inlet and outlet port).
	The filter is dirty.	Change outlet port.

Problem	Possible Cause	What to do
Lost vacuum.	Seal-In is leaking.	Visually check the seal and/or liner for damage, change if necessary. Use the syringe to check if the vacuum is holding.
	Unity Socket Valve is leaking.	To check for this, seal the valve on the inside of the socket with a silicone pad and use a syringe to check if the valve works as intended.
	The tube is not fully inserted.	Make sure the tube is fully inserted onto the inlet port and the Unity Socket Valve.
	Duckbills are incorrectly installed.	Inspect if the duckbills are oriented in the correct direction and change it if needed (Fig. 8).
	The socket is leaking.	Use the provided syringe and the associated equipment to check the vacuum in the socket.
Noise and/or lost vacuum.	The yellow warning label has not been removed from the pyramid of the foot.	Remove the yellow warning label, torque according to specifications (see the adapter instructions for use).
	The female pyramid is incorrectly connected to the male foot pyramid.	Readjust, apply Loctite 243, and torque according to specifications (see the adapter instructions for use).
	The tube is leaking.	Change the tube and fit according to instructions.
	Inlet and/or outlet port have become loose.	Apply Loctite 243 and torque the inlet and/or outlet port to 4 Nm. Make sure the tube is correctly installed upon the inlet ports (Fig. 8).
	Spectra sock restricts the functional performance of the product.	Refit the Spectra sock and make sure it is fitted accordingly (Fig. 9).

CLEANING

Clean the components with a damp cloth and mild soap, isopropyl alcohol or Icross® Clean&Simple Lubricant spray. Rinse with a clean damp cloth and dry thoroughly.

CAUTION: Do not use solvents stronger than isopropyl alcohol as it might degrade the material or get through the valve and into the socket.

ENVIRONMENTAL CONDITIONS

The device is weatherproof.

A weatherproof device can tolerate light fresh water splashing (e. g. rain) but does not allow submersion. Dry thoroughly after contact with fresh water.

No salt water or chlorinated water allowed.

REPORT A SERIOUS INCIDENT

Important notice to users and/or patients established in Europe:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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