ENGLISH

SYMBOLS

Medical Device

Caution: This device contains natural rubber latex which may cause allergic reactions.

INTENDED USE

The device is intended to support the rib cage

Indications for use

Fractures and strains to the rib cage.

No known contraindications.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

USAGE

Device Application

1. Center the device around the rib cage.
2. Pull the two ends of the belt forward evenly and guide them one over the other.
3. When securing contact closure, ensure the device fits tightly enough to provide even, constant pressure without having any impact on the patient's breathing. For Female Rib Belts, the contour fits just below the bust line.

Device removal

Unfasten the belt ends from each other to remove the device.

Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry, do not expose to direct heat.

Note: Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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©Copyright Össur 2021