

## MIAMI TLSO 456

The Miami TLSO 456 from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to use pre assembled Miami TLSO brace offers the convenience of a one-size Universal or a uniquely-sized anatomical design. Compliance: Blue is You™ patient touch-points help send patients home with confidence.

**Adjustability:** The universal belt design allows for setting the belt length on the patient for a more accurate sizing experience. Heat moldable posterior panels allow a customised fit, while the modular design allows step-down support to an LSO.

**Comfort:** Featuring the Miami LSO™ Advanced Compression pulley system with lumbar compression you can feel, and an anatomically contoured design that better immobilises the spine while maximising patient comfort.

**Ease of Use:** Easily applied with over-the-shoulder straps, the Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



## INDICATIONS FOR USE

Conditions requiring gross immobilisation of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilisation
- Spinal Support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolithesis
- Spondylolysis

## FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/doffing (when compared to Miami Lumbar TLSO)
- Fully assembled for fitting with ease
- Accommodates heights 155–193 cm
- Step-down to an LSO as one progresses through rehabilitation
- The convenience of a one-size Universal or individualise with a uniquely-sized anatomical design


## CONTRAINDICATIONS

- Unstable, displaced fractures

## REQUIRED MEASUREMENTS

Size	Waist circumference*
	cm
Universal	66 - 177
S	66 - 86
M	86 - 107
L	107 - 127
XL	127 - 177

\*Belt extensions are required for waist circumferences larger than 127 cm

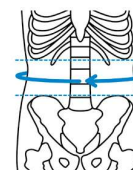

Measure circumference of the trunk at a level midway between the lowest ribs and the upper iliac crest. Standing.

## MIAMI TLSO 456

### ORDERING INFORMATION

#### PTE LORDOTIC

Size	Part#
Universal	MTLSO-2352
S	MTLSO-2112
M	MTLSO-2122
L	MTLSO-2132
XL	MTLSO-2142

#### PTE NON-LORDOTIC, SLIM

Size	Part#
S	MTLSO-1212
M	MTLSO-1222
L	MTLSO-1232
XL	MTLSO-1242

#### PTE NON-LORDOTIC

Size	Part#
S	MTLSO-2212
M	MTLSO-2222
L	MTLSO-2232
XL	MTLSO-2242

#### PTE LORDOTIC, SLIM

Size	Part#
Universal	MTLSO-1352
S	MTLSO-1112
M	MTLSO-1122
L	MTLSO-1132
XL	MTLSO-1142

### OTHER ACCESSORIES

Description	Part#
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-177 cm. Fits size XL.)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-177 cm. Fits size XL.)	MLSO-1EXT