

MIAMI TLSO™ 464

The Miami TLSO from Össur directly addresses the needs of patients and practitioners, simplifying preparation without compromising on fit or performance. This easy-to-use pre-assembled Miami TLSO brace offers three configurations in one. Compliance: Easily applied in over the shoulder or underarm configuration, donning and doffing is further simplified with the conveniently located quick release buckles. Adjustability: The universal belt designs allows for setting the belt length on the patient for a more accurate sizing experience. Comfort: Truly shaped to fit the anatomy, the anatomically contoured design better immobilises the spine while maximising patient comfort. Ease of Use: The Miami TLSO brace requires minimal preparation and is designed to provide optimal patient comfort.



INDICATIONS FOR USE

Conditions requiring gross immobilisation of the thoracic and lumbar spine. Such conditions may include:

- Post-surgical immobilisation
- Spinal Support TLSO attachment is up to T7
- Spinal stenosis
- Herniated disc
- Degenerative spinal fractures
- Spondylolithesis
- Spondylolysis

CONTRAINDICATIONS


- Unstable, displaced fractures

FEATURES AND BENEFITS

- Significantly superior comfort and ease of donning/doffing
- Fully assembled for fitting with ease
- Accommodates heights 155–193 cm
- Posterior thoracic extension option with ergonomic axillary strapping
- Step-down to an LSO as one progresses through rehabilitation
- One-touch telescoping anterior-thoracic extension option with single or double-hinge
- The convenience of a one-size Universal or individualise with a uniquely-sized anatomical design
- Improve comfort and immobilisation with multiple strapping configurations and removable anchors

REQUIRED MEASUREMENTS

Size	Waist circumference
	cm
Universal	66 - 177
S	66 - 86
M	86 - 107
L	107 - 127
XL	127 - 177

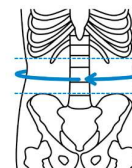

Measure circumference of the trunk at a level midway between the lowest ribs and the upper iliac crest. Standing.

MIAMI TLSO™ 464

ORDERING INFORMATION

ATE/PTE, LORDOTIC

Size	Part#
Universal	MTLSO-2351
S	MTLSO-2111
M	MTLSO-2121
L	MTLSO-2131
XL	MTLSO-2141

ATE/PTE, NON-LORDOTIC

Size	Part#
S	MTLSO-2211
M	MTLSO-2221
L	MTLSO-2231
XL	MTLSO-2241

ATE/PTE, LORDOTIC, SLIM

Size	Part#
Universal	MTLSO-1351
S	MTLSO-1111
M	MTLSO-1121
L	MTLSO-1131
XL	MTLSO-1141

ATE/PTE, NON-LORDOTIC, SLIM

Size	Part#
S	MTLSO-1211
M	MTLSO-1221
L	MTLSO-1231
XL	MTLSO-1241

OTHER ACCESSORIES

Description	Part#
5MM Hex Long Wrench	MTLSO-0004
Axillary strap accessory kit	MTLSO-0003
Quick fit tool	B-040100000
Belt extension, standard (Recommended for waist circumferences 127-177 cm. Fits size XL.)	MLSO-2EXT
Belt extension, slim (Recommended for waist circumferences 127-177 cm. Fits size XL.)	MLSO-1EXT