





# Proprio Foot®

Össur Logic  
User Guide

---

# 1 Download the app

Please note you need to be connected to the internet to sign up to Össur Logic.

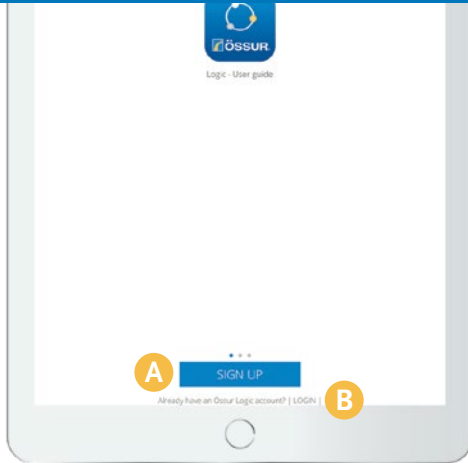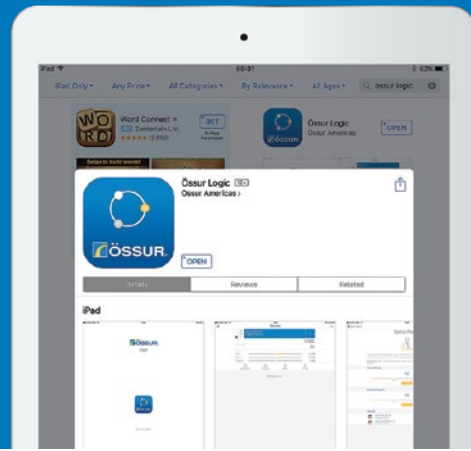

## 2 A Click 'sign up'



if you have never used Össur Logic app before (if you are already a registered user, please continue from 5A).

## B Click 'login'

to Össur logic with your email address and password.

## 3 A Proprio Foot® users

When you are prompted for your customer number, please skip this stage by clicking the skip button.

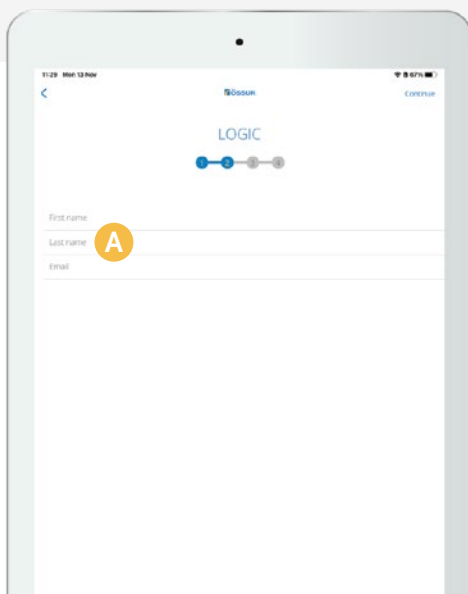
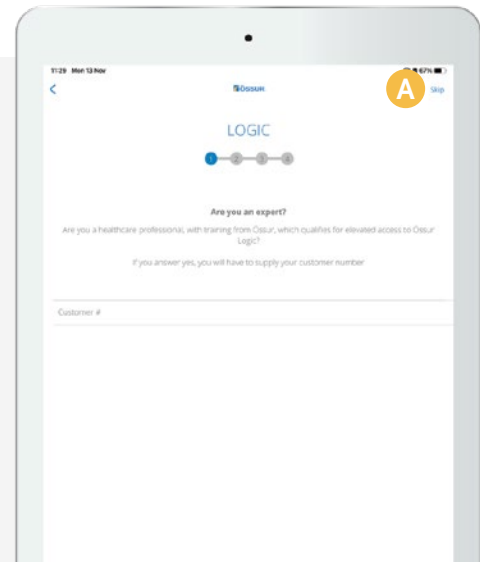


## 4 A Create New User login

Make your own login credentials and password.

Please note after signing up to Össur Logic, you will need to confirm your email address by clicking the link emailed to you.

**5 Click 'Add device'**  
to add a Proprio Foot.

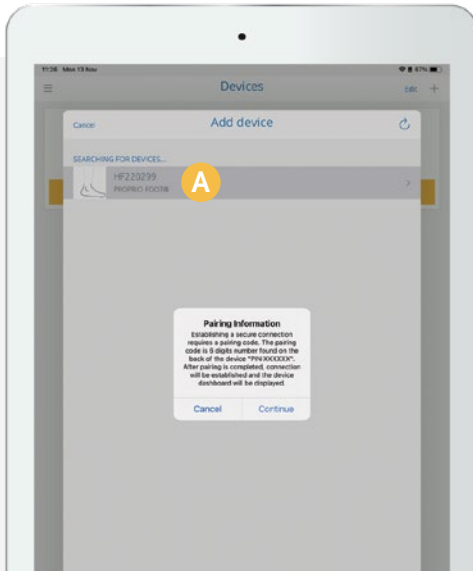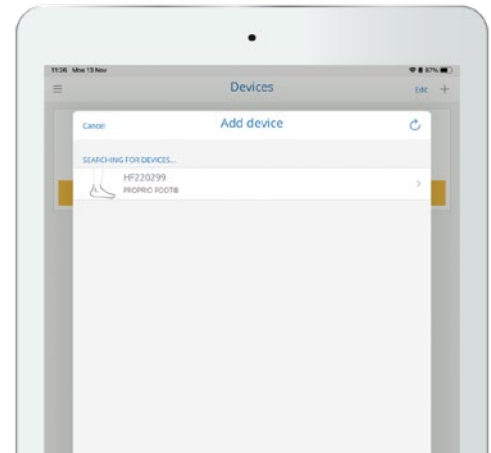

**6 A Connect with Proprio Foot**

Össur Logic will search for all the devices in range. Select your Proprio Foot Serial Number starting with HF (also shown on side of foot). It will then prompt you for a pairing code to connect to the foot.


**7 A Insert 'pairing code'**

This can be found on the back of the foot - an example is shown in the image on the right.




**8 Ready for use**

- A** Battery level
- B** Assign shoe profiles here
- C** Tap here to connect and disconnect
- D** Step count / usage information

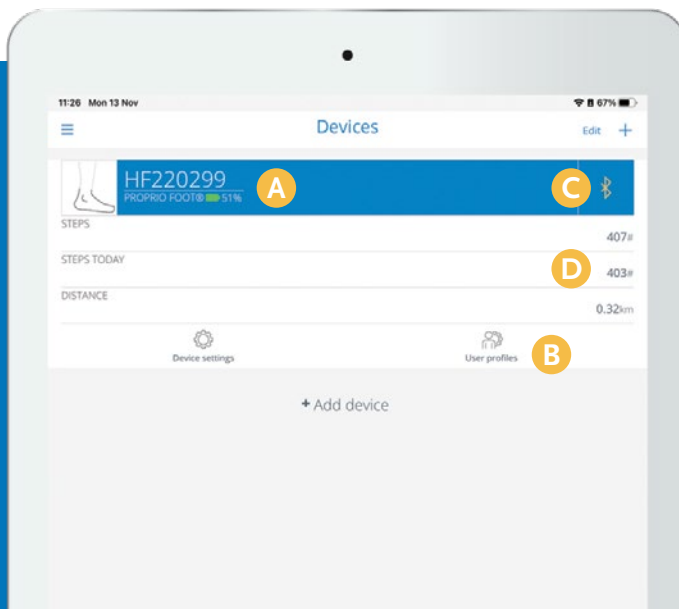




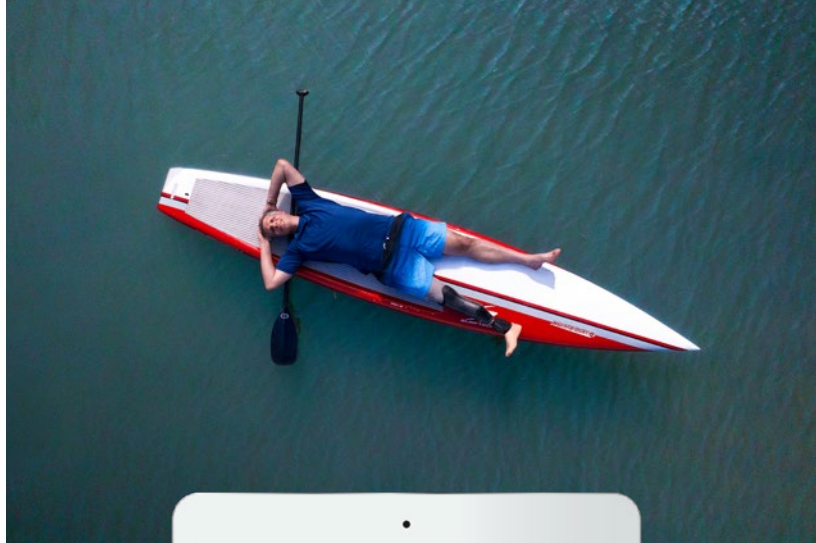

## Adjusting Heel Height


Put prosthesis in shoe in an upright position.  
Select cog image followed by Ankle Alignment.

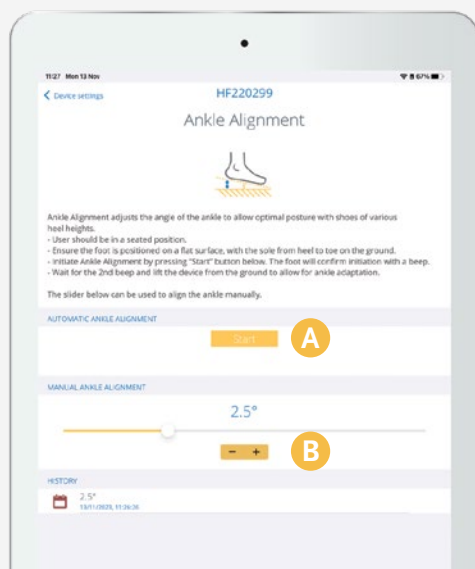

## Chair Exit / Relax Mode and Standby mode

Chair exit and relax mode can be switched on and off here


Use standby mode to eliminate all foot movement, e.g. when driving

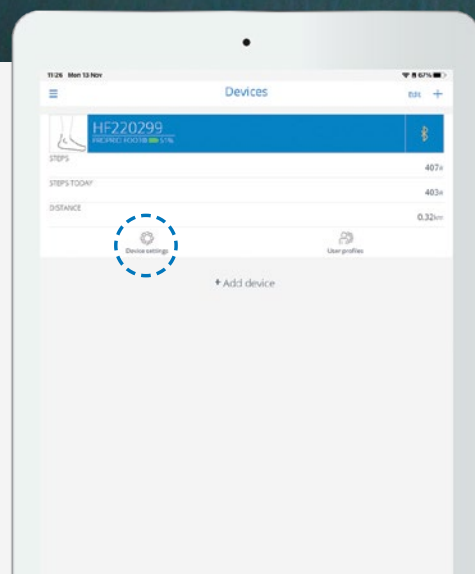

## Select 'Ankle Alignment'



- A** Select 'automatic ankle alignment'. Then wait for the foot to adjust to the new heel height
- B** After it has adjusted, you can also manually adjust the alignment with the slider.

**Alternatively**, you can press the button on the front of the foot twice, in quick succession. Only when the device has beeped for the third time, is the heel height aligned.

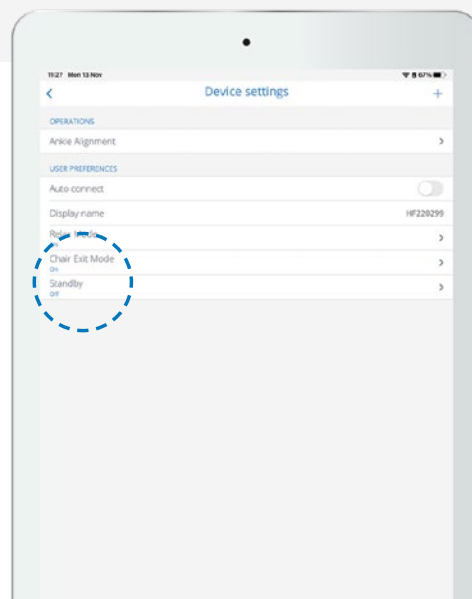

## How to store shoes on the app


Select the button shown on the right.

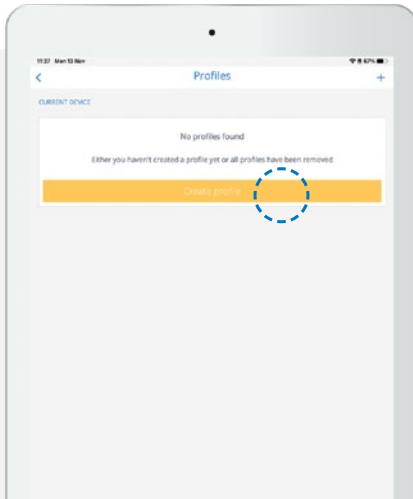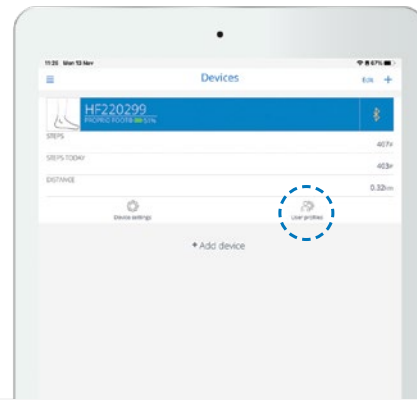

### Select 'Create profile'

This will enable you to store previous alignments of footwear for easy access and adjustment, whenever you change your shoes.

- A** Take a picture of the shoe (or barefoot)
- B** Add name and details for shoe profile
- C** Click on 'Store profile'

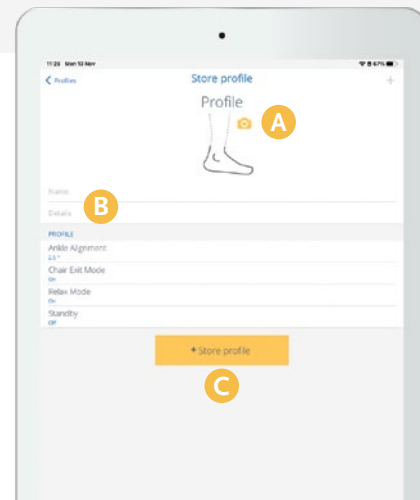

### Battery Life

To check battery life using the front button, push it once and each green light is equal to 10% of battery

FOLLOW ÖSSUR ON



USA (800) 233-6263  
WWW.OSSUR.COM  
CANADA (800) 663-5982  
WWW.OSSUR.CA

